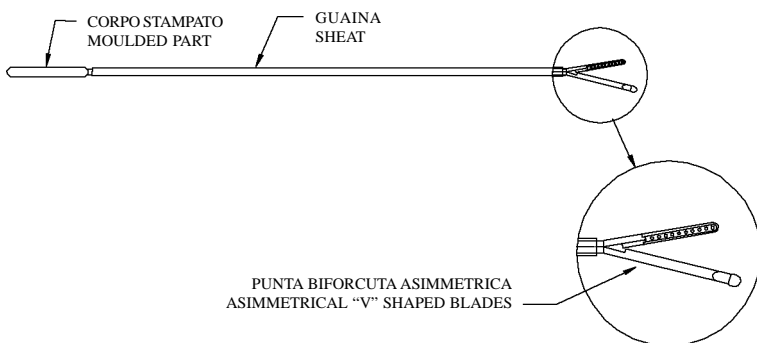
GIMA

CERVIRAM

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ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

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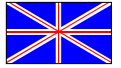
Instructions for use

Before starting the procedure, slide the outer sheath forward to cover the "V" shaped blades. Leave just the final small ball outside. Introduce CERVIRAM through the cervical canal into the uterine cavity and pull the sheath backwards to release the blades. Turn the device 360° leftwards and rightwards to scrape the uterine wall accurately. Slide the sheath forward to the original position once again. Take CERVIRAM out of the uterus. Release the blades and smear the specimen on a microscope slide or drop it into a test tube washing the device tip inside.

Caution: do not go beyond the depth measured by the hystrometer to avoid risk of perforation. Read the instructions before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose of after use according to the laws in force. For use only under trained medical staff.

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