# GIMA

**MANICO FLYER**  
***FLYER HANDLE***  
**MANCHE FLYER**  
***GRIFF FLYER***  
**MANGO FLYER**

**MANUALE D'USO E MANUTENZIONE**  
***USE AND MAINTENANCE BOOK***  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
***BETRIEBS UND WARTUNGS ANWEISUNGEN***  
**MANUAL DE USO Y MANTENIMIENTO**

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## LIGHTSOURCE FOR DIRECT CONNECTION TO ENDOSCOPES

**Warnings:** *Never shine direct into the eyes! Don't touch the hot lamp! Only use with heat resistant Endoscopes! No use for plastic fibres.*

**Switching On/Off:** by turning the cap (1) of the battery box.

**Battery change:** unscrew the cap (1) completely; insert the new batteries (4) with the (+)-pole first. Lifetime of batteries (size AAA): 0.5 hours.

**Attention:** When working with rechargeable batteries only use originals. Other types are in danger to be switched on without control. Never through any batteries in standard waste!

**Lamp replacement / Adapter change:** unscrew the adapter and the lamp (3). Only use original spare lamps!

**Cleansing:** First remove all dirt with a wet cloth; disinfection or autoclaving only after cleaning.

**Attention:** *the batteries have to be removed before disinfection or autoclaving. Batteries only have to be inserted into completely dry handless.*

### Technical data:

Voltage: 3V (2xbattery type AAA);

Lamp: 2.5 V / 700 mA Cod. 30796;

Adapter: type Storz Cod. 30840; Type Wolf Cod. 30841; type ACMI Cod. 30842.

## GUARANTEE

Thank you for having acquired our product.

This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, doppler probes, SpO<sub>2</sub> sensors, lamps, batteries, electrodes, handpieces, washers and packings, resistances, etc.

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
No compensation will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered. Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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GIMA S.p.A.