ARDEA

• CB860

• CB850

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## MANUAL CONTENT

Introduction.....	10
Classification and intended use.....	10
Suggestion for the use .....	11
How to open the wheelchair CB850.....	11
How to fold the wheelchair CB850.....	11
How to open the wheelchair CB860.....	12
How to fold the wheelchair CB860.....	12
Use on step/slope downwards.....	12
Brakes.....	13
Wheels .....	13
General warning.....	13
Cleaning and maintenance .....	14
Technical features .....	14
EC declaration of conformity .....	14
Guarantee .....	15
Warranty Certificate .....	15

## INTRODUCTION

Thank you for purchasing an Ardea wheelchair by Moretti. Its design and its quality guarantee to the user comfort, safety and reliability. The wheelchairs have been projected in Italy from the research and development department of Moretti S.p.A. Italia, in order to meet your requirements in inner and outer spaces. The material used, the accessories and the quality test are continuously checked by Moretti S.p.A. staff. Only the production is made in PRC according to the highest quality and safety standards in accordance with the European Directive on Medical devices (93/42/CEE). This instruction manual give you suggestion for a correct use of the device, together with safety advices.

We recommend to read carefully this instruction manual before using the wheelchair. In case of doubts about the use, please contact your dealer. The Ardea by Moretti wheelchairs are tested with a max loading weight of 75kg.

In case of doubts, please contact your dealer or :

**MORETTI S.p.A.** Via Bruxelles, 3 - Loc. Meleto - 52022 CAVRIGLIA  
Tel. 055 96 21 11 - Fax 055 96 21 200  
e-mail info@**moretti**spa.com  
Website www.**moretti**spa.com

## CLASSIFICATION AND INTENDED USE

The Moretti wheelchairs are in accordance with the European Directive on Medical devices (93/42/CEE) and belong to class I, C category . these wheelchairs, pertaining to International regulation UNI EN 12184 can be used in inner spaces only.

The wheelchair is realized for people with movement difficulty.

Moretti Spa declines all responsibility for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.

## SUGGESTION FOR THE USE

The Ardea wheelchairs are realized to be stable and safe. The patient on the wheelchair should not change the barycentre. Avoid to lean out of the wheelchair. If you really need to lean out, check that front wheels are facing ahead, for a better stability. If you need to take an object placed behind you, do not lean out too much, on order to avoid an overturning. We suggest to use safety belt or belt to fasten feet, to give more stability to the patient.

### HOW TO OPEN THE WHEELCHAIR **CB850**

- a. Lift up the backrest until levers block (pic. 1)
- b. Pull the footrest outward (pic. 2)
- c. Push the seat downward until it is completely opened (pic. 3)
- d. Push-down the armrests until 90° degrees

pic. 1

pic. 2

pic. 3

### HOW TO FOLD THE WHEELCHAIR **CB850**

- a. Lift up the armrests (pic. 4)
- b. Press backrest levers and push it downward (pic. 5)
- c. Pull up seat and footrest until the wheelchair is totally folded (pic. 6)
- d. Pull up the footrest (pic. 7)

pic. 4

pic. 5

pic. 6

pic. 7

### HOW TO OPEN THE WHEELCHAIR CB860

- a. Remove the wheelchair from the bag (pic. 1)

pic. 1

- b. Push on the seat (pic. 2)

pic. 2

- c. Lift up the backrest and lock the levers (pic. 3)

pic. 3

- d. Push down the armrest until 90° degrees

### HOW TO FOLD THE WHEELCHAIR CB860

- a. Unlock backrest levers and push it downward holding it from handles (pic. 4)

pic. 4

- b. Pull seat upward until the wheelchair is totally folded (pic. 5)

pic. 5

- c. Pull the footrest upward near to the foot belt (pic. 6)

pic. 6

### USE ON SLOPE

Before moving towards a slope downwards, be sure of one's own capacity and limits. Please check with qualified staff. The assistance is strictly recommended for slopes more than 10%. If needed, please stop at downwards avoiding abrupt movements. In order to slow down the chair do not use the wheels' handbrakes. Do not abruptly change direction.

## BRAKES

Stop wheels before getting in/out, stretching out or getting on lift. Pull completely lever up to stop chair. If correctly done, brakes press 5/6 mm against the rubber. Pls. check tyre pressure in case of inflate wheels. If needed, use the equipment key to unscrew nut while clamping bit forwards and backwards. Fix correctly nut to adjust brakes.

## WHEELS

The wheels must be periodically checked and lubricated if necessary. Remove the fork taking out the screw and lubricate the peg. Remove the wheel from the fork as well and lubricate the pegs. Assembly again the wheel and the peg and tighten the screws well. The front wheels are height adjustable in three positions.

## SOLID TYRE

Solid tires need maintenance: clean them with a damp cloth periodically. Once the tires are damaged or wear, change them. Check the pressure of the pneumatic tires periodically. It must be about 1,5/2,0 bar.

## GENERAL WARNING

The wheelchair cannot be used as a seat in any motor vehicle. Always transfer the patient from the wheelchair to the car seat.

Do not tilt the wheelchair more than 10% without any assistance.

Block the wheels each time you transfer the patient or when you are in an elevator. Before transferring the patient on the wheelchair or from the wheelchair to anywhere else, make sure that the wheelchair is on a plane and stable surface and that the wheels are locked. Before tilting forward or stand up make sure that the front wheels are straight and the rear wheels are locked. Do not lean out of the backrest and always keep the barycentre inside the perimeter of the wheelchair.

## CLEANING AND MAINTENANCE

Please periodically check the following:

Object	Frequency
Tires pressure	Monthly
Brakes	Monthly
Seat cleaning	If necessary

General check by qualified personnel each two years.

Clean the chromed and metallic parts with water and soap normally used for car wash.

The upholstery can be removed and cleaned with a damp sponge and neutral soap.

## TECHNICAL FEATURES

MODELLO													Ruote Anteriori	Ruote Posteriori
CB850	85	58	38	89	65	43	33	37	41	44	29	7	13	20
CB860	69	47	34	90	67	42	35	30	35	51	30	7	13	15

All the data can be modified according to production, without notice from Moretti.

## EC DECLARATION OF CONFORMITY

Moretti S.p.A declares under its exclusive responsibility that the products made and put into market by same MORETTI S.p.A , as part of the category of MANUAL WHEELCHAIRS are in conformity with the applicable disposition of the EEC DIRECTIVE 93/42 concerning MEDICAL DEVICE. To such purpose MORETTI S.p.A guarantees and declares under its own exclusive responsibility what follows:

1. The concerned devices meet the essential requisites as according to the 1° enclosure EEC Directive 93/42 as prescribed by the enclosure VII° of the aforesaid directive.
2. A complete list of the concerned devices is mentioned in the enclosure A of the present declaration.
3. The concerned devices ARE NOT INSTRUMENTS OF MEASURE.
4. The concerned devices ARE NOT NOT DESTINED FOR CLINICAL SURVEYS.
5. The concerned devices are sold IN A NON STERILE packing.
6. The concerned devices are to be considered as belonging to the 1° class.
7. Any use or installation of the concerned devices that differs from the destination of use, as it is foreseen by MORETTI S.p.A, IS NOT ALLOWED.

8. MORETTI S.p.A keeps and makes available to the competent authorities the technical documentation that proves the conformity to EEC Directive 93/42 .

### GUARANTEE

This device has 2 years warranty from the purchase date which is certified by receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge.

The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembling by un-authorized individuals.

In the case you need to get your device repaired and you fulfil the conditions of warranty, send the device to the nearest retailer together with the purchase receipt.

### WARRANTY CERTIFICATE

Product \_\_\_\_\_

Purchasing date \_\_\_\_\_

Authorized dealer \_\_\_\_\_

Street \_\_\_\_\_ Place \_\_\_\_\_

Purchased by \_\_\_\_\_

Street \_\_\_\_\_ Place \_\_\_\_\_

MORETTI S.p.A. - [www.morettispa.com](http://www.morettispa.com)

E-MAIL: [info@morettispa.com](mailto:info@morettispa.com)

MADE IN P.R.C.

**MORETTI S.P.A.**  
Via Bruxelles 3 - Meleto  
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11  
Fax +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)