Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)

**RP756S**  
• Fisso  
• Fixed

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MADE in P.R.C.

I Class Medical Device  
according to 93/42/EEC directive

**Codes RP754S - RP756S - RP756T**

Thank you for choosing a folding shoulder walker **MOPEDIA** by Moretti S.p.A. product range, manufactured to support people with movement difficulty. It is realised in chrome plated steel tubing with four swivel wheels two which of with brakes.

The **RP754S** and **RP756S** models are equipped with padded shoulder rests, adjustable height, width and depth.

The **RP756T** is equipped with padded table.

- **RP754S** ..... **foldable** - fixed in height
- **RP756S** ..... **foldable** with padded table - fixed in height
- **RP756T** ..... **foldable** - adjustable height every 2,5cm from 117 to 135 cm

**RP756S and RP754S assembling instruction**

1. Start assembling your walker with the pole of the back frame which has to be inserted in the tubes of the lateral frame and fixed with two brass buttons
2. Use the two horizontal poles to install in the back frame to strengthen the whole walker. Fix the poles with the four screw-buttons
3. Adjust the rests to the shoulder rest to the patient's desired height and fix them with the two lock system

4. The seat can be easily assembled or removed thanks to the hooks on the poles of the lateral frame. (**RP757 optional**)

**RP756T assembling instruction**

## VIEW OF THE PARTS

List of the parts

- A) Padded table with handles
- B) Handles lock knobs
- C) Table support poles  
(length 56 cm or 72 cm depending on the model)

## ASSEMBLY

1. Start assembling your walker with the pole of the back frame which has to be inserted in the tubes of the lateral frame and fixed with two brass buttons.
2. Use the two horizontal poles to install in the back frame to strengthen the whole walker. Fix the poles with the four screw-buttons.
3. To assembly the padded armrest, insert the poles (ref. C) into the support tubes, once they are at the same height block them with the levers on the walker.
4. Place the padded table (ref. A) and adjust the height, in order the user can rest his arms forming a 90° angle.
5. Adjust the handles at the desired opening and block them with the screw-buttons supplied with the padded table.
6. Assure that the screws are duly tighten.


7. The padded table is now correctly assembled and ready to use. As shown in (Fig. 2)

8. The seat can be easily assembled or removed thanks to the hooks on the poles of the lateral frame. (RP757 optional)

## TECHINCAL FEATURES

- Steel chrome tubular frame
- Rest table composed by foam rubber
- Leatherette cover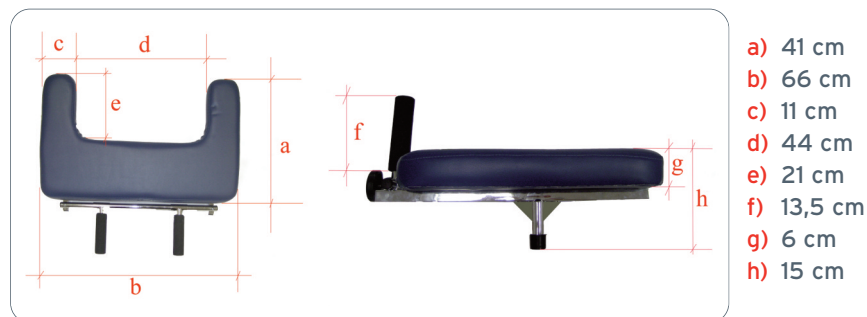

Temple length: (RP759/1) = 56 cm - (RP759/2) = 72 cm  
 Total weight inclusive temple length: 2 Kg

 **WARNING**

Beware of electrical cords in way, wet surface, rugs, toys or debris, furniture, little steps and gaps in floors. Do not use the walker on stairs or escalator.

 **WARNING! USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES**

Avoid to leave the walker in contact with water for long periods and check its sturdiness periodically. Keep it away from children to avoid any possible risks

## Spare parts

- **RV7076** Rubber wheels for walkers-pairs
- **RV7077** Rubber wheels for walkers with brakes-pairs
- **RV7090** Spare cover for antibrachium rests-pairs

## Accessories

- **RV7072** Padded shoulder rest-pairs
- **RV7089** Antibrachium rest-pairs
- **RP759-1** Padded table for RP753A - RP753S - RP754A - RP754S

## WARRANTY

All moretti products are guaranteed against defects in materials or workmanship for a period of two (2) years from the product purchasing date. Except for any limitations and exclusions specified below. This warranty does not apply in case of misuse, abuse or modification of the product and the non-adherence to the instructions for use. The correct use of the product is indicated in the present manual. Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disasters, not authorized maintenance or repair, damage due to power supply problems (where applicable), use of parts or components not supplied by moretti, non-adherence to the guidelines and instructions for use, unauthorized modification, shipping damage (other than original shipment by moretti), or from a non-service as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

## REPAIRS

### *In-warranty repair*

If a product moretti defects in materials or workmanship during the warranty period, moretti will evaluate with the customer if the product defect is covered by the warranty. Moretti, in its sole discretion, may replace or repair the item under warranty through a moretti dealer or from his location. The labor costs for repairing the product may be covered by moretti if the repair is under warranty. A repair or replacement does not renew or extend the warranty.

### *Repair of a product not covered by warranty*

A product not covered by warranty can be returned for repairing, only after receiving authorization from moretti customer service. The labor and delivery costs for a repair not covered by the warranty will completely be at customer or reseller charge. Repairs on products not covered by the warranty are guaranteed for 6 (six) months starting from the day you receive the repaired product.

### *The non - defective - products -*

The customer will be notified, in case moretti, after having examined and tried a returned product, claims it is not defective. The product will be returned to the customer. In this case shipping cost will be at customer's charge.

### *Spare parts*

The original moretti spare parts are guaranteed for 6 (six) months from the date you receive them.

## DISCLAIMER

Except as expressly stated in this guarantee and within the limits of the law, moretti doesn't offer any other declarations, warranty or condition, express or implied, including any declarations, warranties or condition of merchantability, fitness for a particular purpose, non infringement and non interference. Moretti don't warrant that the use of the moretti product will be uninterrupted and without errors. The duration of any implied warranties that may be imposed by law is limited to the warranty period, within the limits of the law. Some states or jurisdictions do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, some exclusions or limitations of this warranty may not apply to you. This warranty is subject to change without notice.

## WARRANTY CERTIFICATE

Product \_\_\_\_\_

Purchasing date \_\_\_\_\_

Authorized dealer \_\_\_\_\_

Street \_\_\_\_\_ Place \_\_\_\_\_

\_\_\_\_\_

Purchased by \_\_\_\_\_

\_\_\_\_\_

Street \_\_\_\_\_ Place \_\_\_\_\_

\_\_\_\_\_

**MORETTI S.p.A.**

INTERNET: [www.morettispa.com](http://www.morettispa.com) E-MAIL: [info@morettispa.com](mailto:info@morettispa.com)

MADE in P.R.C.