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EN - Instruction manual

Packaging and Components

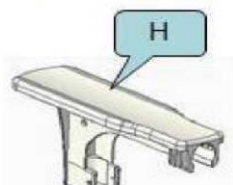
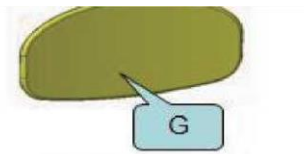
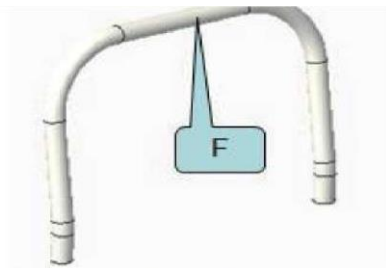
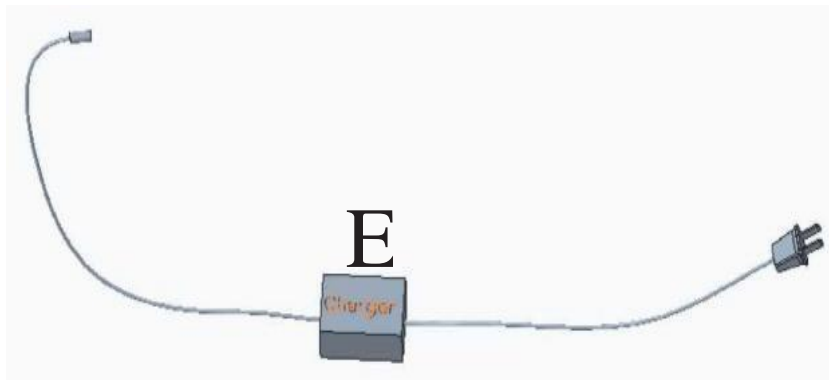
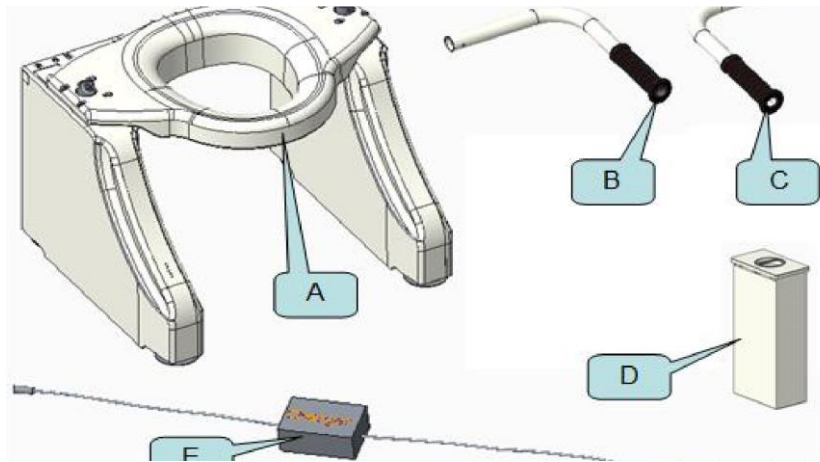
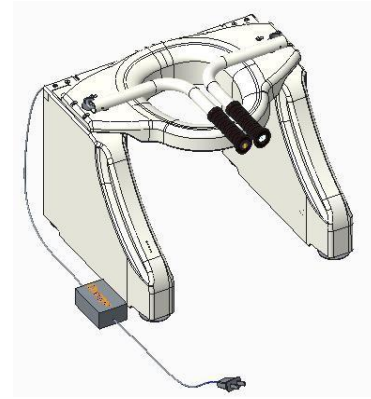
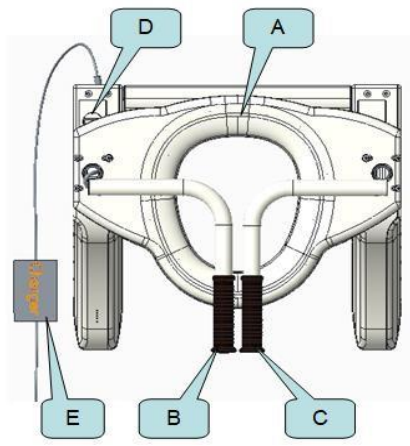
Check all components after unpacking, please contact the supplier if they are missing or damaged.

Package list:

- Main part A: Toilet lift seat base, 1 unit
- Handrail B (with control switch) 1 unit; Handrail C (with SOS alarm button) 1 unit; Battery D (inserted in main part of base A) 1 unit
- Power supply E (to charge battery and power supply) 1 unit
- Backrest support F 1 unit
- Backrest G 1 unit
- Arm support H 2 units
- Installation tool bag

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NOTE: The installation positions (left or right) of handrail B and handrail C are optional, the right hand control is the default installation, the left hand control can be customized.

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Battery installation

Fully charge the battery before using it for the first time, full power will lift it over 100 times.

1. Insert the charger interface vertically into the charging port of the lithium battery and switch it on (Fig. 1)
2. The charger's indicator light will glow red to show that it is charging
3. When the lithium battery is fully charged, the charger indicator light will be green
4. Insert the lithium battery into the battery compartment of the device (as shown in figure 2)
5. The power display is located in the position shown in Figure 3. To save power, the battery is only displayed while the unit is in operation. Referring to the power comparison indicator, the lithium battery should be charged when only the red light is on.
6. Lift the lithium battery after lifting the seat of the device, otherwise the lithium battery may not be removed properly.

It is recommended to buy a spare lithium battery. When using a lithium battery, you can charge another lithium battery and replace it regularly.

ATTENTION: Make sure that the surface of the battery is not covered by anything, charging in the bathroom or any other wet place is not allowed. Dry your hands during the charging process. Only the charger supplied by the manufacturer is allowed to charge the battery.

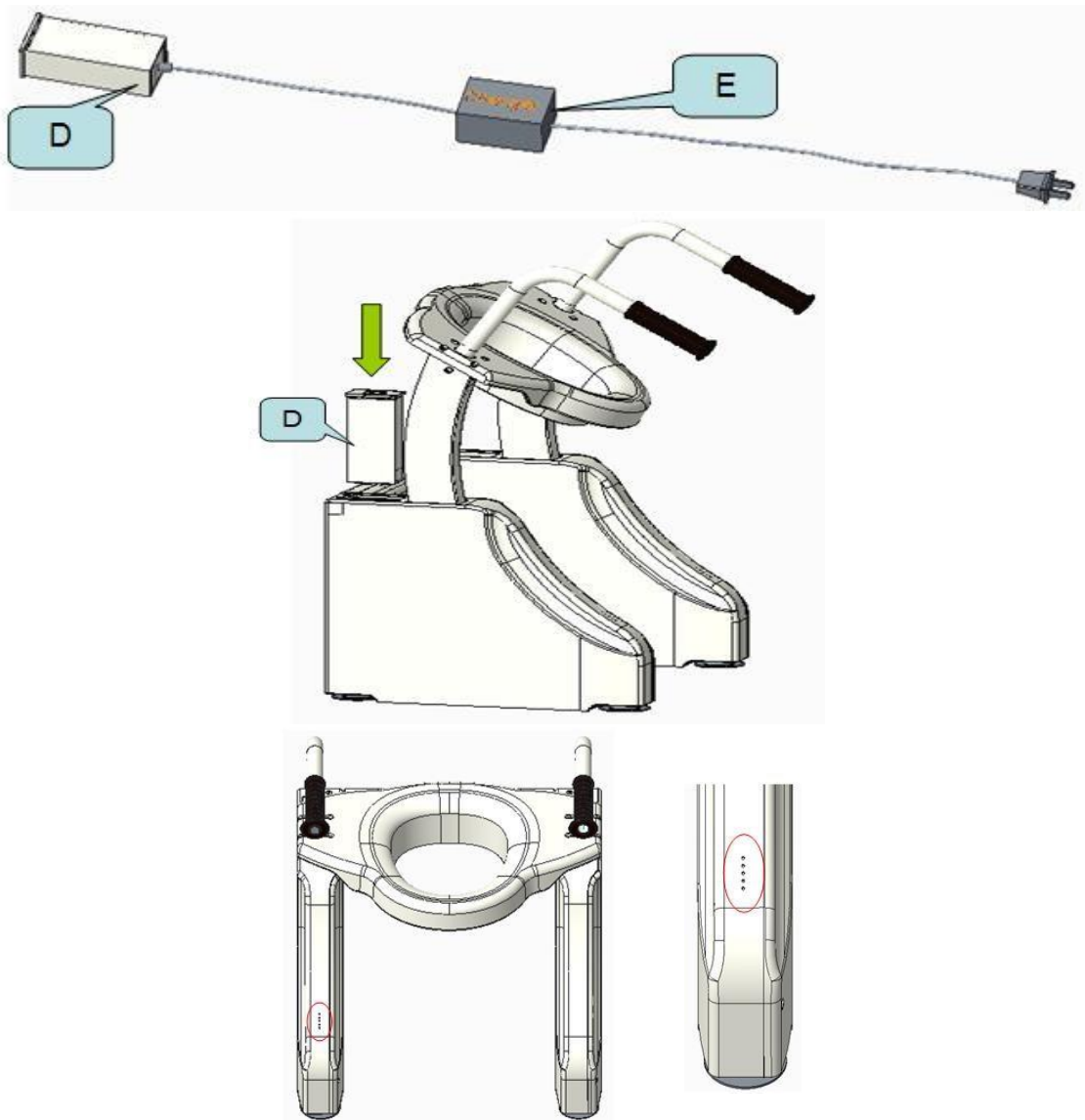
Figure 3 Power indicator

Installing the handrail

Installing the handrail See figure 4, screw the armrest base to the body of the auxiliary lift base A.

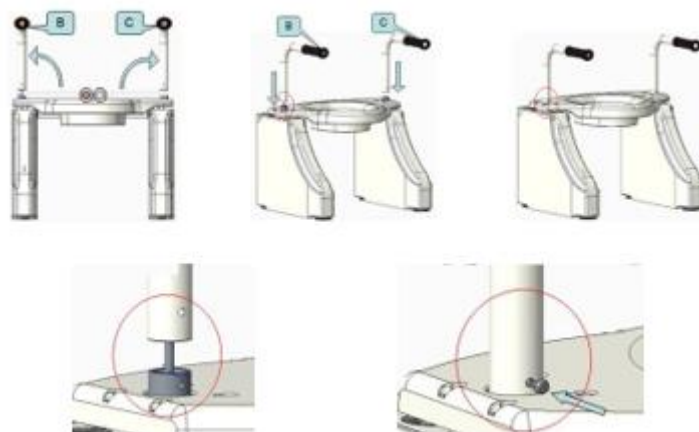
Figure 4 Installation of the handrail

Note: Do not pull during the installation of the handrail B with the control switch circuit to avoid damage to the circuit.

Installation du dossier

Refer to figure 5 for the installation of the backrest. Prepare the two openings of the bracket at the same time with the square tubes, then press the middle of the upper part of the bracket, push the screw into the hole, and press the backrest cushion against the backrest. It can be on the support.

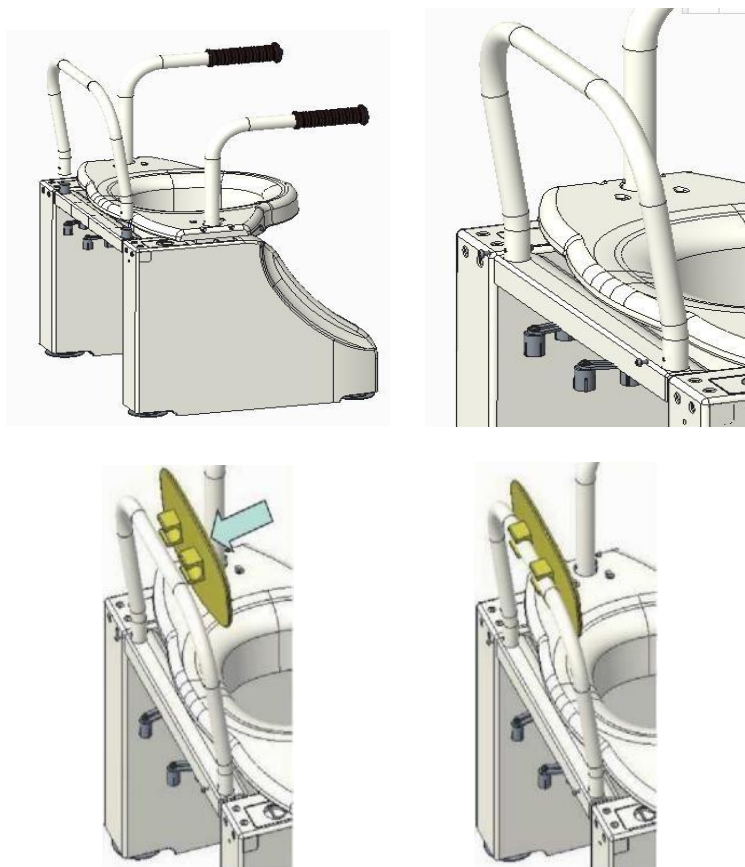
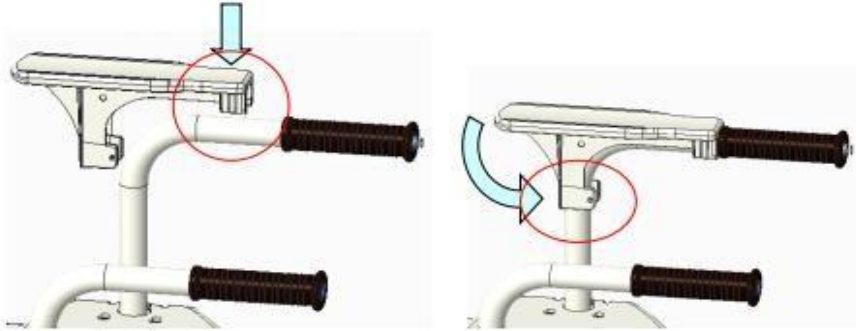
Figure 5 Installation of the backrest

Installing the armrest

For the installation of the armrest, see figure 6. Press the armrest onto the armrests on both sides and lock it with the screw nut.

Leg adjustment

4 support legs (can be replaced by 4 wheels with brakes) are set to the minimum height before delivery. Rotate the support legs (figure 8) to change the height of the equipment, avoiding contact between the equipment and the toilet.

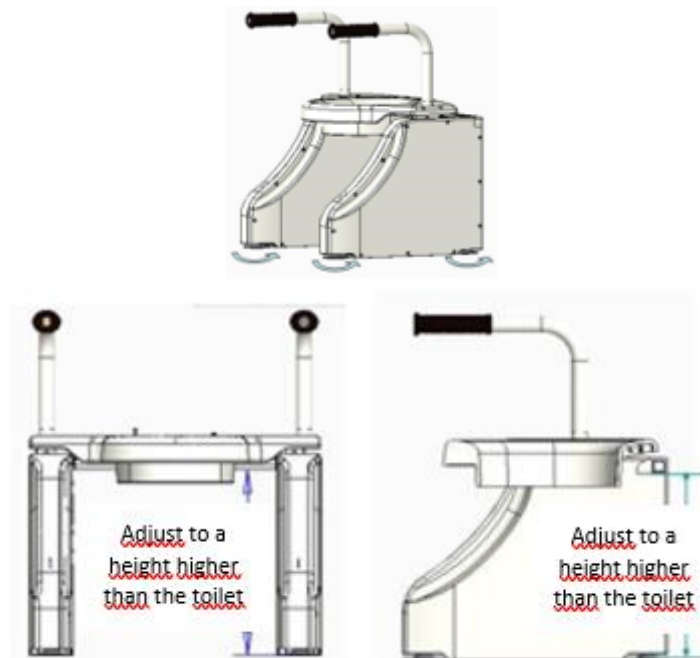
Figure 9: Confirm the height of the toilet seat

Attention : Adjust the 4 legs to make the equipment stable, and make sure that the lid surface is in a horizontal position. At the same time, make sure that the outermost lower edge of the seat of the equipment is higher than the upper edge of the ceramic base of the toilet. Otherwise, the equipment may function abnormally.

Fixation with the toilet

1. Remove the toilet cover and the fixing screws, only the ceramic base of the toilet remains.
2. Lift the appliance (seat in the lowest state) and move it up the toilet base and lower it slowly to make sure that the seat of the appliance does not touch the toilet base.
3. Check the position of the machine on the toilet, to avoid the machine colliding with the toilet during use.
4. For users with poor body balance, the crossbar of the main part A of the equipment should be fixed to the toilet with bolts (as shown in Figure 10, Figure 11).

Fixing method 1: (directly fixed with bolts)

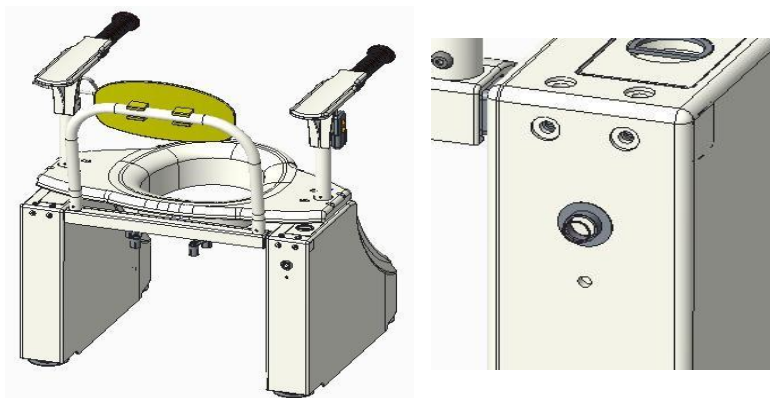
Figure 7: The equipment is attached to the toilet with bolts

Fixing method 2: (For a special position of the toilet bowl, the distance must first be adjusted by an extension piece and then fixed).

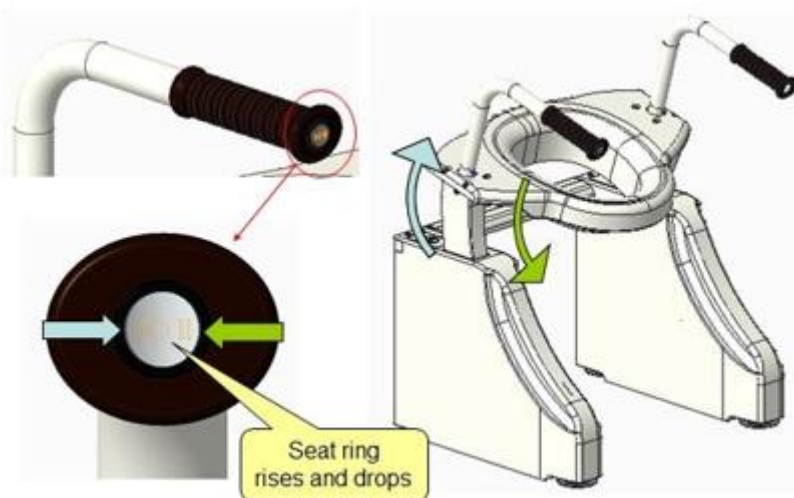
Figure 11 Fixing the equipment with the toilet after adjusting the distance d'extension.

Hand-operated switch

The control switch for the device is located on the front of the handrail B, users can press the left and right buttons continuously to control the raising and lowering of the device (as shown in Figure 13). The equipment stops operating when the switch is released.

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Figure 13: Operating the switch

The SOS alarm button is located on the front of the C-handrail. When the user needs help in an emergency, simply press the SOS button and the equipment will sound an SOS alarm. Press the button again to silence the alarm when the rescue workers have arrived.

Figure 14 Using the SOS alarm button

Charger Manual

1. Charging the battery
Insert the charger plug into the charging port of the battery and connect the power supply (as shown in Figure 16). The charger indicator will turn red during charging and will turn green when the battery is full.

Figure 15 Battery charging diagram

2. Operating the device
 Insert the charger into the unit's power supply interface and connect the power supply (as shown in Figure 17). The battery must be inserted into the battery box.

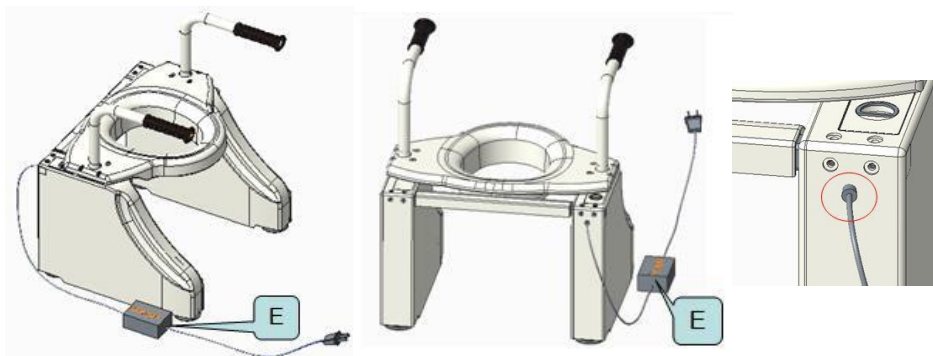
Figure 16 Alimentation de l'appareil

Product dimensions

Product dimensions The product dimensions are shown in figure 18

1. Width: 606 mm
2. Length: 525 mm
3. Height: 795 mm
4. Distance from handrail: 456 mm
5. Distance from the base of the machine: 422 mm
6. Minimum seat height: 481 mm

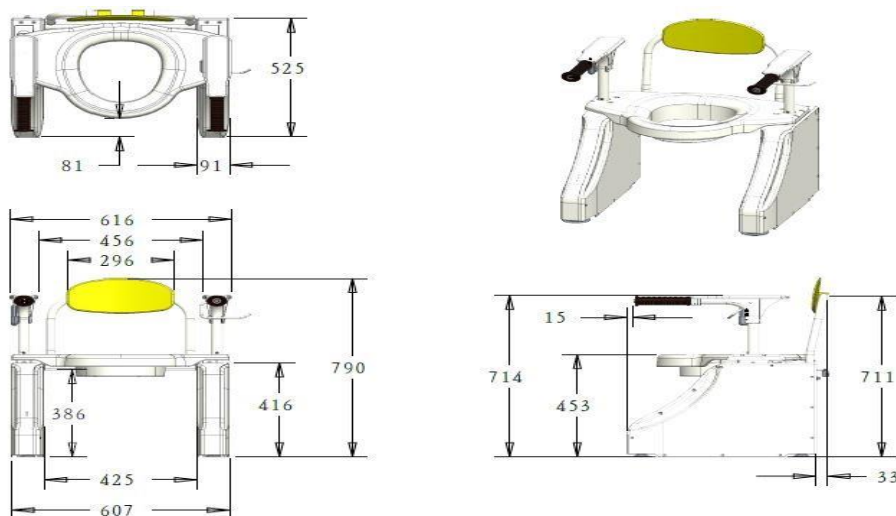
Figure 11: Product dimensions

Technical data

Technical specifications

Maximum load: 200kg

Power: 85V~265V

Rated battery output voltage: 24V

Operating voltage: 24V

Rated product power : 130w

Waterproof level: IP44

Cleaning the equipment

Clean the equipment with mild, non-abrasive sanitary products, if the equipment is used by more than one person, the equipment should be disinfected each time. The equipment is not waterproof, do not clean it with a manual cleaning device, do not spray the equipment or immerse it in water.

Repairs and maintenance

This product offers a worldwide warranty, 1 year warranty for the whole device, lifetime maintenance services.

This device (excluding the battery) can be used more than 10,000 times if the user uses and takes good care of the equipment according to the user's manual

All the components of the device are not allowed to be repaired by the user himself, contact the authorized dealer for the repair.

Warning: Children are not allowed to use or play with this product.