


ACORES flip-up shower seat

REF 812298

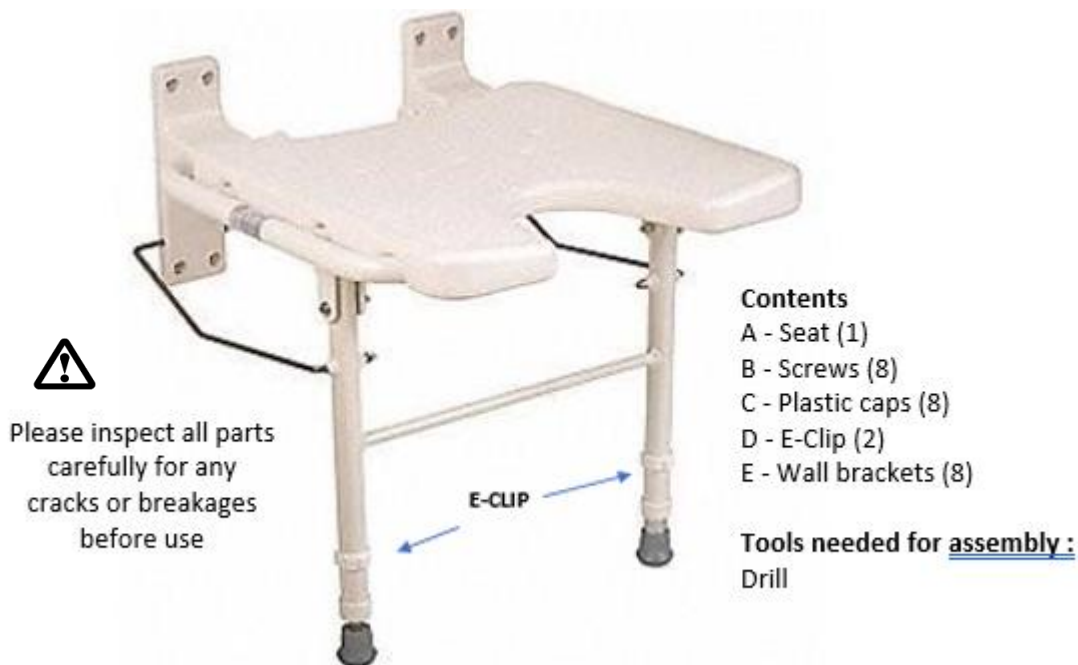

Assembly instructions:

Adjust the height of the stand to achieve a comfortable position for the user. Fix the height by placing the E-clip in the height adjustment groove.

Locate the wall uprights and place at least one bracket in a wall upright.

Mount the rear brackets to the wall using the screws.

CAUTION :

- Do not stand on the seat.
- Make sure that all screws are securely fastened to the wall.
- The legs must always be locked in the same height position.
- Make sure that the legs are completely clear of the wall.



CAUTION :

- ***Do not stand on the seat.***
- ***Make sure that all screws are securely fastened to the wall.***
- ***The legs should always be locked in the same height position.***
- ***Make sure that the legs are completely clear of the wall.***



Persons with severe disabilities or persons unable to walk unaided should not attempt to use this product without assistance.



Ensure that all clips are locked in place to ensure a secure, locked position.



Maximum supported weight :