**EASYRED®**

MA EM52x-541 01 A 01 A\_01-2014

**CINTURE**  
**MANUALE D' ISTRUZIONE**

---

**BELTS**  
**INSTRUCTION MANUAL**

## Indice

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**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

**www.morettispa.com email: info@morettispa.com****MADE IN P.R.C**

INDEX

1. Codes .....	pag.8
2. Introduction .....	pag.8
3. Intended use.....	pag.8
4. Ec declaration of conformity.....	pag.8
5. General warnings .....	pag.9
6. Before use .....	pag.9
7. Warning for use.....	pag.9
8. How to use.....	pag.9
9. Maintenance.....	pag.9
10. Cleaning and disinfection .....	pag.9
11. Conditions of disposal.....	pag.9
11.1 General condition of disposal .....	pag.9
12. Caratteristiche tecniche .....	pag.10
13. Warranty .....	pag.10
13.1 Repairs.....	pag.10
13.2 Spare parts.....	pag.10
13.3 Exempt clauses.....	pag.10

I Class Medical Device  
according to Directive 93/42/EEC and further modifications

### 1. Code

- EM521 - Belt set pressure buckle and lock type connection
- EM526 - Belt set pressure buckle and plastic-snap lock type connection
- EM528 - Belt set pressure buckle and car type buckle
- EM541 - Spider straps

### 2. Introduction

Thank you for purchasing a EASYRED by Moretti S.p.A. device. These products are realized for patient immobilization during transport. They are made of polyester fiber, they are adjustable and they are available with different connections.

The spider straps is realized for patient immobilization during transport with spine board stretcher.

This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

### 3. Intended use

Device for immobilization during rescue operation.

#### WARNINGS!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

### 4. EC Declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the cane aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.

To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

### 5. General warnings

- READ carefully the entire manual before using the device .
- The product should be used only by qualified personnel
- Make sure that the product is delivered complete and keep it packed away from heat sources
- SERVICE LIFE- the device use limit is defined by the wear of the parts
- Store in a dry place
- DO NOT allow children to play or operate the device

## 6. Before use

- Always check the state of wear of the parts in such a way as to ensure it is used in complete safety for persons and property

## 7. Warnings for use

- Check the length of the belts during transport, cause the belt can unloosen.
- Take special care in fractures.
- Periodically check its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out .
- In case of doubts about use, adjustment or maintenance, contact your dealer
- Keep it away from children to avoid any possible risks.

## 8. How to use

### Standard belts

- Unfasten the belt
- Insert the belt in the holes of the stretcher and adjust the length according to the patient.

### Spider straps

- Place the patient on the spinal board.
- Lay the spider strap down the patient body; place the Y-shaped part round the head and adjust the other belts.

## 9. Maintenance

The EASYRED devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

## 10. Cleaning and disinfection

Use water and a mild disinfectant, dry away from direct sunlight.

Note Never use acids, alkalis or solvents such as acetone

## 11. Condition of disposal

### 11.1 - General condition of disposal

General conditions of disposal. In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

## 12. Technical features

Code	Specifications
EM521	Material polyester fabric - 180cm x 5h - pressure buckle and lock type connection
EM526	Material polyester fabric - 180cm x 5h - pressure buckle and snap lock type connection
EM528	Material polyester fabric - 180cm x 5h - pressure buckle and car type connection
EM541	2-color fabric - 5 cm height

### 13. Warranty

Moretti products are guaranteed from material or manufacturing faults for a period of ( two ) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product , and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage ( other than original shipment by Moretti ), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

#### 13.1 Repairs

Warranty repair: In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

Repair not covered by warranty: A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non defective device: The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer .

#### 13.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

#### 13.3 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference.

Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

## WARRANTY CERTIFICATE

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN P.R.C**

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)