MOPEDIA®

MA MP165_MPI67_01 A ITA_12-2015

SPONDE UNIVERSALI

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ITALIANO

INDICE

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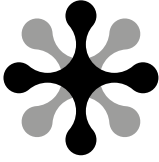
 Class I medical device
 according Directive 93/42/EEC and subsequent amendments

1. CODES

MP165 - Aluminum bedside rails with support on the ground

MP167 - Aluminum reversible telescopic bedside rails

2. INTRODUCTION

Thank you for choosing an accessory MOPEDIA by Moretti. The bedside rails have been designed and built to meet all your needs for a practical use, correctly and safely. This manual contains some suggestions for proper use of the accessory of your choice and valuable tips for your safety. You should read carefully all of this manual before using the bedside rails.

CAUTION!

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.
- Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.
- Always pay attention to the presence of children.

3. INTENDED USE

The bedside rail is a universal accessory designed to prevent an accidental fall of a person from a any bed. It may provide a useful support to facilitate the raising and the seat from the bed and can be placed, optionally on both sides of the bed.

4. EC DECLARATION OF CONFORMITY

Moretti S.P.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the support aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive.
2. The full list of the devices at issue are specified in Annex A to the declaration.
3. The devices in question ARE NOT MEASUREMENT DEVICES.
4. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
5. The devices in question are sold in NON STERILE packing.
6. The devices in question should be considered as belonging to class I.
7. DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
8. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications.

5. GENERAL WARNINGS

- For proper use of the device refer to this manual.
- For proper use of the device to consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts can not be repaired and/or replaced.
- Always pay attention to the presence of children.
- The bedside rail is not intended to carry the full weight of an individual. This product is only intended to provide balance and support while sitting and standing

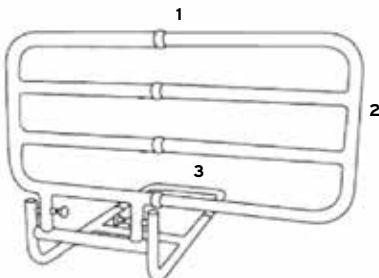
6. GENERAL DESCRIPTION

The universal bedside rails are designed to fit any bed thanks to the telescopic tubes to be placed under the mattress.

6.1 MP165 parts list

1. Handle upper (pc.1)
2. Frame (pc.1)
3. Cross-member connection (pc.1)
4. Legs (pcs.2)
5. Utility pocket (pc.1)
6. Security strap (pc.1)

6.2 MP167 parts list

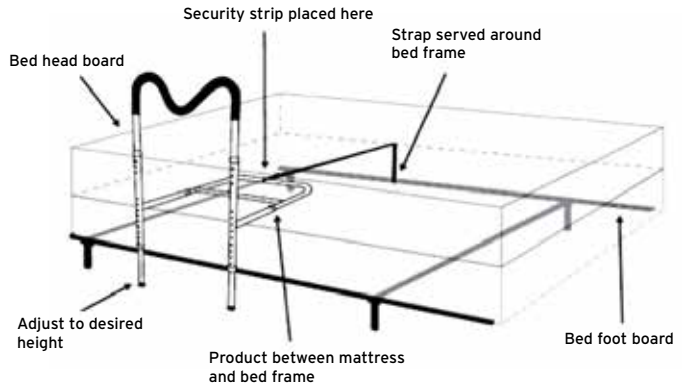
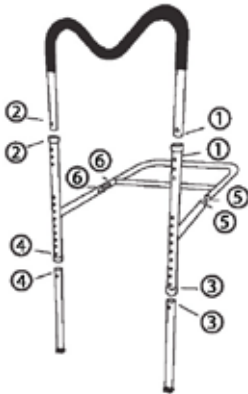
1. Handle telescopic upper (pc.1)
2. Frame (pc.1)
3. Cross-member connection (pc.1)
4. Security strap (pc.1)

7. INSTALLATION/REMOVAL

7.1 Assembly MP165



Follow the below instructions:

1. Insert the handle-M on the frame, adjust the height by manipulating clips and fit the utility pocket. (1-1 and 2-2).
2. Place the legs on the frame by adjusting the length of the legs depending on the height of the bed by manipulating clips. (3-3 and 4-4) Use the adjustment screw on the bottom for a perfect contact with the floor.
3. Insert the Cross-member connection connecting the main frame. (5-5 and 6-6).
4. Place the assembled structure between the mattress and the bed frame.
5. Tighten the belt to the bed frame on the opposite side on the bed.

7.2 Assembly MP167

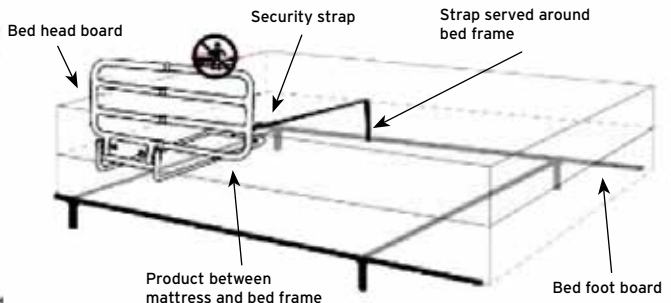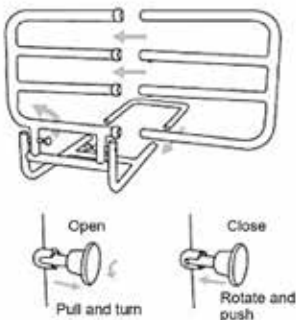1. Insert the cross-member connection in the main frame using the clips.
2. Insert the telescopic tubes on the frame and adjust the width of the universal telescopic bedside rail.
3. Place the assembled frame between the mattress and the bed frame.
4. Tighten the belt on the bed frame on the opposite side of the bed.

7.3 Dismantling

Follow the procedure described above in reverse.

8. BEFORE TO USE

- Always check the wear of the mechanical parts in the game so as to ensure use of the device in total safety for persons and things.
- Double check for proper assembly of the device and pay attention fixing the bedside rails on the bed frame.

9. HOW TO USE

MP165

Assembly the device according to instruction manual, adjust the height bedside rail to suit your needs by clips. Place the bedside rail between the mattress and the bed frame.

NOTE: Always check the bedside rail is in the locking position.

MP167

Assembly the device according to instruction manual, adjust the width of the telescopic bedside rail to suit your needs by clips. Place the bedside rail between the mattress and the bed frame. To up and down the bedside rail use the black knobs. Pull them outward to release the bedside rail, then lift it or lower it.

NOTE: Always check the bedside rail is in the locking position.

10. MAINTENANCE

The MOPEDIA devices by Moretti S.P.A. are checked carefully and supplied with CE mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

11. CLEANING AND DISINFECTION

11.1 Cleaning

Use only a damp cloth and mild soap. Afterwards dry thoroughly before using.

11.2 Disinfection

If you need to disinfect the product using a common disinfectant.

NOTE: Never use acids, alkalines or solvents such as acetone or thinner

12. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

13. SPARE PARTS AND ACCESSORIES

For spare parts and accessories, refer to the MORETTI master catalogue. Only use original MORETTI accessories. Failure to do so will entail our refusing all liability for any incident caused by improper use.

14. TECHNICAL FEATURES

14.1 MP165 Universal bedside rail with support on the ground

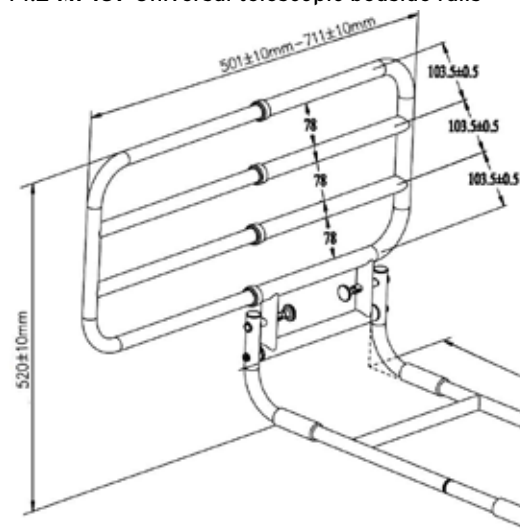
Dimension

- Leg height adjustable from ground from 34 to 52 cm
- Bedside rail height adjustment from bed from 47 to 52 cm
- Width 45 cm
- Length 64 cm

Technical specifications

- Frame anodized aluminium
- Handle support polyurethane adjustable in 3 levels of height, useful support to facilitate the get up and the seat from the bed
- Height adjustable leg in 9 positions
- The stabilizing frame is to be inserted under the mattress and is provided with thicknesses soft polyurethane anti-slip of fixed length

14.2 MP167 Universal telescopic bedside rails

Dimension

- Height from the bed frame 50 cm
- Width adjustment from 50 to 71 cm
- Length 70 cm

Technical specifications

- Frame anodized aluminum
- When not in use it may be folded in the lower part of the bed
- Useful support to facilitate the get up and the seat from the bed
- The stabilizing frame is to be inserted under the mattress and is provided with thicknesses soft polyurethane anti-slip of fixed length

15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device

16. REPAIRING

16.1 Warranty repair

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

16.2 Repair not cover by warranty

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

16.3 Non defective device

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

17. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

18. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

 MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com