GARANZIA

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WARRANTY

This forearm crutche has **2-YEAR** warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In the case you need to get your forearm crutche repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

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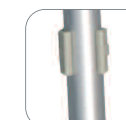
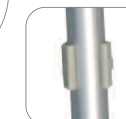
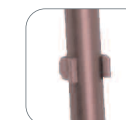
FOREARM CRUTCHES INSTRUCTION MANUAL

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I class medical device according
to EEC directive 93/42

Code RP701 (Available colours: silver, blue, red, green, black)

Thank you for purchasing forearm crutch, **MOPEDIA** by Moretti manufactured to support people with movement difficulty. It consists of lightweight anodised aluminium tubing, anatomic non-slip handle to give you comfort and security, and non-slip soft rubber tips. It is height adjustable every 2,5 cm:

- RP701 (all models) _____ from 74 to 97 cm

To adjust height:

To adjust the height press the push button and slide the cane to the desired height. Be sure the push button is fully protruded through the desired hole when finished. Maximum endurable weight 100kg.

ATTENTION: USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES

Avoid to leave the forearm crutch in contact with water for long periods and control its sturdiness periodically. Keep it away from children to avoid any possible risks. Replace rubber tips every time they are worn out.

• Blu
RP701B

• Verde
RP701V

• Argento
RP701A

• Rosso
RP701R

• Nero
RP701N

• Blue
RP701B

• Green
RP701V

• Silver
RP701A

• Red
RP701R

• Black
RP701N