MOPEDIA®

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CE I Class Medical Device

according to Directive 93/42/EEC and further modifications

1. Code

RC220-43 Commode and folding chair on castors - 43 cm

2. INTRODUCTION

Thank you for purchasing a **MOPEDIA** commode chair by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

The commode chairs are an aid for evacuation and mobility for people with walking difficulties.

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the bath aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

5. WARNINGS

5.1 General warnings

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Keep the packed bath chair away from heat sources
- **SERVICE LIFE**- the product use limit is defined by the wear of the parts
- **ALWAYS** keep hands and fingers clear of moving parts to avoid injury

- DO NOT allow children to play on or operate the bath chair

5.2 Warnings for use

- Make sure the tank has not been moved or removed
- Do not lean on the legrests to sit down or get off the commode chair
- Do not lean out from the commode chair to reach items on furniture or on other shelf
- Avoid sitting on the edge to prevent the convenient one can overturn

6. GENERAL DESCRIPTION

RC220-43 Seat L 44x43 W cm
FOLDABLE COMMODE CHAIR ON CASTORS

7. ASSEMBLING

The MOPEDIA commode chair is provided with backrest and footrest disassembled. Open the frame acting on the outer tubular blocking the frame by the seat using the specific clips.

Fit the padded seat. For the assembly of the backrest is sufficient to put the two ends of the tubes into the holes provided in the back of the chair. Then fix backrest using clips provided. The footrest must be placed outside the chair by inserting two pins in the upper part of the footrest. The footrest must always be within the chair. To secure the footrest to make sure that the lever locking is secure within the chair.

7.1 Legrest adjustment

The legrest of the commode chairs can be adjusted according to the height that offers the best comfort loosening the push button. Always check that the two legrest are at same height.

7.2 Disassembly

Follow backward the assembly.

8. BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things;
- Check the correct assembly of the device.

9. WARNING FOR USE

- The commode chair is for internal use only, can not be used as a seat on motor vehicles;
- To push the commode chair, always use only the rear handle, Never push using other supports;
- Before moving on an inclined surface to make sure of their abilities and limitations.
Consult movement techniques with qualified personnel;
- It 'still recommended care assistance for land with a slope greater than to 10%. If you need to stop on an inclined surface, avoid sudden movements;

- If two people raise the commode chair, avoid contesting in points dangerous for the patient as footrests and armrests, but use only the frame and rear handles.

9.1 Brakes

Lock the wheels of the chair before you ascend, descend, stretch forward. The brakes are operated by pressing the lever on all wheels of the chair (**Fig. 1**).
Image for illustrative purpose only.

Fig. 1

9.2 Wheels

The wheels must be periodically checked and lubricated if necessary.

10. HOW TO USE

Note: make reference to “GENERAL WARNINGS” chapter

WARNING!

- Getting in and out of the commode chair may be performed independently, provided you have a adequate mobility and upper body strength
- Do not stand on removable parts **DO NOT STAND ON THE FOOTRESTS WHEN GETTING IN OR OUT OF THE COMMODE CHAIR, OVERTURNING RISK!!!**

10.1 Get in the commode chairs

1. Position the commode chair as close as possible to you
2. Engage wheel locks (only for models with castors). We suggest to remove the armrest to ease the patient getting in the commode chair
3. Lean on the frame with arms and slowly go down towards the seat

10.2 Get out of the commode chairs

1. Position the commode chair as close as possible along side the seat to which you are transferring
2. Engage wheel locks (only for models with castors). We suggest to remove the armrest to ease the patient getting out of the commode chair
3. Shift body weight into seat with transfer

11. MAINTENANCE

The MOPEDIA devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair use only original spare parts. Before using follow the “WARNINGS” chapter. Moretti MOPEDIA commode chairs require few elementary and periodic maintenance, below we list the checks to be carried out periodically.

Element	Frequency
Braking capacity	Monthly
Seat cleaning	As needed
General inspection by qualified personnel	Biennial

12. CLEANING AND DISINFECTION

12.1 Cover and padding

The cover and padding are in Nylon. Clean with warm water and mild soap.

12.2 Toilet cleaning

The toilet must be regularly cleaned and disinfected using

12.3 Frame and metallic parts

Clean with warm water and mild soap. Dry with a soft cloth in order to not scratch the painting.

12.4 Brakes and wheels

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous use the wheelchair with wet wheels.

12.5 Disinfection

If you need to disinfect the device use a common disinfectant cleaner.

Note: Do not use solvent and abrasive product

13. CONDITIONS OF DISPOSAL

13.1 - General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

14. TECHNICAL FEATURES

14.1 Dimensions and weight

Dimensions and weight												
Measures in CM												
RC220-43	95	60	44	91	75	52	42÷52	43	36	53	12,5	12

14.2 Technical specification

Frame	Powder coated steel
Fabric	PVC
Seat	Plastic
 Max weight	110 kg
Environmental temperature	- 20 C° / + 40 C°
Storage	Store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C)
Wheels	ø 12 cm

15. SPARE PARTS/ACCESSORIES

For spare parts only refer to MORETTI master catalogue.

16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17. REPAIR

17.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty.

Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

17.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

17.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

18. SPARE SPARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

19. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE****Product** _____**Purchased on (date)** _____**Retailer** _____**Address** _____ **Town/city** _____**Sold to** _____**Address** _____ **Town/city** _____

Via Bruxelles, 3 - Melegnano 20122 Cavriago (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C.**

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

MORETTI S.p.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com