MOPEDIA®

MA RS809-815-817-827-829-831-833-834-838-839 01 A 01 A_12-2015

AUSILI DA BAGNO

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INDICE

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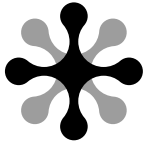
CE I Class Medical Device according to Directive 93/42/EEC
and further modifications

1. CODES

RS809	Plastic bathtub seat with handle
RS815	Foldable bath bench
RS817	Bath stool
RS827	Bath bench without backrest
RS829	Bath bench with backrest
RS831	Bath bench u-shape without backrest
RS833	Bath bench u-shape with backrest
RS834	Transfer bath bench
RS838	Shower chair with armrest and without backrest
RS839	Shower chair with armrest and backrest
RS908	Toilet safety rails painted steel height adjustable and removable
RP770	Commode frame in painted aluminum, height adjustable

2. INTRODUCTION

Thank you for purchasing a MOPEDIA bath products by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE: Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

The bath chair by Moretti is intended for disabled patient assistance in bath tub

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the bath aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.

To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive
2. A complete list of the concerned devices is mentioned in the enclosure A of the declaration

3. The devices in question ARE NOT MEASUREMENT DEVICES
4. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION
5. The devices in question are sold in NON STERILE packing
6. The devices in question should be considered as belonging to class I
7. DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
8. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

5. GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Keep the packed bath chair away from heat sources
- SERVICE LIFE- the product use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the bath chair

6. GENERAL DESCRIPTION

6.1 RS809

Plastic bathtub seat with anti-slip plastic handle and holes for the water. Adjustable width for better adaptability.

6.2 RS815

Folding bath chair with painted steel structure and plastic seat, folding to save space during storage. Non-slip rubber tips.

6.3 RS817

Round shape bath seat. With anodized aluminum frame.

6.4 RS827-RS829

Bath seats with or without backrest, with painted aluminum frame, seat and back (where present) in HDPE (high density polyethylene). Adjustable in height for better adaptability. Non-slip rubber tips. Assembling without tools.

6.5 RS831-RS833

Bath seats with or without backrest, with painted aluminum frame, seat with U-shape and back (where present) in HDPE (High density polyethylene). Adjustable in height for better adaptability. Non-slip rubber tips. Assembling without tools.

6.6 RS834

Transfer bath bench. Aluminum frame, seat and backrest in HDPE (High density polyethylene). Adjustable in height for better adaptability. Non-slip rubber tips. Assembling without tools.

6.7 RS838-RS839

SBath seats with armrests with or without backrest, with painted aluminum frame, seat and back (where present) in HDPE (high density polyethylene). Adjustable in height for better adaptability. Non-slip rubber tips. Assembling without tools.

6.8 RS908

Support stand. Support to get up from the toilet, made of painted steel, width adjustable for better adaptability. Padded armrests for better comfort, height adjustable legs, anti-slip rubber tips.

6.9 RP770

Safety handles for WC.

Realized in anodized aluminum tube diameter 2.5cm, armrests in soft sponge. Useful aid to sit and get up from the toilet. Adaptable to any type of toilet. Height-adjustable legs, antislip rubber tips.

7. ASSEMBLING

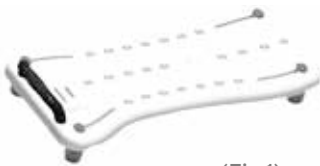
7.1 RS809

The product is supplied unassembled and it is easy to assemble without the use of any tools.

Is composed by:

- Seat (1 pc)
- Stoppers (4 pcs)
- Handle (1 pc)
- Screws

Screw the four large plastic screws on the underside of the seat. Tighten them well with a screwdriver or a small coin. To assemble the handle, position the seat with the stoppers facing downwards, introduce the handle into the holes at the ends of the seat. The handle should face the center of the seat. (Fig.1)

(Fig.1)

Seat adjustment

(Fig.2)

- Turn the seat, loosen the 4 screws so that they can move
- Slide the 4 screws into the center of the seat centre so that they can easily adapt to the bathtub
- Turn to the seat again so you place in the tub with the handle facing the wall.
(The handle must always be on the side of the wall)
- Place the bath seat in the best position.
- Slide the cleats so that they touch the inside of the tub and tighten the thumbscrews
- Flip back the seat so that the cleats are facing upwards and adjust outward to about 2 mm on each cleat. Now tighten the brackets so that they do not move easily.

- Turn the hot seat from the right and press down gently in the bathroom (not force). If the seat slides along the tub, you may need to adjust the supports further outward. To do this, repeat steps from 2 to 7

7.2 RS815

The product is supplied assembled.

7.3 RS817

The product is supplied unassembled and it is easy to assemble. In the package you will find:

- Round seat with frame (1 pcs)
- Legs (4 pcs)

7.4 RS827, RS829, RS831, RS833

The product is supplied unassembled and it is easy to assemble without the use of any tools. In the package you will find:

- Legs (4 pcs)
- Seat (1 pc)
- Backrest supports (2 pcs) (only RS829 - RS833)
- Backrest (1 pc) (only RS829 - RS833)

- Assemble the 4 legs on the seat supports thanks to push buttons
- If necessary, rotate the rubber feet to have a correct support surface
- Assemble the backrest support on the backrest thanks to buttons (only RS829 - RS833)
- Combine the seat and backrest (only RS829 - RS833)

7.5 RS834

The product is supplied unassembled and it is easy to assemble without the use of any tools. In the package you will find:

- Legs (4 pcs)
 - Seats (2 pcs)
 - Armrest/Support (1 pc)
 - Backrest (1 pc)
- Combine the two seats by Fixing button
 - Assemble the 4 legs on the seat supports thanks to pins
 - If necessary, rotate the rubber feet to have a correct support surface
 - Assemble the armrest on the seat
 - Combine the seat and backrest

7.6 RS838, RS839

The product is supplied unassembled and it is easy to assemble without the use of any tools. In the package you will find:

- Legs (4 pcs)
- Seats (2 pcs)
- Armrest (2 pcs)
- Backrest supports (2 pcs)(only RS838 - RS839)
- Backrest (1 pc) (only RS838 - RS839)
- Assemble the 4 legs on the seat supports thanks to the butterfly screws
- If necessary, rotate the rubber feet to have a correct support surface
- Fix the armrest with the same procedure
- Assemble the supports on the backrest thanks to push buttons. *(RS839)
- Combine the seat and backrest. (only RS839)

7.7 RS908

The product comes disassembled, but it is easily assembled without tools.

In the package there are:

- Lateral frame (2 pcs)
- Connecting bar (2 pcs)
- Legs (4 pcs)

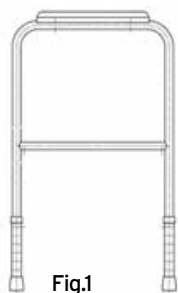
Fig.1

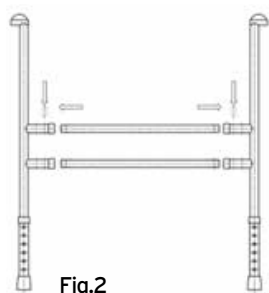
Fig.2

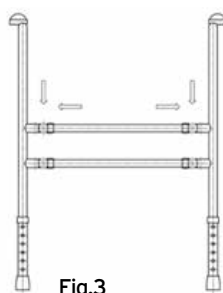
Fig.3

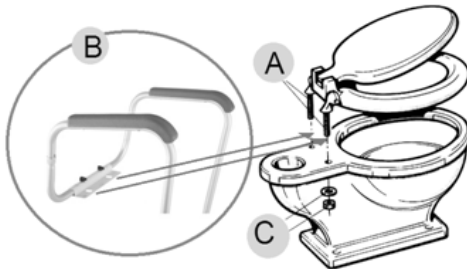
- Remove all packaging and prepare all the components for assembly.
- Connect the two lateral frames with the connecting bars, for fixing, use the clips supplied.
- The frame is width adjustable, to adjust it, remove the connecting bars from the frame and secure it with the clip to the desired location.
- Once the frame is fully assembled with all four clips, you can adjust the height of the legs (see section 10.7 How to use)

7.8 RP770

The product comes disassembled, but it's easily assembled without tools.

In the package there are:

- Legs with rubber tips (2 pcs)
- Mounting plate to fix the frame on the toilet (1 pc)
- Armrest frame (2 pcs)

A - Unscrew and lift the toilet seat.

B - Put in the legs in the main frame.

C - Connect the frame with the mounting plate to the toilet.

- Position the handles in the same attachment point of the toilet seat
- Reposition the tablet and screw
- Adjust the height of the legs. Press the stop, slide the tube bringing it to the desired height
Release the stop, check the stability of the product and check all legs are the same height

8. BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

9. CAUTIONS FOR USE

The manufacturer and distributor are not responsible for any responsibility for the consequences of improper use of the product.

- Always help the patient with walking difficulties to position themselves on the seat
- Do not use with patients who exceed the maximum weight shown on the label placed on the product frame
- Avoid sudden movements during positioning
- Be careful on wet floors, on wet or slippery surfaces, it could make difficult the patient positioning
- Do not store the product outside. Periodically check the stability of the product and the correct fastening screws and the status of protections rubber
- In case of doubt for the use, the adjustment or the maintenance of the product, please contact your dealer

⚠ Do not exceed the maximum load capacity for any reason!

10. HOW TO USE

10.1 RS809

- Place the seat in the tub and tighten the stoppers to make it stable
- Place the handle on the side opposite to the point of access to the tub
- If necessary help the patient to position on the seat
- Use the handle as an aid to move and position themselves correctly in the seat

10.2 RS815

Position the seat and open it on a smooth flat surface. Check the stability of the seat before use.

10.3 RS817

Height adjustment

Press the pin to adjust the seat by removing the pole and putting it into the holes to the desired height and the maneuverability of the patient. Return the stop in the hole to lock it. Check that all four legs are of the same height.

Place seat on a flat, level surface. Check the stability of the seat before use

10.4 RS827-RS829-RS831-RS833-RS838-RS839

Height adjustment

Press the pin to adjust the seat by removing the pole and putting it into the holes to the desired height and the maneuverability of the patient. Return the stop in the hole to lock it. Check that all four legs are of the same height.

Place seat on a flat, level surface. Check the stability of the seat before use

10.5 RS834

Height adjustment as point 10.4

Position the transfer bath bench so that two legs of the seat stay on the bottom of the bathtub and two on the floor outside. Check the stability of the seat before sitting on.

10.6 RS908

Aid to facilitate the getting up and sitting down from the toilet. Make sure not to push the entire body weight on the help because the aid is not fixed, and it is only a SUPPORT.

10.7 RP770

It is useful to get more stability in getting up and sitting down on the toilet.

Make sure not to push the entire body weight on the aid, because it is only a SUPPORT.

11. CLEANING AND DISINFECTION

10.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water. Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

10.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

12. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

13. TECHNICAL FEATURES

RS809

*Dimension:  L: 76 cm  W: 35 cm  H: 16 cm

 Weight: 2,4 kg

 Max loading weight: 160 kg

RS815

*Dimension:  L: 49 cm  W: 35 cm  H: 47 cm

Sitting: 49x29 cm

 Weight: 3 kg

 Max loading weight: 110 kg

RS817

*Dimension:  L: 34 cm  W: 34 cm  H: 39÷57 cm

Sitting: Ø 31 cm

 Weight: 1,5 kg

 Max loading weight: 110 kg

RS827

*Dimension:  L: 49,5 cm  W: 40 cm

Sitting: 49,5 x 29 x h45÷62,8 cm

 Weight: 1,8 kg

 Max loading weight: 110 kg

RS829

*Dimension:  L: 49,5 cm  W: 40 cm  H: 74,5÷87,3 cm

Sitting: 49,5 x 29 x h45÷62,8 cm

 Weight: 3 kg

 Max loading weight: 110 kg

RS831

*Dimension:  L: 50 cm  W: 46 cm  H: 41÷53,7 cm

Sitting: 48 x 33 cm

 Weight: 2,3 kg

 Max loading weight: 110 kg

RS833

*Dimension:  L: 50 cm  W: 53 cm  H: 71÷84 cm

Sitting: 48 x 33 x h41÷53,7 cm

 Weight: 3 kg

 Max loading weight: 110 kg

RS834

*Dimension:  L: 86 cm  W: 51 cm  H: 88,6÷106,3 cm

Sitting: 68,5 x 43 x h54,1÷71,9 cm

 Weight: 5 kg

 Max loading weight: 110 kg

RS838

*Dimension:  L: 51 cm  W: 54 cm  H: 47÷65 cm

Sitting: 40 x 33 cm

 Weight: 2,5 kg

 Max loading weight: 110 kg

RS839

*Dimension:  L: 51 cm  W: 54 cm  H: 80,5÷98,5 cm

Sitting: 40 x 33 x h47÷65 cm

 Weight: 3,2 kg

 Max loading weight: 110 kg

RS908

*Dimension:  L: 61-69 cm  W: 43 cm  H: 66÷76 cm

 Weight: 1,4 kg

 Max loading weight: 100 kg

RP770

*Dimension:  L: 58-63 cm  P: 49 cm  H: 70÷82,5 cm

 Weight: 5 kg

 Max loading weight: 100 kg

* The measurements refer to the overall dimensions

14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product , and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15. REPAIR

14.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

14.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

14.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

16. SPARE SPARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

17. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

WARRANTY CERTIFICATE

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ Place _____

Sold to _____

Street _____ Place _____

 MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com