MA ST323/4 02 A - 09_2015

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MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

morettispa.com - info@morettispa.com
MADE IN ITALY

INDEX

CODE	pag.8
INTRODUCTION	pag.8
INTENDED USE	pag.8
EC DECLARATION OF CONFORMITY	pag.8
GENERAL WARNING	pag.8
GENERAL DESCRIPTION	pag.9
BEFORE USE	pag.9
HOW TO USE	pag.9
MAINTENANCE	pag.9
CLEANING AND DISINFECTION	pag.9
Cleaning	
Disinfection	
CONDITIONS OF DISPOSAL	pag.9
General conditions of disposal	
TECHNICAL FEATURES	pag.10
Dimensions	
Technical specifications	
WARRANTY	pag.11

I Class Medical Device according to
Directive 93/42/EEC and further modifications

CODE

ST323 - PVC water cushion with central hole

ST324 - PVC water cushion

INTRODUCTION

Thank you for purchasing a **LEVITAS** water cushion by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly

NOTE CHECK ALL PARTS FOR SHIPPING DAMAGES AND TEST BEFORE USING. IN CASE OF DAMAGE, DO NOT USE THE DEVICE. CONTACT YOUR DEALER FOR FURTHER INSTRUCTIONS

INTENDED USE

The water cushion have been designed for bedsores care

EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the said directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC and further modifications to the proper authorities

GENERAL WARNINGS

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment otherwise injury or damage may occur
 - Advise with physician or therapist to determinate the correct adjustment and the correct use of the cushion
 - Keep the item away from heat sources
- 

- SERVICE LIFE - the wheelchair use limit is defined by the wear of the parts
- DO NOT allow children to play on or operate the cushion
- In case of extended use, is suggested to empty the cushion

GENERAL DESCRIPTION

The water cushion is made of electrosoldered PVC, to guarantee maximum strength. It has interior bellows to avoid wave-like motion of water and a refilling tube.

BEFORE USE

- Inspect the frame for any damage in order to guarantee a safe use for people and things
- Verify the filling valve is duly closed

HOW TO USE

1. Insert the refilling tube into the valve
2. Start pouring water with the help of a funnel
3. When the desired quantity is reached, add the disinfectant
4. Close correctly the filling valve

MAINTENANCE

The LEVITAS devices by MORETTI S.p.A. are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories

CLEANING AND DISINFECTION

- Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use

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If you need to disinfect the device use a common disinfectant cleaner

N.B. NEVER USE ACIDS, ALKALIS OR SOLVENTS SUCH AS ACETONE

CONDITIONS OF DISPOSAL

- General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the cushion in the appropriate disposal areas for recycling.

DISPOSAL INSTRUCTIONS IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2002/96/EC

At the end of its service life, the product may not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted disposal centre or to a reseller that provides this service. Sorted waste disposal reduces the environmental and health impact of improper disposal and enables the recycling of materials, thus resulting in energy and resource savings. To highlight the requirement for sorted 21 disposal of electro medical equipment, the product bears the barred bin symbol

TECHNICAL FEATURES

- **Material** k80 PVC - non-toxic
- **Thickness** 4/10 for paramedical use
- **Dimensions** 45x50 cm
- **Weight** **ST323** 400gr - **ST324** 450gr
- **Max user weight** 90 kg

WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

CLAUSOLE ESONERATIVE

Moretti does not offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**MORETTI****WARRANTY****LEVITAS**

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ Place _____

Sold to _____

Street _____ Place _____

**MORETTI S.P.A.**

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

morettispa.com - info@morettispa.com**MADE IN ITALY**

Via Bruxelles, 3 - Melegnano
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com