SKEMA[®]FORM

PARAVENTI IN METALLO

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INDICE

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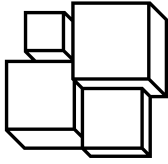
CE I Class medical device,
according Directive 93/42/EEC and subsequent amendments

1. CODES

- MO411A** Aluminum ward screen with 3 panels
- MO412C** Chromed steel ward screen with 3 panels
- MO414C** Chromed steel ward screen with 4 panels
- MO415A** Aluminum ward screen with 4 panels

2. INTRODUCTION

Thank you for choosing a **metal ward screen** from the **SKEMAFORM by MORETTI S.p.A.** Their design and quality are a guarantee of comfort, safety and reliability. The metal ward screens from the **SKEMAFORM by MORETTI S.p.A.** are designed and built to meet all your demands for a practical and correct use. This user manual provides for some suggestions as how to use the item you have chosen correctly and offers a lots of valuable advices for your safety. Please read carefully this manual before using the metal ward screen. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

The metal ward screen by SKEMAFORM is a device formed by 3 or 4 panels linked together by zippers, intended and used as a mobile or partition wall.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF EC CONFORMITY

MORETTI S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by MORETTI S.p.A. and belonging to the WARD SCREEN category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC: To this aim MORETTI S.p.A. guarantees and declares as follows, under its own responsibility

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the mentioned directive.
2. The complete list of the devices in question is available in annex.
3. The devices in question are not measurement tools.
4. The devices in question are not intended for clinical investigation.
5. The devices in question are sold in non sterile packing.
6. The devices in question should be considered as belonging to class I.
7. Do not use/install the devices in question for any other purpose than that one expressly indicated by MORETTI S.P.A.
8. Moretti S.P.A. maintains and submits the technical documentation certifying the conformity of the products in question with the directive 93/42/eec to the proper authorities

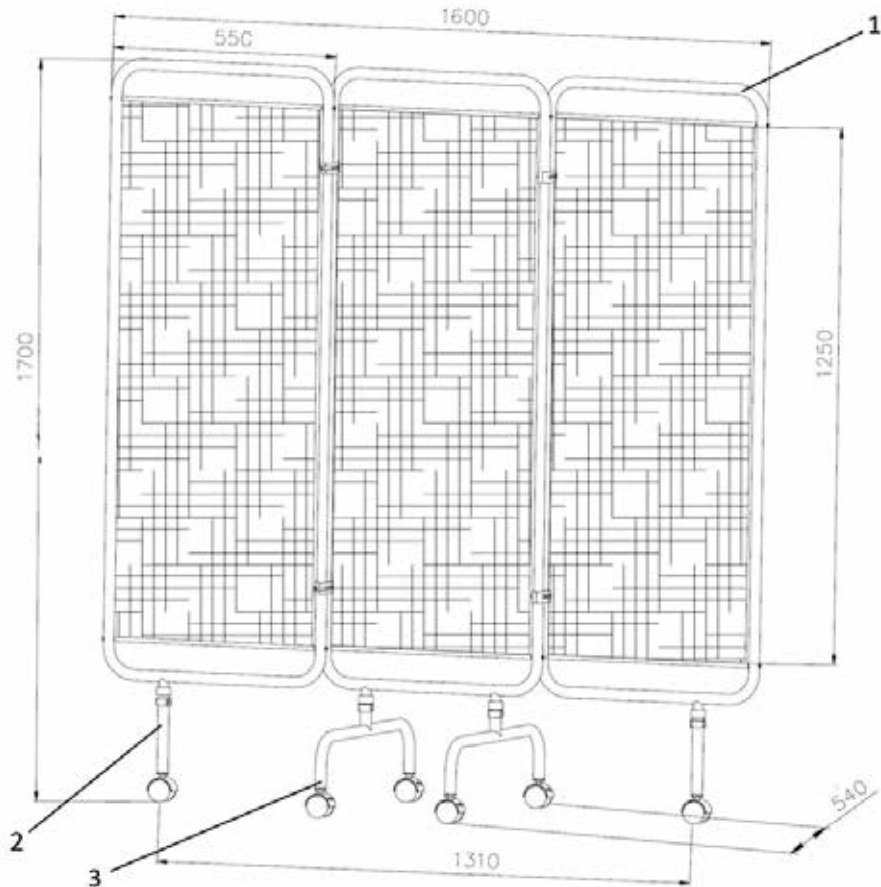
5. GENERAL WARNINGS

For a proper use of the device refer to this manual.

- Keep the packaged product away from any source of heat because the packaging is made of cardboard
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and/or replaced.
- Before using the device, always check that the mounting screws are properly tightened.
- Be careful when children are nearby.

6. GENERAL DESCRIPTION

6.1 Parts view and description M0411A and M0412C


List of the parts

1. Pre-assembled frame with 3 doors (1 pc)
2. Leg with castor (2 pcs)
3. Central leg with swivel castors and brake (2 pcs)

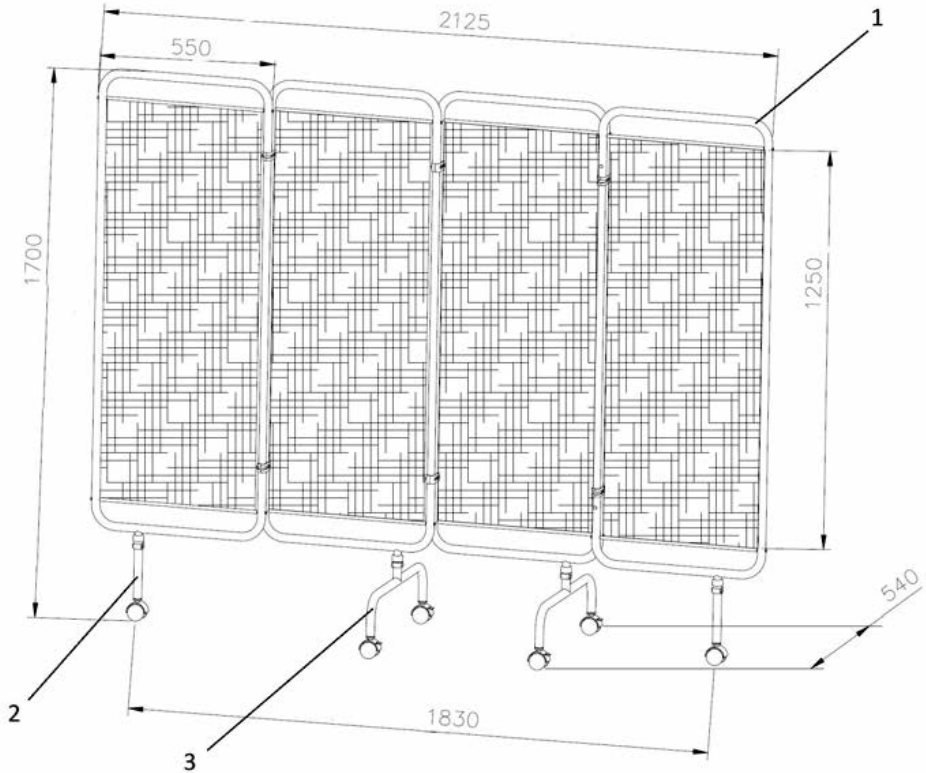6.2 Parts view and description MO414C and MO415A

List of the parts

1. Pre-assembled frame with 4 doors (1 pc)
2. Leg with castor wheel (2 pcs)
3. Central leg with swivel castors and brake (2 pcs)

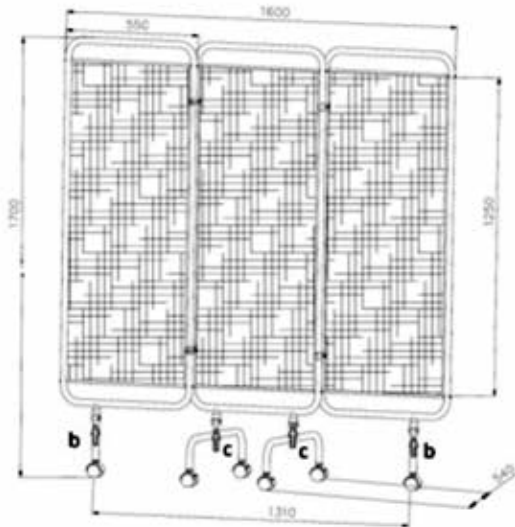
7. ASSEMBLY

WARNING!


Perform these operations paying close attention to prevent injury and/or property damage when moving the device

7.1 Installation instructions for MO411A and MO412C

- A. Unpack the product.
- B. Insert the legs (n.2) into the special housing on the frame (n.1) as in figure, and fix using the pins provided.
- C. Insert the legs (n.3) in the special housing on the frame (n.1) as shown, and fix using the pins provided.
- D. At this point the screen is mounted and ready for use.

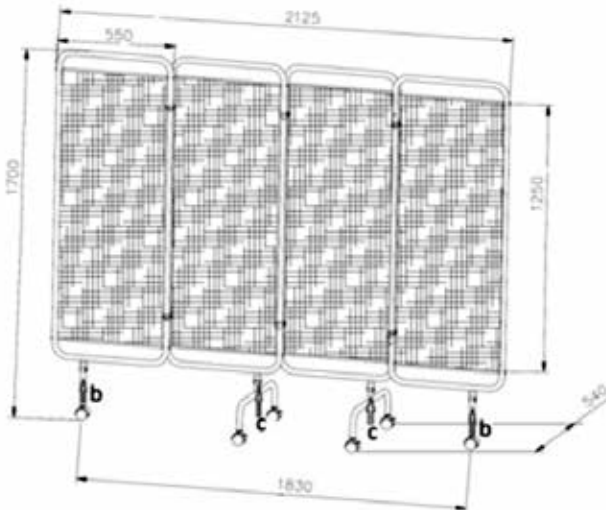7.2 Installation instructions for MO414C and MO415A

- A. Unpack the product.
- B. Insert the legs (n.2) into the special housing on the frame (n.1) as in figure, and fix using the pins provided.
- c. Insert the legs (n.3) in the special housing on the frame (n.1) as shown, and fix using the pins provided.
- D. At this point the ward screen is mounted and ready for use.

8. PRECAUTIONS FOR USE

- Before each use always check the state of wear of the mechanical parts to ensure a safe use of the device for persons and things.
- Before each use check the correct assembly of the device, with particular attention to the joints connecting screws, wheels, walls.
- Be careful to entrapment while moving the ward screen.
- When the ward screen is not moving, make sure that the parking brakes are blocked

9. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

- More specifically, please check that all screws and bolts are well tightened

10. CLEANING AND DISINFECTION

10.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

10.2 Disinfection

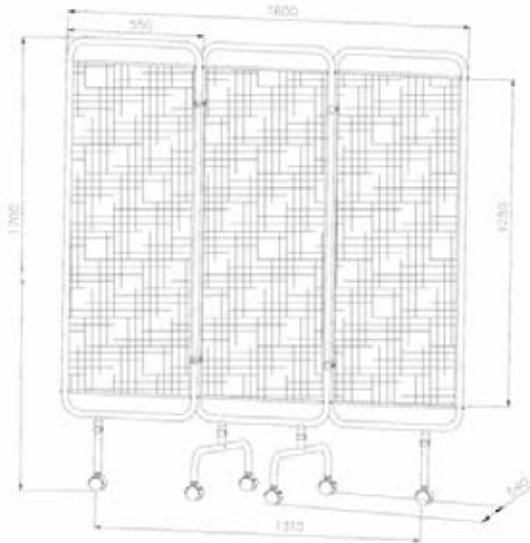
If you need to disinfect the product use a common disinfectant neutral detergent.

11. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

12. TECHNICAL FEATURES


12.1 Dimensions and weight of 3 panels ward screens MO411A and MO412C

- 3 swiveling panels screen in aluminium MO411A or chromed steel MO412C
- Washable and flame retardant PVC curtains
- 2 Pivoting castors Ø50mm
- 2 Pivoting castors with brakes Ø50mm

Height:	170 cm
Width:	160 cm
Depth:	54 cm
Weight MO411A:	7,7 kg
Weight MO412C:	9,2 kg

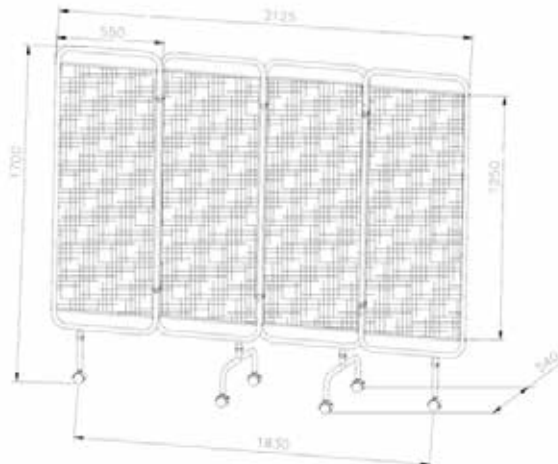12.2 Dimensions and weight of 4 panels ward screens MO414C and MO415A

- 3 swiveling panels screen in aluminium MO415A or chromed steel MO414C
- Washable and flame retardant PVC curtains
- 2 Pivoting castors Ø50mm
- 2 Pivoting castors with brakes Ø50mm

Height:	170 cm
Width:	212 cm
Depth:	54 cm
Weight MO414C:	14 kg
Weight MO415A:	10,5 kg

13. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

14. REPAIRING

14.1 Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

14.2 Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

14.3 Non defective device

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

15. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

16. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com