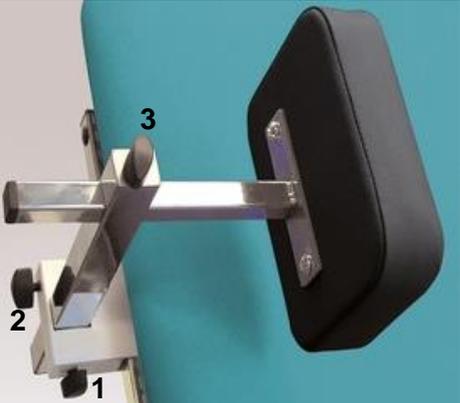
AC0517 – PADDED SUPPORTS
COPPIA SPINTORI TRONCO

Manufacturer	<p>CHINESPORT S.P.A., Via Croazia, 2-33100 Udine, Italia. Phone. +39 0432 621 621 – Fax +39 0432 621 620 – website: www.chinesport.com The company produces according to UNI EN ISO 13485:2012</p>				
Intended use and equipment description	<p>Shaped side supports that offer a larger contact surface with the patient. These are supplementary or alternative to article code AC0003 and can be adjusted in height, width, and depth. (2 pieces).</p>				
Composition	<p>Steel and leatherette.</p>				
Storage	<p>In case of storage the following ambient conditions must be respected:</p> <ul style="list-style-type: none"> - relative humidity 10% / 90 %; - temperature -10°C / +50°C. 				
Special warning and use restrictions	<p>These devices must not be used:</p> <ul style="list-style-type: none"> • without making all the settings required and recommended setups. • when use is not advisable or causes the patient serious discomfort. • if the original product has been tampered with and/or modified • when set incorrectly or when there is any suspicion that it may harm the patient 				
Collateral effects and contraindications	<p>There is no collateral effects or contraindications during proper use. Use the device always in the proper way. The knowledge of this instruction manual is recommended.</p>				
Unpacking and mounting	<p>Simply remove the device from the package. Never use cutters to open box to prevent damages. Properly dispose of the packaging. The carrier and the supplier must be immediately notified of any damage that may have occurred during transport. Damaged equipment cannot be used.</p>				
Installation and use	<p>Apply the painted element to the guide on the sides of the bed. Insert the perforated rod inside the painted element. Insert the stem of the pad into the hole of the perforated rod. Adjust the height, depth and width of the pusher using clamping keys 1, 2, 3 respectively. Tighten the keys at the end of the adjustment.</p> 				
Maintenance	<p>Check the condition of the safety of the device after each situation of "rough treatment" (ex.: fall, heavy impacts, liquid spills, etc.) and any time you are unsure about the state security apparatus. Routine maintenance tasks are to be carried out at least once every 6 months.</p> <table border="1" data-bbox="395 1823 1453 1944"> <thead> <tr> <th data-bbox="395 1823 624 1854">FREQUENCY</th> <th data-bbox="628 1823 1453 1854">TASK</th> </tr> </thead> <tbody> <tr> <td data-bbox="395 1861 624 1944">Every six months</td> <td data-bbox="628 1861 1453 1944"> General cleaning Visual and functional inspection of the equipment Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation </td> </tr> </tbody> </table>	FREQUENCY	TASK	Every six months	General cleaning Visual and functional inspection of the equipment Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation
FREQUENCY	TASK				
Every six months	General cleaning Visual and functional inspection of the equipment Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation				

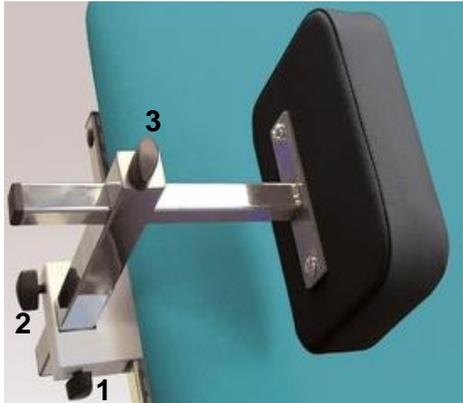
AC0517 – PADDED SUPPORTS
COPPIA SPINTORI TRONCO

Cleaning	Dust gently and in case of light stains pass the surface with a damp cloth or a neutral detergent, rinse with water. Dry avoiding direct exposure to the sunlight. The following instructions are recommended for small located and more obstinate stains (oil, fat, cosmetics, ink, coffee, liquor, chewing-gum etc.): dissolve immediately with clear ethylic alcohol diluted in water to 20%, dab and then treat with a neutral detergent solution. Rinse abundantly.
Disinfection	For the disinfection of surfaces such as chassis, sky, hand control, etc. we recommend the use of disinfectants such as AMUCHINA® 10% or equivalent disinfectant with sodium hypochlorite 0.1%
Additional informations	Please visit www.chinesport.com

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