



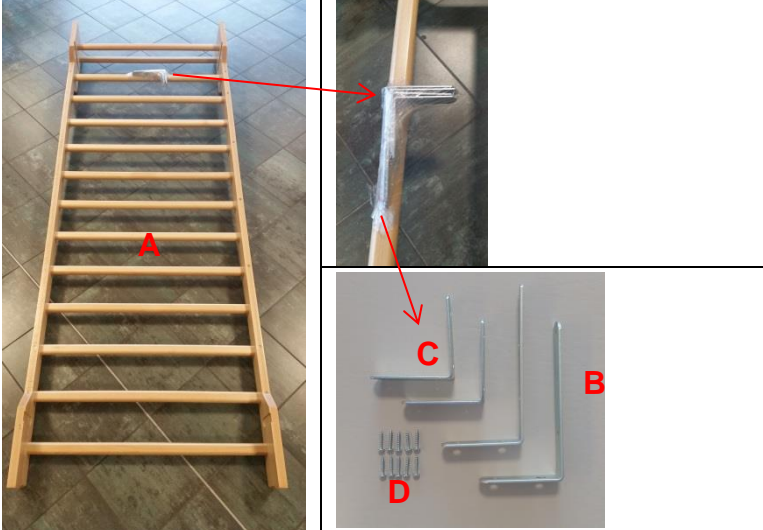

Manufacturer	CHINESPORT S.P.A. , Via Croazia, 2-33100 Udine, Italia. Phone: +39 0432 621 621 – Fax +39 0432 621 620 – website: www.chinesport.com The company produces according to UNI EN ISO 13485:2012									
Intended use and equipment description	Swedish beech-wood wall bars with single span, are natural varnished and supplied with anchoring white steel clamps galvanized. The white steel clamp have to be used at the four angles to fix the wall bars. The wall bars are made up of 14 ovoid rungs. They have a diameter of 3,5 cm and there is a space of 14 cm between each other. The wall bars can be used to carry out some gymnastic and rehabilitation exercises: muscle and joint stretching, corrective exercises of various paramorphisms and they can also be used by children for exercises of joint looseness. The two rungs on the top are positioned one more projecting and the other more re-entering to make it possible some suspension exercises or passive cervical tractions.									
Storage	In case of storage the following ambient conditions must be respected: - relative umidity 10% / 90 %; - temperature -10°C / +50°C.									
Special warning and use restrictions	These devices must not be used: <ul style="list-style-type: none"> • without making all the settings required and reccomended setups. • when use is not advisable or causes the patient serious discomfort. • if the original product has been tampered with and/or modified • when setted incorrectly or when there is any suspicion that it may harm the patient • straight on a plasterboard wall Do not spill liquids on the metal parts. The physical and psychological suitability of the user to make use of the aids must be approved by a specialist.									
Collateral effects and contraindications	There is no collateral effects or contraindications during proper use. Use the device always in the proper way. The knowledge of this instruction manual is recommended.									
Unpacking	Simply remove the device from the package. Never use cutters to open box to prevent damages. Properly dispose of the packaging. The carrier and the supplier must be immediately notified of any damage that may have occurred during transport. Damaged equipment cannot be used.									
Content of the package		<table border="1"> <tr> <td>A – WALL BARS</td> <td>[1 PC]</td> </tr> <tr> <td>B – UPPER BRACKET</td> <td>[2 PCS]</td> </tr> <tr> <td>C – LOWER BRACKETS</td> <td>[2 PCS]</td> </tr> <tr> <td>D – SCREWS</td> <td>[10 PCS]</td> </tr> </table>	A – WALL BARS	[1 PC]	B – UPPER BRACKET	[2 PCS]	C – LOWER BRACKETS	[2 PCS]	D – SCREWS	[10 PCS]
A – WALL BARS	[1 PC]									
B – UPPER BRACKET	[2 PCS]									
C – LOWER BRACKETS	[2 PCS]									
D – SCREWS	[10 PCS]									
Mounting instructions	<p>1 Take the brackets</p>  <p>2 Fix the upper and the lower brackets to wall bars sides by using the screws provided [D].</p>									

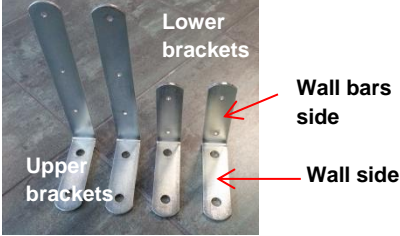


04540
SINGLE WALL BARS
CE

	<p>Upper side x2</p> <p>Lower side x2</p> <p>3 Fix the wall bars to the wall by using suitable fixing dowels (not provided).</p>				
Warning!	<p>Use only suitable fixing dowels depending on the wall structure. Don't fix the wall bars straight on a plasterboard wall. Use a proper fixing counterframe structure to install the wall bars on a plasterboard wall.</p>				
Maintence	<p>Maintenance work must be performed by specialized technicians or people with knowledge of the contents of this instruction booklet. Check the condition of the safety of the device after each situation of "rough treatment" (ex.: fall, heavy impacts, liquid spills, etc.) and any time you are unsure about the state security apparatus. Routine maintenance tasks are to be carried out at least once every 12 months.</p> <table border="1" data-bbox="359 1568 1444 1646"> <thead> <tr> <th>FREQUENCY</th> <th>TASK</th> </tr> </thead> <tbody> <tr> <td>Half-yearly</td> <td>General cleaning of the equipment Visual and functional inspection of the equipment</td> </tr> </tbody> </table>	FREQUENCY	TASK	Half-yearly	General cleaning of the equipment Visual and functional inspection of the equipment
FREQUENCY	TASK				
Half-yearly	General cleaning of the equipment Visual and functional inspection of the equipment				
Cleaning	<p>Dust gently and in case of light stains pass the surface with a damp cloth or a neutral detergent, rinse with water without squeezing. Let the fabric dry avoiding direct exposure to the sunlight. The following instructions are recommended for small located and more obstinate stains (oil, fat, cosmetics, ink, coffee, liquor, chewing-gum etc.): dissolve immediately with clear ethylic alcohol diluted in water to 20%, dab and then treat with a neutral detergent solution. Rinse abundantly.</p>				
Disinfection	<p>For the disinfection of surfaces such as chassis, sky, hand control, etc. we recommend the use of disinfectants such as AMUCHINA® 10% or equivalent disinfectant with sodium hypochlorite 0.1%</p>				



Please visit www.chinesport.com

Additional informations

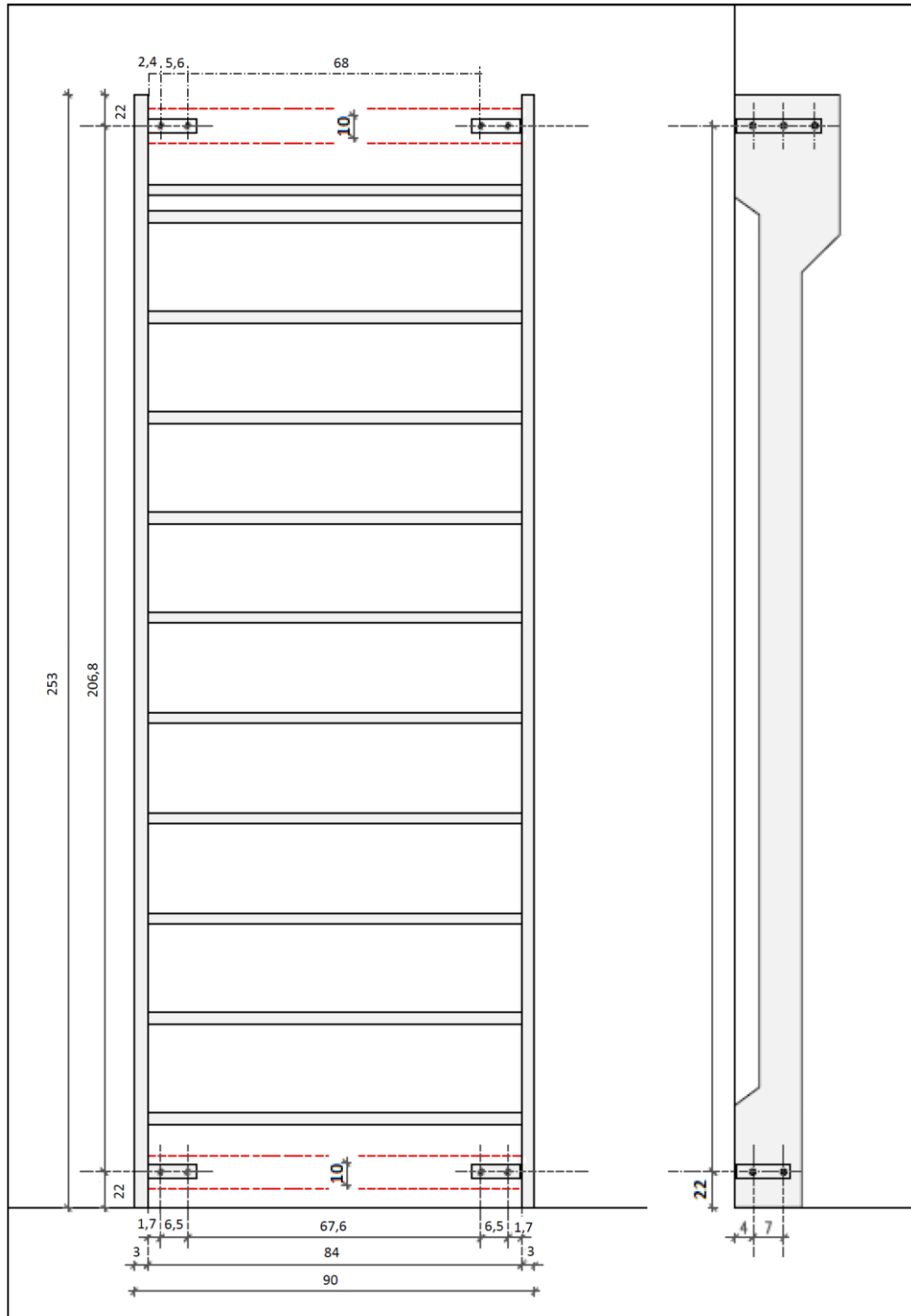

The red lines represent the fixing bracket to use with a plasterboard wall. A 5 mm depth bracket is recommended. We also recommend to fix the bracket to a proper bearing structure and to pierce the bracket only when ready to fix the wallbars. Please note that the dimensions are approximate.