## MANUALE D'USO BALNEO SOLLEVATORE DA PISCINA

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## USER MANUAL BALNEO

## INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

## GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

## DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

## SAFETY

Passive lifts allow the lifting from bed / wheelchair / stretcher of patients with important functional limitations. Below are safety tips.

It is advisable to check the lift in all its parts before use.

- Check the connection between the remote control and the battery.
- Check that the battery is charged.
- Check the harness for any signs of wear or damage.
- Check the brake seal.

Maximum load: 200 kg.

Warning! Do not exceed the maximum range.

Warning! The battery should only be assembled and repaired by qualified personnel.

## DESCRIPTION

The electric lift for POOL BALNEO in stainless steel, helps patients to enter and exit the pool. It is equipped with a removable LINAK control unit, to be able to recharge it more conveniently, located on the supporting column. The lift is equipped with a remote control with electric cable, 5-point slingbar with safety system, handle to facilitate rotation, base with 4 pins to fix BALNEO on the edge of the pool, a RED MUSHROOM emergency button that blocks the flow of electricity, a mechanical emergency lowering system in case of power failure and an additional electronic emergency lowering system in case of remote control malfunction. The mobile arm descends to the level of the pool and rotates 360 ° (MANUAL ROTATION).

Il lifter can be combined with:

- Sling (sku 01.01.20013)



- Seat (sku 01.01.BALNEOSEAT)

## TECHNICAL SPECIFICATIONS

### BALNEO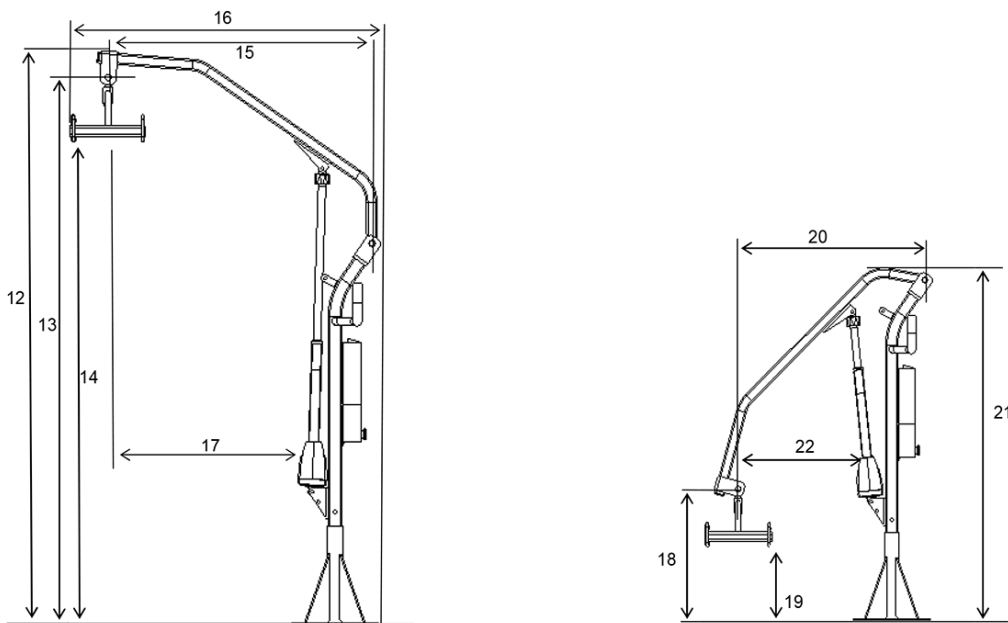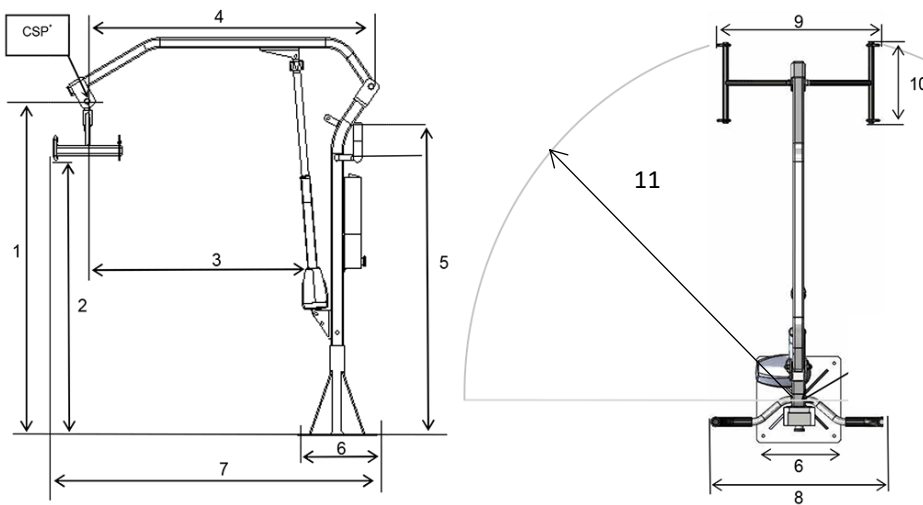

\*CSP: Central suspension point

#### Dimensional characteristics:

1 - CSP average height: 1333 mm  
 2 - Average height of the slingbar from the ground: 1039 mm  
 3 - Useful internal width: 736 mm  
 4 - Maximum arm width: 1056 mm  
 5 - Floor / handle height: from 1120 to 1240 mm  
 6 - Width of the fixing plate to the floor: 320 mm  
 7 - Maximum total length: 1210 mm



8 - Total width of the handles: 668 mm  
 9 - Length of the slingbar: 620 mm  
 10 - Depth of the slingbar: 300 mm  
 11 - Turning radius: 1292 mm  
 12 - Overall height in high position: 2040 mm  
 13 - Maximum height CSP: 1941 mm  
 14 - Maximum height of the slingbar from the ground: 1647 mm  
 15 - Lifting arm at the highest point of the CSP: 903 mm

16 - Maximum length: 1060 mm  
 17 - Useful internal width at maximum height: 583 mm  
 18 - Minimum CSP height: 608 mm  
 19 - Minimum height of the slingbar: 314 mm  
 20 - Sling bar at the lowest point: 710 mm  
 21 - Height in lowest position: 1438 mm  
 22 - Useful internal width at minimum height: 390 mm  
 23 - Lifting distance (13-18): 1333 mm

## BALNEO e BALNEOSEAT





### Dimensional characteristics:

- 13 - Max Height: 2040 mm
- 14 - Maximum height of the CSP: 1941 mm
- 15 - Maximum seat height: 482 mm
- 16 - Length of the lifting arm at the highest point of the CSP: 903 mm
- 17 - Useful internal width at maximum height: 344 mm
- 18 - Maximum base-seat length: 1240 mm
- 19 - Seat height from the ground (maximum height): 328 mm
- 20 - Minimum CSP height: 608 mm
- 21 - Lift arm width at the lowest point of the CSP: 710 mm
- 22 - Height of supporting column: 1438 mm
- 23 - Floor / seat height in low position: - 798 mm
- 24 - Floor / seat height: -1005 mm (under the edge of the pool)
- 25 - Internal width usable at minimum height: 122 mm
- 26 - Base-seat length (minimum height): 1117 mm
- 27 - Lifting distance (14-20): 1333 mm
- 28 - Front seat width: 400 mm
- 29 - Rear seat width: 450 mm30 – Profondità sedile: 380 mm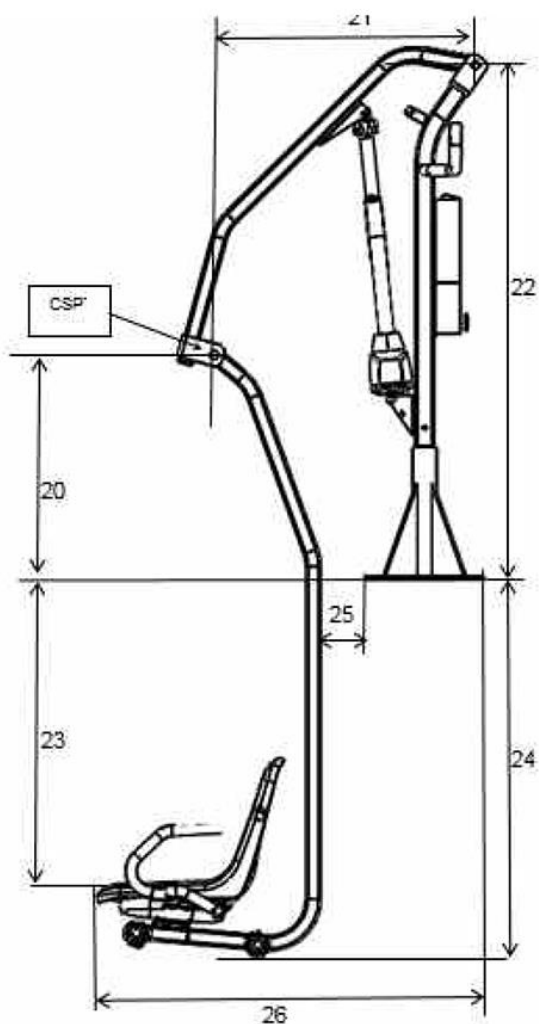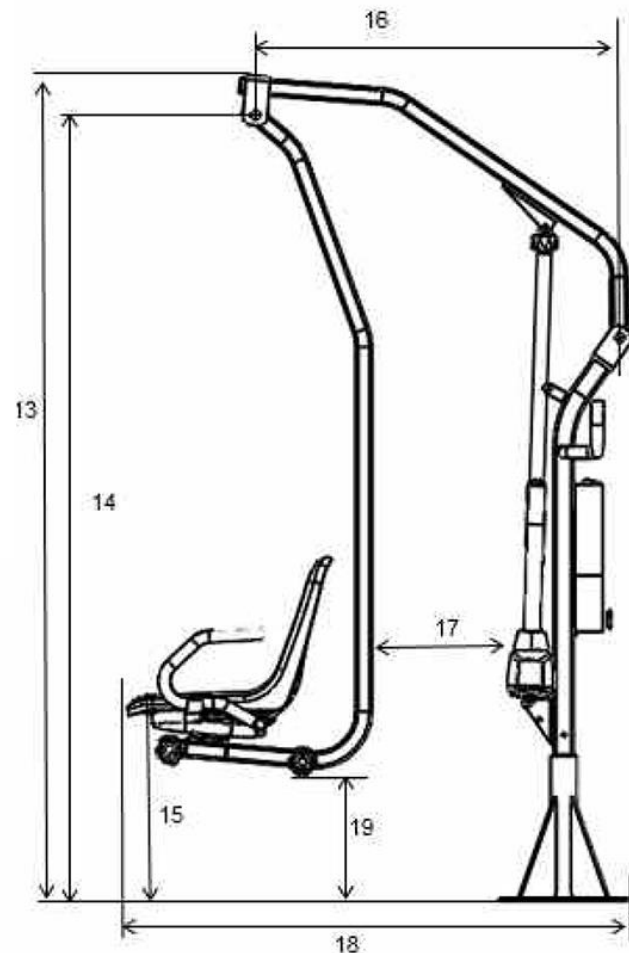


\*CSP: Central suspension point

## BATTERY

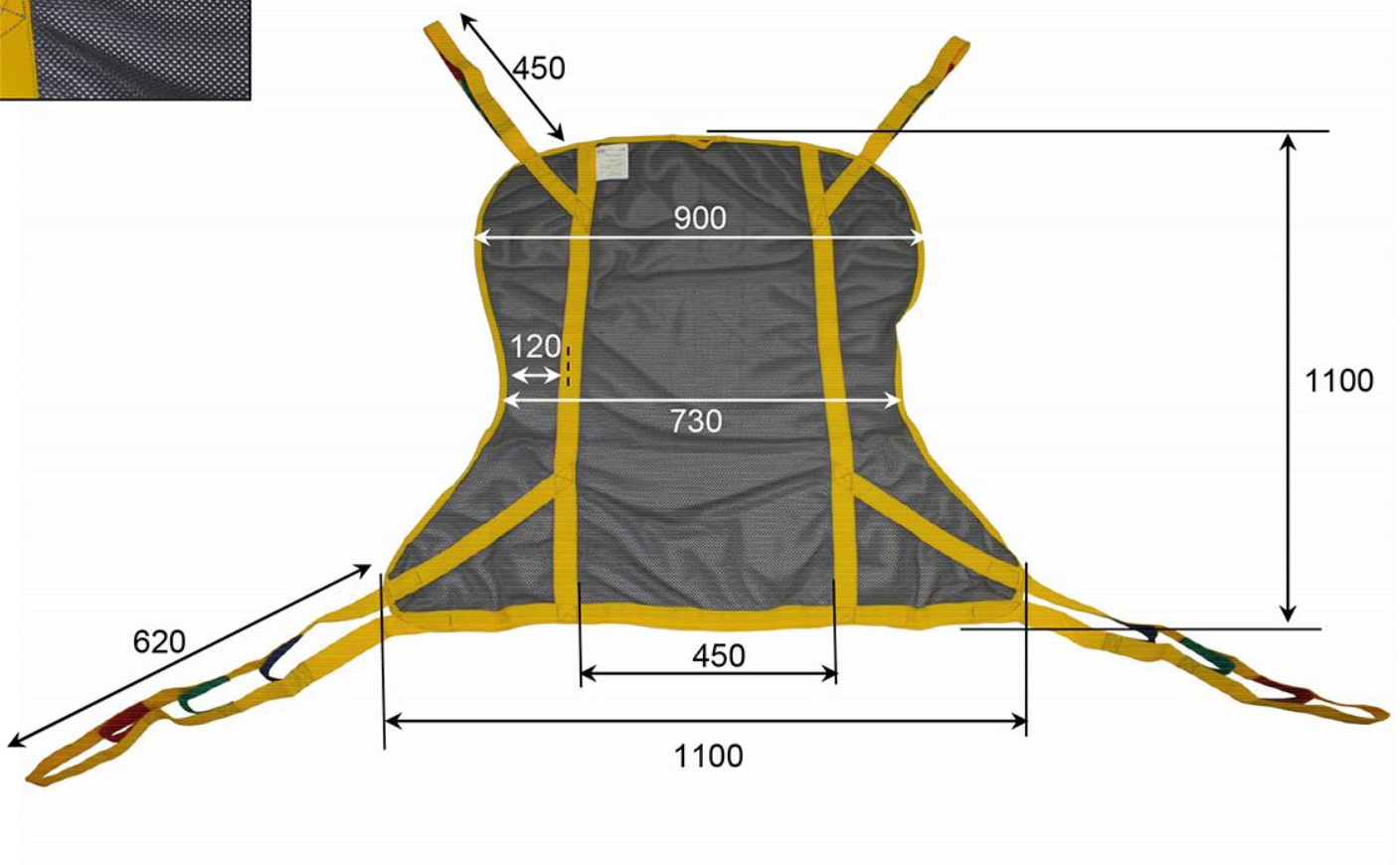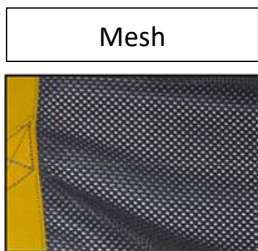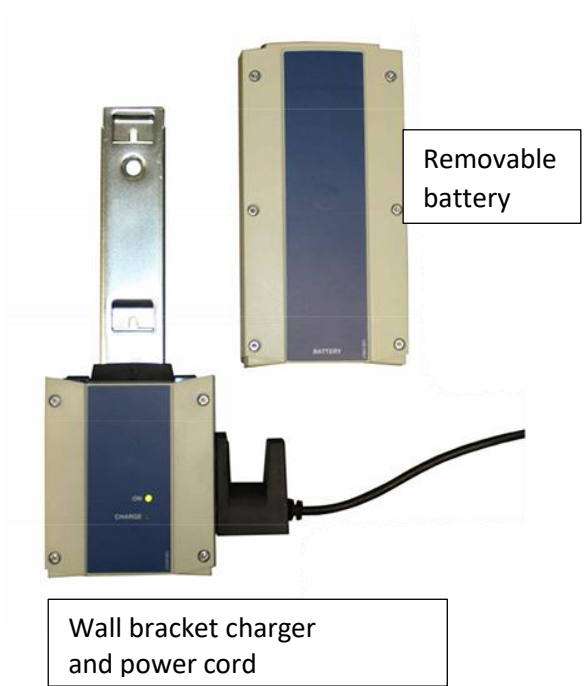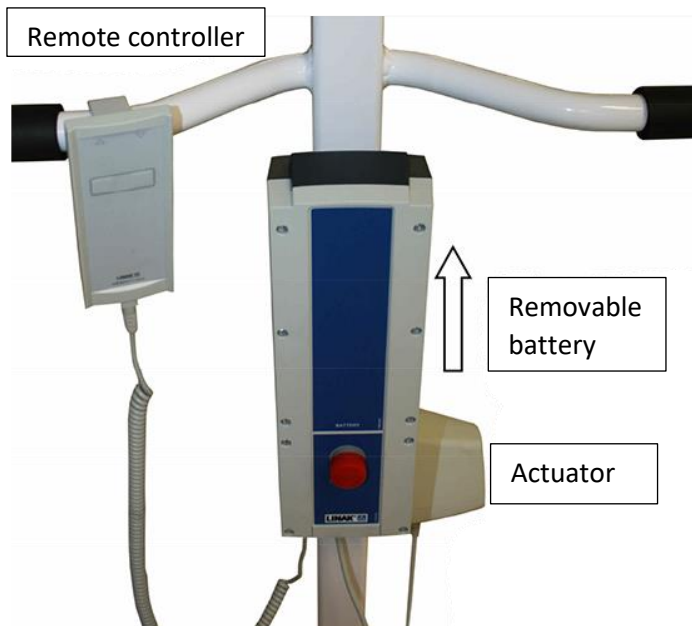\* Dimensions (mm)

## SEAT FASTENING STRAPS (01.01.BALNEOSEAT)

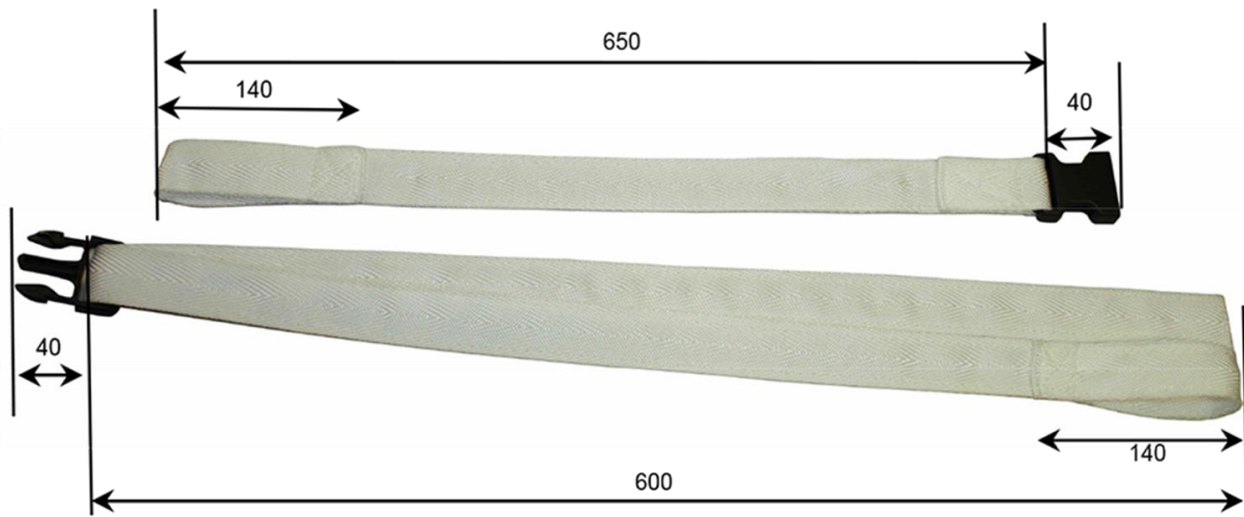\* Dimensions (mm)

### WARNINGS

1. Avoid bumping or hitting the electrical parts of the lift;
2. Check all electrical connections to make sure they are connected and securely fastened;
3. When the battery indicator on the display is fully illuminated, the battery is charged and all electrical components are functioning normally.

### INSPECTION

Daily inspection:

1. Visually inspect the lifter for damage or wear;
2. Check if the screws and nuts are well tightened;
3. Check the emergency lowering function.

If in doubt, contact your dealer.

### CLEANING

Never rinse the product with water or contact it directly with water.

This product can be easily wiped clean with a damp cloth. Never use chemicals to clean the sling. This may cause corrosion and tearing of the sling. Clean it with a wet towel and a mild soap solution until it is completely dry.

### GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

## **TREATMENT OF EXHAUSTED BATTERIES (DIRECTIVE 2006/66 / CE)**

### **BATTERY REGISTRATION NUMBER IT21040P00006953**

The battery contained in the appliance must not be disposed of as urban waste (Directive 2006/66 / CE), as it is subject to separate collection to avoid damage to the environment.

The user must therefore give the exhausted battery-accumulator free of charge to the appropriate municipal centers for the separate collection of electrical and electronic waste, or deliver it to the special containers provided by the points of sale.

To remove the battery: SEE THE BATTERY CHAPTER.

## **WARNINGS FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2012/19 / EU.**

### **AEE REGISTRATION NUMBER IT21050000012993**

At the end of its useful life, the product must not be disposed of with municipal waste.

It can be delivered to the appropriate separate collection centers set up by municipal administrations, or from retailers that provide this service. Dispose of

separately, the product avoids possible negative consequences for the environment and for health deriving from its inappropriate disposal. Allows also materials to be recovered in order to obtain an important saving of energy and resources. For point out the obligation to dispose of electromedical equipment separately on the product the mark of the crossed out mobile rubbish bin is shown.

## **WARRANTY**

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.