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**MANUAL**  
**LIFTY4 LIFT**  
**CODE 01.01. LIFTY4D**

## INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

## GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

## DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

## SAFETY

Passive lifts allow the lifting from bed / wheelchair / stretcher of patients with important functional limitations. Below are safety tips.

It is advisable to check the lift in all its parts before use.

- Check the connection between the remote control and the battery.
- Check that the battery is charged.
- Check the harness for any signs of wear or damage.
- Check the brake seal.

Maximum load: 200 kg.

Warning! Do not exceed the maximum range.


Warning! Always keep the lift on a level surface.

Warning! Do not attempt to cross passages, steps or other obstacles.

Warning! The battery should only be assembled and repaired by qualified personnel.

## GENERAL DESCRIPTION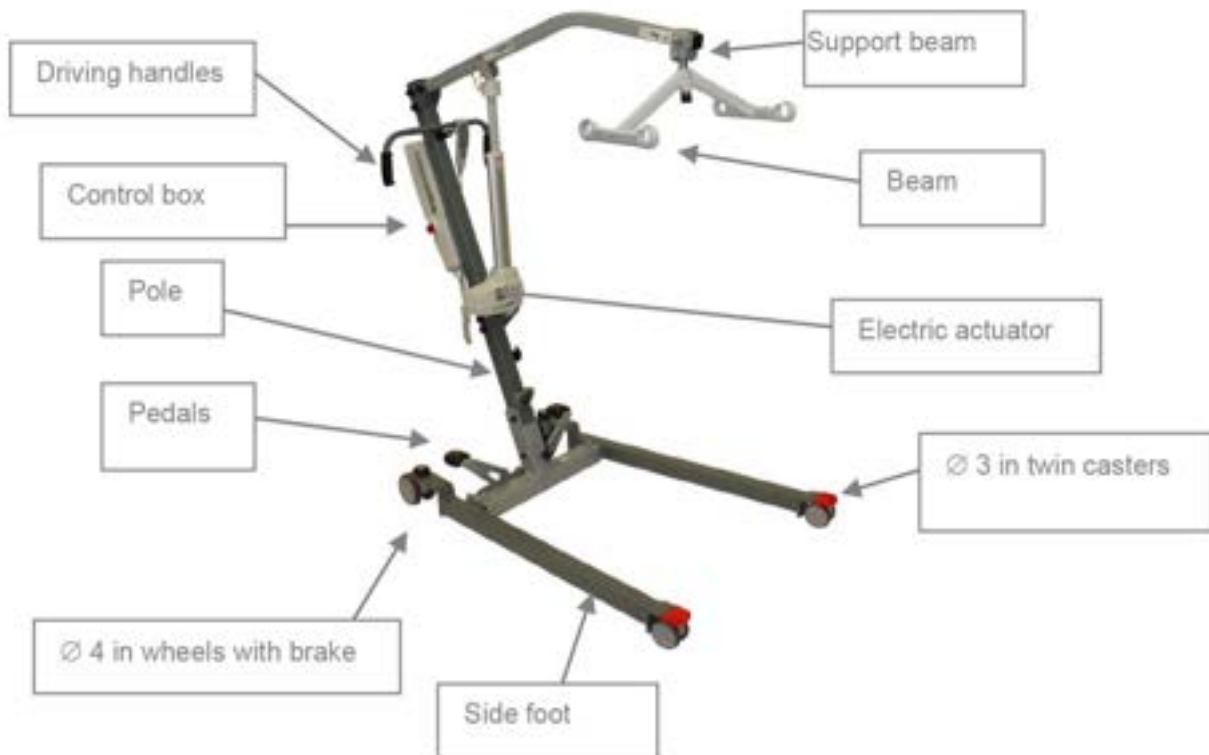

**Materials :**

**Finish :**

**Class:**

**Weight (without support device):**

steel

Epoxy paint coating

I

46 kg / 101 lbs

**Maximum patient weight to be transferred:**

200 kg / 31.5 stones

**Electric protection classification :**

**Applied parts :**

Class II

Type B

**Electrical equipment conformed to NF EN 60 601-1 and to NF EN 60 601-1-2**

**Duty cycle:**

**Input:**

**Actuator input :**

**Battery charge required time :**

**Battery autonomy:**

**Electrical equipment protection :**

2 min./18 min. (10%)

230V ~ 50 HZ

24 V CC


15 h

± 40 cycles

Remote control : IP X6

Control box : IPX4

Actuator : IP X4

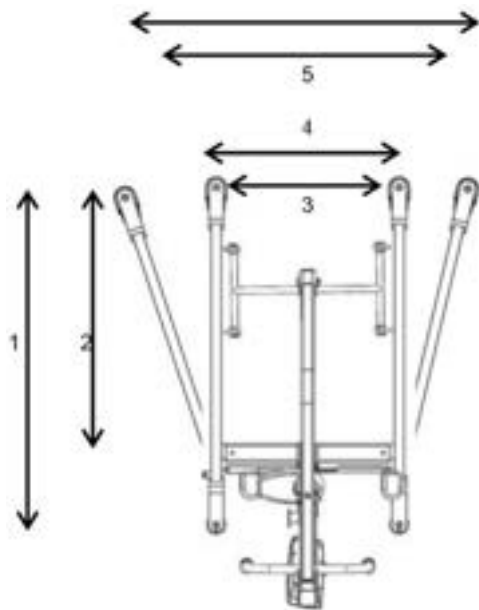

### Dimensional characteristics of the base

- 1 - Maximum external length : 43.7 in
- 2 - Maximum internal length : 33.6 in
- 3 - Minimal internal width : 20.5 in
- 4 - Minimal external width : 25.6 in
- 5 - Maximum internal width : 35.4 in
- 6 - Maximum external width : 41 in
- 7 - Minimum ground clearance : 1.2 in
- 8 - Maximum front height : 3.5 in
- 9 - Maximum back height : 5.1 in

- 1 - Lifting radius at the highest CSP : 29.9 in
- 2 - Total lift (3-4): 45.8 in
- 3 - Maximum CSP height : 76 in
- 4 - Minimal CSP height : 35 in
- 5 - CSP height at maximum lifting radius : 52 in
- 6 - Lifting radius at lowest CSP : 35 in
- 7 - Maximum lifting radius : 37.8 in
- 8 - Low position floor/hooks height : 24 in
- 9 - High position floor/hooks height : 65.3 in
- 10 - Low position overall dimension height : 54 in
- 11 - High position overall dimension height : 83 in
- 12 - Overall dimension length : 53.5 in
- 13 - Floor/handles height : 43.3 in

\* CSP : Central Suspension Point

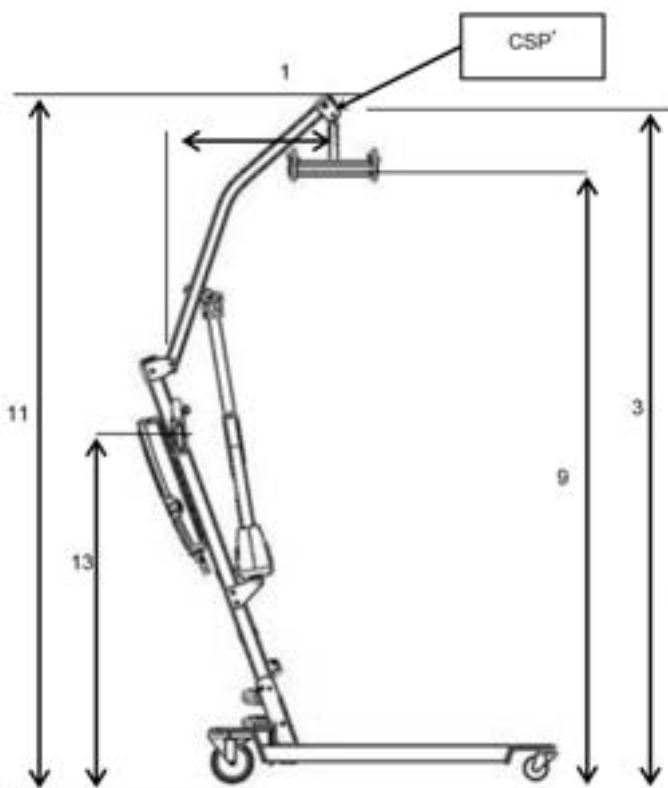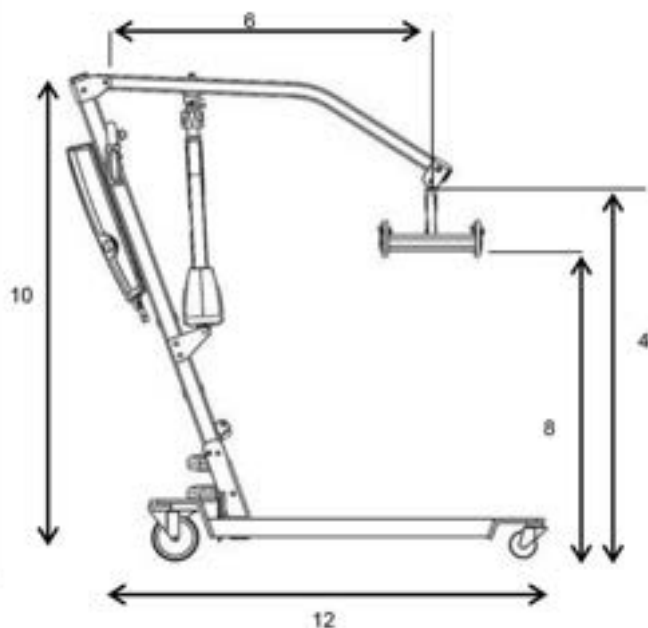## PACKAGE CONTENTS

Control box (x1)

Fixing screw  
CHC 6x80 (x2)

Spanner 6 section 5  
(x1)

Anti-pulling up  
Cable kit (x1)

Remote control  
(x1)

Main cable  
(x1)

## RECOMMENDED CHECKS

Please carefully consider how important it is to check and maintain the lift evenly to ensure that it is used to standard specifications and according to our recommendations. We will not be responsible for any damage due to abnormal use or too high a number of cycles. You will find all the information you need for your inspection and maintenance phases on the following pages.

	Item to check	Check <i>Periodicity: at least once a year and before renting it.</i>	* Replacement after 11000 cycles
1	Pole / beam support connection	- Check brass ring wear - Check fixing screw wear - Check stop-nut presence and assembling	✓
2	Control box	- Check emergency stop operating - Check battery charging	
3	Electrical cables	- Check cables (pinching, plug state, cut ...)	
4	Actuator	- Check charging operating	
5	Actuator / pole connection	- Check fixing screw worn state - Check stop-nut presence and assembling	✓
6	Pedals	- Check pedals operating - Check connecting rods wear	
7	Back wheels	- Check braking system	
8	Side foot / cross piece connection	- Check fixing wear (screw nut + brass rings)	✓
9	Actuator / beam support connection	- Check cotter clip presence and assembling	✓
10	Emergency release	- Check electrical and manual release operating	
11	Beam support / beam axle connection	- Check stop-nut presence and assembling - Check fixing screw wear - Check fixing metal tongue welding	✓
12	Beam	- Check fixing screw wear - Check control and operation of safety springs	
13	Sling	- Check seams	
14	Beam hooks	- check welding and assembling	
15	Pole / cross piece connection	- Check removable axle wear and assembling - Check safety β pin presence or Ø 10 stop nut - Check screw / stop-nut connection (presence, wear and assembling) - Check pole welding	✓

number of runs / a day	Oldness of the hoist (years)									
	1	2	3	4	5	6	7	8	9	10
1	365	730	1095	1460	1825	2190	2555	2920	3285	3650
2	730	1460	2190	2920	3650	4380	5110	5840	6570	7300
3	1095	2190	3285	4380	5475	6570	7665	8760	9855	10950
4	1460	2920	4380	5840	7300	8760	10220	11680	13140	14600
5	1825	3650	5475	7300	9125	10950	12775	14600	16425	18250
6	2190	4380	6570	8760	10950	13140	15330	17520	19710	21900
7	2555	5110	7665	10220	12775	15330	17885	20440	22995	25550
8	2920	5840	8760	11680	14600	17520	20440	23360	26280	29200
9	3285	6570	9855	13140	16425	19710	22995	26280	29565	32850
10	3650	7300	10950	14600	18250	21900	25550	29200	32850	36500
11	4015	8030	12045	16060	20075	24090	28105	32120	36135	40150
12	4380	8760	13140	17520	21900	26280	30660	35040	39420	43800
13	4745	9490	14235	18980	23725	28470	33215	37980	42705	47450
14	5110	10220	15330	20440	25550	30660	35770	40880	45990	51100
15	5475	10950	16425	21900	27375	32850	38325	43800	49275	54750
16	5840	11680	17520	23360	29200	35040	40880	46720	52560	58400
17	6205	12410	18615	24820	31025	37230	43435	49640	55845	62050
18	6570	13140	19710	26280	32850	39420	45990	52560	59130	65700
19	6935	13870	20805	27740	34675	41610	48545	55480	62415	69350
20	7300	14600	21900	29200	36500	43800	51100	58400	65700	73000
21	7665	15330	22995	30660	38325	45990	53655	61320	68985	76650
22	8030	16060	24090	32120	40150	48180	56210	64240	72270	80300
23	8395	16790	25185	33580	41975	50370	58765	67160	75555	83950
24	8760	17520	26280	35040	43800	52560	61320	70080	78840	87600
25	9125	18250	27375	36500	45625	54750	63875	73000	82125	91250
26	9490	18980	28470	37960	47450	56940	66430	75920	85410	94900
27	9855	19710	29565	39420	49275	59130	68985	78840	88695	98550
28	10220	20440	30660	40880	51100	61320	71540	81760	91980	102200
29	10585	21170	31755	42340	52925	63510	74095	84680	95265	105850
30	10950	21900	32850	43800	54750	65700	76650	87600	98550	109500

Keep using the hoist.


Foresee axis and spare parts replacement.

Replace axis and spare parts.


**IMPORTANT!**

Contact your dealer for more detailed information on maintenance procedures and available spare parts.


1/ Unscrew the Ø10 stop-nut.  
2/ Unscrew the axis and remove it then pick up the pole.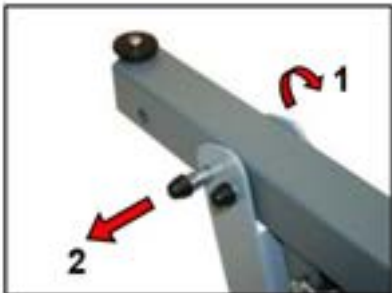

3-4 / Engage the axis and screw it in until it stops.  
5/ Lock it with the safety β pin or with the Ø8 stop-nut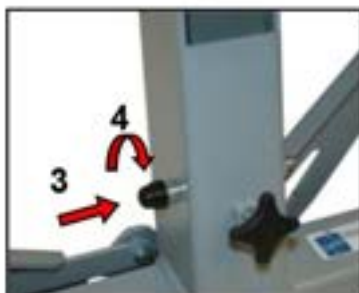

6/ Unlock the beam from the support beam and pick the support beam up.  
7/ Fix the head of the electric actuator on the support beam in locking the safety ring.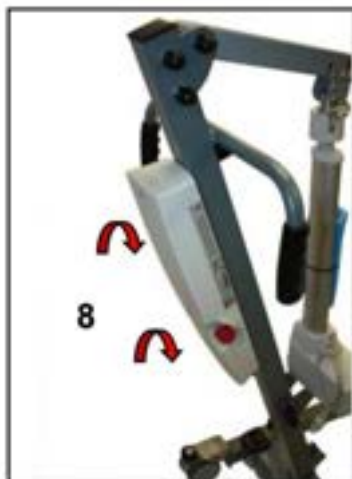

8 / Fix the control box to the pole in screwing the 2 cross screws.  
9 / Plug the remote control and the electric actuator to the control box (see chap. 3.3).  
**Your Lifty4 hoist is now ready to run.**

**⚠ IMPORTANT WARNINGS :**

- NEVER USE THE HOIST WITHOUT LOCKING THE AXLE FIXING THE POLE WITH THE SAFETY β PIN OR THE Ø 8 STOP-NUT.


- BEFORE ANY USE CHECK THAT THE AXLE FASTENING OF THE ACTUATOR IS PROPERLY POSITIONED AND LOCKED WITH THE SAFETY RING.

## PLUGGING

Plugging/blocking the actuator cable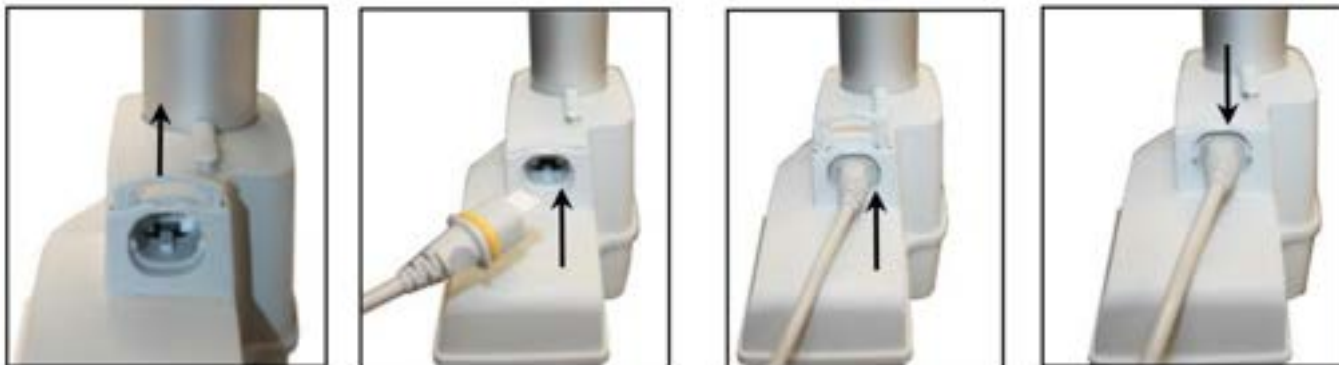

1/ Take out the lock clip

2/ Put the cable plug in position and lean on it to plunge into the lodging.

3/ Push the clip to lock the cable

Anti-Pulling up cable kit.

Plug the remote control and the actuator cables and plug the shutter.

Put the anti-pulling up hood in placing the cables through the opening.

Screw the hood to the control box with a crossed screw-driver.

## FIRST USE

Even if battery box is immediately ready to run as it is charged when delivered, make sure the hoist installation has been strictly done as shown on the manual. Test it off load first to get used to it.


## BATTERIES

The CONTROL UNIT is fixed by sliding it, from top to bottom, on parallel supports on the SUPPORT COLUMN. The movement is fluid and does not require blocks / hooks. Important! Get to the end of the line.

## SLING

Reminder : Sling is adjustable. User is responsible for adjusting it in order to not hurt the patient. Lifty4 hoist is delivered with a 20015 sling. See the user instruction bellow.

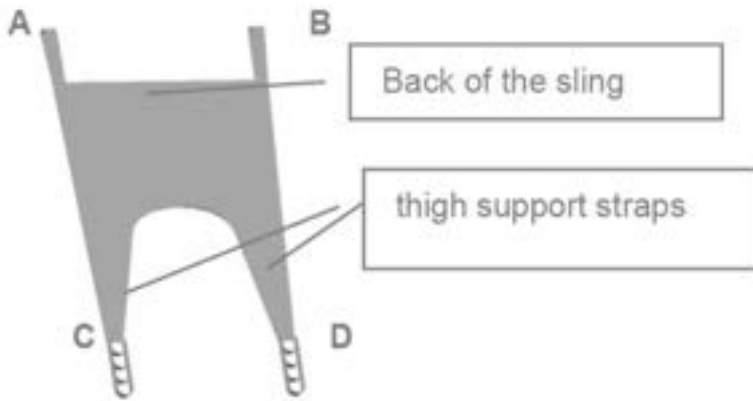- 1/ Put the back of the sling (largest part of the sling) on the patient back in order to have the 2 hanging-up straps on both sides of the patient.
- 2/ Hang up each hanging-up strap to the beam hooks (A1 and B2)
- 3/ Softly lift up the patient right thigh to put the right thigh support underneath.
- 4/ Softly lift up the patient left thigh to put the left thigh support underneath.
- 5/ Cross the two thigh support sling.
- 6/ Hang up each strap to the beam hooks (C3 and D4)

→ You can adjust the patient tilt with the different hanging-up holes. Make sure to always hang-up the A and B straps at the same hole level (same colour). Act the same way for C and D straps.

### Cautions :

- This sling is designed for DEMARTA hoists only.
- Always hang-up the 4 straps at the same level.
- Never adjust straps while the hoist is running.
- Always make sure that straps are well-attached on the beam before lifting the patient up.
- Never exceed the maximum running load.

### Transferring the patient:

- 1/ Set the hoist in a way that the beam stands above the patient (if necessary, open or close side foot to get closer to the patient).
- 2/ Lock brakes
- 3/ Set the sling
- 4/ Push up button  of the remote control to lift the support beam (make sure the strap does not get stuck and does not hurt the patient while lifting him).
- 5/ Unlock brakes to transfer the patient.
- 6/ Lock brakes again when transfer done.
- 7/ Push down button  of the remote control to go down.

## SAFETY

Crushing safety: Lifty4 runs with a single-acting actuator. Any obstacle on its way will automatically stop its going-down.

Emergency stop : the Lifty4 emergency stop button is located on the control box.

Push it to stop the lifting or going-down. Push it again to unlock it.


Electric safety : lifting and going-down are locked when Lifty4 is connected to charge battery.

Emergency going-down (only use when patient is stuck above and that it is impossible to go down using the remote control) : There are two actuator release systems:

1/ Electrical release :

To go down, push the blue switch located under the emergency stop.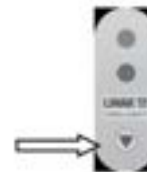

2/ Manual release :

Push down (1) and turn to the right (2) the red ring of the actuator.

This function can be used if the electrical system is not efficient.

A third person must assist the patient during the entire emergency going-down process.


## CONTROL BOX OPERATING, BATTERY CHARGE

If the charging level is too weak, an alarm sounds (in a constant way) when the control box is activated. The charging of the battery is then necessary.

The control box has its own integrated charger. To charge batteries, connect the control box to the 220 VAC main power line, until the green led turn on indicating batteries are charged (approximately 12 hours).

Warning, the loading of the batteries works only if the emergency button is out of power (put it on the

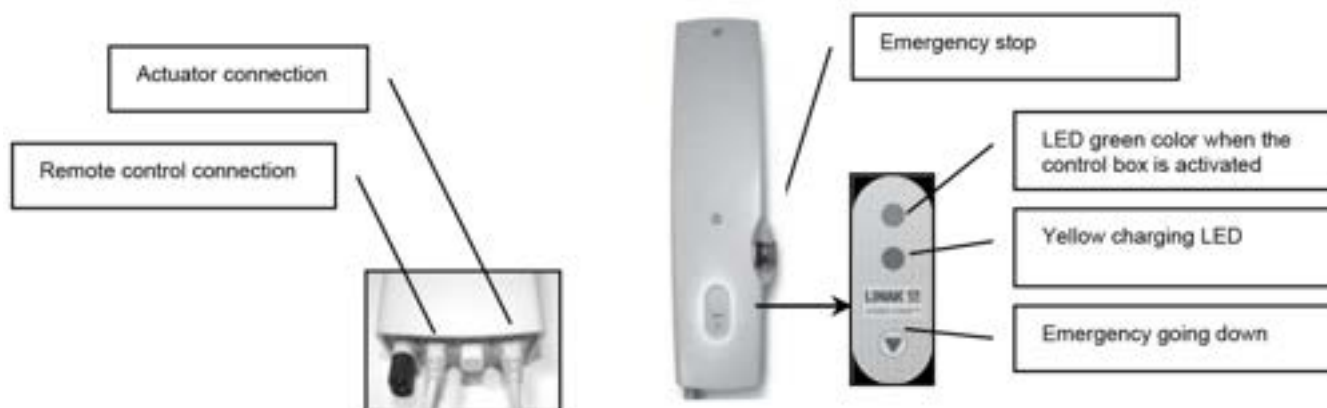To make batteries last longer, they must be recharged every night.

## STORAGE INSTRUCTIONS

Do not expose to moisture.

Store in a temperate dry place (5°C to 40°C).

Charge battery before using it after a long time off.

Handling and stocking of the lift has to be done in order to avoid any impact, falling or damage of the material.

## WEEKLY MAINTENANCE

### Sling :

Check the sling by looking at its seams condition: immediately replace a sling that starts to show any tear.

### Electrical parts:

Test the emergency stop switch while lifting and lowering the support beam.

Look at the electrical cables cover.

Test crushing safety system by placing an item on the support beam going-down way.

### Chassis :

Check that the safety systems are correctly installed : safety ring on the actuator/support beam fixing axis and safety  $\beta$  pin on the pole axis fixing.

Check the brake of the back wheels.

Look at the global geometry on the Lifty4 (no bent item).

Make sure that the 4 wheels are well positioned on the ground.

Make sure that there is no play between the support beam and the pole.

## MAINTENANCE

This product is a mature hoist that requires regular maintenance. Maintenance is very important, and if you follow the instructions in this section for maintenance checking, you can use this product for years with ease. You can do some routine checking by yourself, while others are assisted by dealers. If any problems with the maintenance or operation of this product, please contact the dealer of this product. Warning! This product, like most electrical or mechanical equipment, is susceptible to the natural environment. In any case, it should be avoided the humid environment. Directly contacting with water or humid environment may cause malfunction of the electrical equipment or mechanical equipment of the hoist. Water may cause corrosion of the electronic components and the frame may rust!

If the product is exposed to water, please perform the following maintenance:

1. Dry the water as much as possible with a rag and place it in a dry place;
2. Perform all safety and operational tests prior to use;
3. If there is any abnormality, please contact the dealer of this product.

## GENERAL GUIDANCE

1. Avoid knocking or hitting the electric control box, especially the controller handle;
2. To avoid prolonged exposure to harsh conditions, such as overheating, too cold or humid environment and keep the electric control box clean;
3. Check all electrical connections to ensure they are securely connected and secure;
4. When the lamp of the battery status table is fully illuminated, the battery is fully charged, and the controller and electrical system are working normally;
5. If there is only one space on battery status table, it indicates that the battery is low and needs to be charged, but the electrical system works normally;
6. Check all cable connections to make sure they are securely connected and free of corrosion;
7. All wheel bearings are lubricated and sealed. They do not require the addition of lubricating oil.

## PRODUCT INSPECTION

Daily Inspection :

1. Visually inspect the lift for external damage or wear;
2. Check if the screws and nuts on the boom, electric push rod and base are tight;

3. Check the hooks and slings for damage;
4. Check if the manual control, electric push rod and width adjustment function are normal.
5. Check the emergency descent function. If you have any questions, please contact your dealer;

Weekly Inspection :

Disconnect the manual connector and the electric push rod connector from the electric control box, check the connection and check for corrosion. If necessary, please contact the dealer;

Ensure that all screws are securely in place and that there are no breaks or bends;

Check if the brake function is normal.

Monthly inspection:

1. Check the wear of the universal wheel. If maintenance is required, please contact your dealer.
2. Check the four wheels for wear and looseness, which may indicate that adjustments are needed or the bearings need to be replaced. Please contact the dealer for repair or replacement;
3. Keep the product clean and free of debris, such as hair, food, beverages, residues, etc.

Other related circumstances

Temperature (some parts of the lift are susceptible to temperature)

Charger: At extremely low temperatures, the battery may freeze. There are many factors that may cause freezing at special temperatures, such as charger type, usage, and battery composition (such as sealed lead-acid batteries or gel batteries);

High temperatures may slow the lift. The safety of the electrical box protects the motor and other electronic components from damage.

## CLEANING

Never rinse the product with water or contact it directly with water.

This product can be easily wiped clean with a damp cloth. Never use chemicals to clean the sling. This may cause corrosion and tearing of the sling. Clean it with a wet towel and a mild soap solution until it is completely dry.

## FAULTS AND REMEDIES

The following table shows common faults and corresponding solutions. If you still can't solve it, please contact an authorized company.

Faults	Remedies
Do not work	Check if the emergency stop button is pressed Check if the wires of the manual or electric push rod are connected correctly. Check if the battery is seriously low. Check the battery contact panel for problems or damage. Find an authorized supplier
Can not Charge	Check if the charger is plugged into the socket Check the battery contact panel for problems or damage. Find an authorized supplier of DEMARTA
Suddenly stop during lifting	Check if the emergency stop button is pressed Use emergency descent device Find an authorized supplier

## GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

## TREATMENT OF EXHAUSTED BATTERIES (DIRECTIVE 2006/66 / CE)

### BATTERY REGISTRATION NUMBER IT21040P00006953

The battery contained in the appliance must not be disposed of as urban waste (Directive 2006/66 / CE), as it is subject to separate collection to avoid damage to the environment.

The user must therefore give the exhausted battery-accumulator free of charge to the appropriate municipal centers for the separate collection of electrical and electronic waste, or deliver it to the special containers provided by the points of sale.

To remove the battery, open the door on the control unit, remove the battery by sliding it out of its housing.

## WARNINGS FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2012/19 / EU. AEE REGISTRATION NUMBER IT21050000012993

At the end of its useful life, the product must not be disposed of with municipal waste.

It can be delivered to the appropriate separate collection centers set up by municipal administrations, or from retailers that provide this service. Dispose of

separately, the product avoids possible negative consequences for the environment and for health deriving from its inappropriate disposal. Allows also materials to be recovered in order to obtain an important saving of energy and resources. For point out the obligation to dispose of electromedical equipment separately on the product the mark of the crossed out mobile rubbish bin is shown.

## WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.