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USER MANUAL 4 IN 1 COMMUNE CHAIR-STEEL ROCCIA

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

The 4 in 1 commode chair is a useful aid for carrying out the daily hygienic functions of disabled and elderly people. Thanks to the various functions it can be used in healthcare facilities and at home.

WARNINGS

- Do not exceed the maximum capacity (120 Kg).
- Use with caution on wet and slippery surfaces.
- Always assist the patient during handling operations.
- Before transferring the patient from / to the comfort, make sure that the device is placed on a flat and stable surface.
- Avoid leaving the device outside.
- Periodically check the stability and strength of the product as well as the condition of the support tips. Replace them when worn.

DESCRIPTION

The **4 IN 1 COMMODE CHAIR ROCCIA** has a steel frame. It is foldable and height adjustable and easy to assemble.


It can be used as:

- toilet lift,
- toilet support,
- commode chair,
- shower seat.

TECHNICAL FEATURES

Code	Length	Width	Floor to Seat Height	Weight Capacity	Net Weight	Carton Size
Measures in Cm						
01.04.32	51	54	45/55	120	5.7	62x35x51

ASSEMBLY

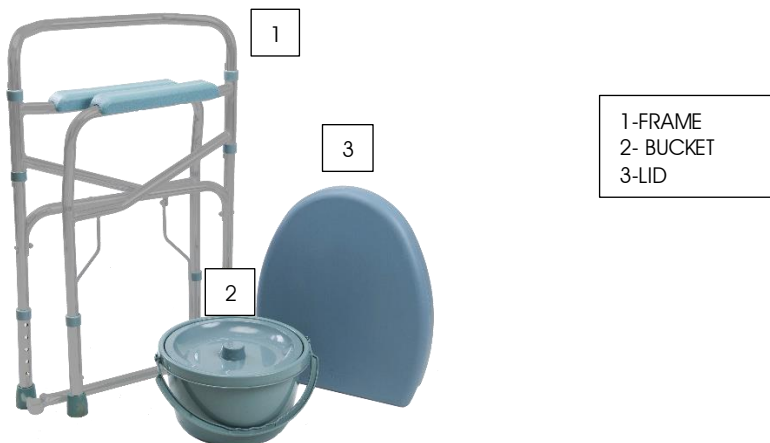Assemble the frame (1): Position the legs of the frame.

Hook the central bracket.

Place the bucket (2) and lid (3) on the frame just assembled.

USE

- Sit by spreading the weight over the entire seat. Do not sit pushing on one side only.
- Always lean on both armrests to stand up and maintain balance.
- In case of use of the chair by people with particular mobility difficulties, the healthcare staff or caregiver will always have to help the user.

CLEANING

We recommend cleaning once a week with a simple detergent diluted in water
Do not use corrosive products, acids, gasoline or similar
If necessary, disinfect the product using a gentle disinfectant detergent
The toilet bowl must be cleaned after each use

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems. It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.