MANUALE D'USO BARELLA DOCCIA

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USER MANUAL SHOWER STRETCHER

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly. The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

1. Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
2. Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
3. Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
4. DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
5. Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation

Introduction

The 02.01.BDI001 and 02.01.BDE001 shower trolleys are a valuable aid for the ergonomic and accurate washing of patients who have problems staying seated under running water. Comfort is optimal for both the user and the operator.

The stretchers consist of a mobile base structure and a height-adjustable support surface with a fireproof and water-repellent mattress.

The shower stretchers have been designed with particular attention to sturdiness, operational simplicity and manageability. Proper use and adequate maintenance guarantee years of trouble-free operation.

Safety

	Use the device for washing and disinfecting human beings only. Do not direct streams of water mixed with disinfectant towards the patient.
	Use the device for washing and disinfecting human beings only. Do not direct streams of water mixed with disinfectant towards the patient.
	When opening and closing the side rails, make sure that the patient's hair, arms and feet do not interfere with the moving parts and that the wheels are locked.
	During any movement, make sure to keep the patient's hair, arms and feet within the outline of the stretcher.
	To prevent the patient from falling, never leave him unattended on the stretcher. If you leave temporarily, make sure that the brakes and sides are locked.
	Use only original spare parts. Failure to comply with this warning can jeopardize the correct operation of the stretcher and invalidate the guarantee.
	To clean and sanitize the stretcher, use non-aggressive detergents and disinfectants. NEVER clean with ammonia, acids, abrasive detergents. NEVER use sponges or metal brushes. These products could cause irreparable damage to the device.
	Public or private operators, who in the exercise of their activity detect an accident involving a medical device are required to notify the Manufacturer and the competent authority of the Member State in which the user and / or patient is established. Public or private health workers are required to communicate to the Manufacturer any other inconvenience that may allow the adoption of measures to ensure the protection and health of patients and users.

Features

General features


Structure entirely made of galvanized steel for optimal resistance to mechanical stress and corrosion
Wheelbase dimensioned for maximum stability in both static and dynamic conditions

Large diameter wheels for greater ease of sliding, with brake and rotation lock

Ergonomic pedals on both sides for height adjustment by hydraulic pump (02.01.BDI001)

Height adjustment by electric actuator operated by remote control (02.01.BDE001)

Technical features

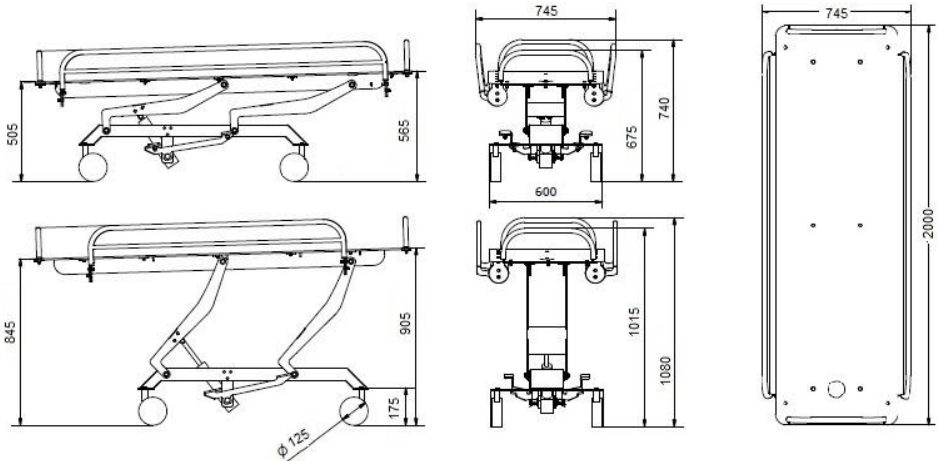External dimensions	2000 mm x 745 mm x 740/1080 mm (l x p x h)
Usable area	circa 1925 x 695 mm
Drain water	1,5 m x \varnothing 40 mm
Wheel diameter	125 mm
Weight	93 kg
Maximum capacity	190 kg
02.01.BDE001 - Power supply (input)	100-240 VAC 50/60 Hz, 0,28 A max
02.01.BDE001 - Power supply (output)	24V CC – 4,5A max (attuatore) – 500mA max (centralina)
02.01.BDE001 - Work cycle	10% max, 2min/18min
02.01.BDE001 - Class Ip	IPx5

Unpacking, assembly and commissioning

The stretcher is supplied on a wooden pallet, protected by cardboard packaging.

Make sure that the unpacking operations are carried out by trained personnel equipped with adequate protection devices.

Open the packaging, taking the appropriate precautions.

Inside the package you will find the shower trolley already fully assembled and this manual. The packaging of the electric version BDE 001 also contains the remote control, the battery and the battery charger.

Cut the fastening straps to the pallet, unlock the wheels of the stretcher, then let it off the pallet.


Connect the remote control and insert the battery on the control unit (BDE 001).

At first start-up, carry out all the commands described in the next chapter, then proceed with the maintenance that follows each use, then repeat all the commands. Check that all movements take place smoothly, without clicks or suspicious noises and that there is no leakage of liquids from the ducts.

Controls

Control unit (02.01.BDE001)

- 1 - Remote control socket
- 2 - Actuator socket
- 3 - Power socket

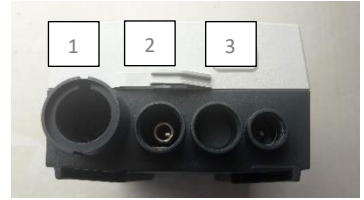Hydraulic raising / lowering of the shower trolley (02.01.BDI001)

The height of the stretcher is managed by acting on the special two-lever pedal.

To raise the stretcher, use your foot to push down and repeatedly release the longer lever, until you reach the desired height.

To lower the stretcher, use your foot to push and hold the shorter lever. The stretcher goes down smoothly. Release the lever when the stretcher has reached the desired height.

Electric raising / lowering of the shower trolley (02.01.BDE001)

1. Up - Press and hold the button to lift.
Release the button to stop lifting.

2. Down - Press and hold the button to lower.
Release the button to stop the descent

Make sure that during the lifting and lowering operations there are no obstacles above or below the stretcher surface. Any obstacles could affect the correct handling of the stretcher and cause pinching or crushing of the parts of the patient's body.
Do not exceed the maximum capacity of the stretcher.

Lifting / lowering of the side rails

The side rails of the stretcher can be easily folded down for safe patient transfer and can be raised for transport.

Bring the sideboard to a vertical position (1), pull the sideboard up to release the two end stops (2) and rotate it downwards by 180 ° (3).

Raise the side rail simply by turning it back 180 °. Pushing the rail downwards the two latches will lock into place.

Wheel brakes

To use the wheel brakes (5), press the red lever downwards with your foot until you hear a "click".

To unlock them, press the raised part of the red lever with your foot.

To increase stability, also activate the green wheel direction lock (6).

Connecting the drain hose to the drain

One end of the outflow tube is equipped with a threaded ring nut. Bring this end to the unloading of the stretcher and connect it firmly by screwing the threaded ring nut to the unloading.

To disconnect the hose, unscrew the threaded ring nut.

Fixing the mattress to the stretcher

To attach the mattress to the stretcher, pass the mattress straps under the stretcher surface and fasten them with the velcro. To remove the mattress, proceed in reverse.

Emergency commands (02.01.BDE001)

Emergency stop (red button)

This command should only be used in an emergency eg. if the stretcher does not respond or does not stop after pressing a button on the remote control.

If the emergency button has been activated, all functions stop immediately. The emergency stop can be released by turning the red button $\frac{1}{4}$ to the right.

Emergency descent (pen button)

If the remote does not respond, insert a pen into the hole and press to lower the stretcher. The battery must be charged to use this command.

Operation

The typical sequence of operations that you will carry out with the shower trolley includes: transfer of the patient - from bed or wheelchair - to the shower trolley, transport from the room to the washing area, washing the patient, transport from the washing area to the room and transfer from the shower trolley to the bed or wheelchair.

Transfer of the patient on the shower trolley

From bed

Make sure the patient bed wheels are locked.

Position the shower trolley with the long side adjacent to the long side of the bed and so that the drain of the stretcher is close to the patient's feet.

Make sure the bed rail opposite the stretcher is raised.

Turn the patient onto his side with his face facing away from the direction of the stretcher.

Unlock and lower the side rail of the stretcher adjacent to the bed.

Raise the stretcher so that the surface is at the same level as the mattress of the bed, bring it as close as possible to the bed, then block the wheels.

Move the patient from the bed to the shower trolley with the aid of a transfer sheet.

Raise the stretcher until the side rail near the bed can rotate.

Rotate the side rail of the stretcher adjacent to the bed and lock it in the raised position.

From the wheelchair

The presence of two assistants is recommended for the operation.

Lock the wheels of the stretcher.

Adjust the height of the stretcher to a minimum.

Lower one of the side rails.

Move the wheelchair to the side of the stretcher with the side rail down.

Remove the armrest of the wheelchair or rotate the footrest leaving the side free for the transfer.

Position the wheelchair centrally with respect to the stretcher, then lock the wheels.

Transfer the patient sideways between the wheelchair and the stretcher, using a patient lift or, if absent, supporting the patient under the arms.

Position the patient comfortably on the stretcher, then close the side rail.

Unlock the wheels of the wheelchair and move it.

Patient transport

Make sure both sides are up.

Adjust the height of the stretcher to the position that is most comfortable for you to carry.

Unlock the wheels of the stretcher. To increase stability in motion, activate the direction lock on the wheel equipped with this function.

Patient washing

Position the shower trolley in such a way that the drain hose can reach a water drainage point.

Lock the wheels of the stretcher.

Adjust the height of the stretcher to the position that is most comfortable for you to wash.

Free the drain pipe and place it in the drain point.

Make sure the water temperature is adequate.

Wash the patient by keeping him in the supine position with the help of a shower hose.

After washing the patient, wrap him in a bath towel to prevent him from getting cold.

At the end of the wash, make sure that most of the water has drained, then fasten the drain hose again to prevent dripping during transport.

Transfer of the patient from the shower trolley

Reverse the actions described for transfer to the shower trolley.

Maintenance

For each use

After each cycle of use, carefully wash the stretcher, the mattress, the pillow and the drain hose.

Use the special sanitizing panel, if available, otherwise use liquid soap or non-abrasive, chlorine-free detergents and disinfectants.

No solvents of any kind must be used to clean the mattress and the headrest pillow.

Rinse thoroughly with running water.

Eliminate any incrustations or limescale residues with specific products with a mild action.

To clean the stretcher surface, remove the mattress.

Thoroughly dry the mattress and pillow to prevent them from being damaged in the long term by stagnant water in the corners.

The mattress and pillow can be dried with hot air, as long as the temperature does not exceed 80 ° C.

Check that the stretcher wheels are clean and if necessary wash them, carefully removing dirt and hair.

Battery charging (02.01.BDE001)

The shower trolley is equipped with a battery that requires regular charging. Giavazzi recommends recharging the battery every time the shower trolley is not used or every night. The shower trolley can be charged with mains power (wall socket) using the supplied battery charger.

1. Make sure the emergency stop is not pressed.
2. Connect the battery charger to the control unit.
3. Insert the plug of the battery charger into the mains socket.
4. Make sure the power cord is not strained.

The battery charger is equipped with a LED, which is orange when recharging and green when the battery is fully charged.

Weekly maintenance

Check carefully that the mattress and the pillow do not have cracks or breaks that could injure the patient.

Check the correct functioning of the sides: unlocking, rotating down, rotating up and locking.

Monthly maintenance

Check that all screws and fasteners are tight.

Make sure the hydraulic lift system moves smoothly.

Annual maintenance

Check all the moving mechanical parts and lubricate them with silicone spray.

GENERAL DISPOSAL CONDITIONS

In case of disposal of the device, never use the normal municipal solid waste disposal systems. It is recommended to dispose of the device through the common municipal ecological islands for the envisaged recycling operations of the materials used.

TREATMENT OF EXHAUSTED BATTERIES (DIRECTIVE 2006/66 / EC)

BATTERY REGISTRATION NUMBER IT21 040P00006953

The device contains a non-removable battery and must not be disposed of as urban waste, as it is subject to separate collection to avoid damage to the environment.

In compliance with the directives 2002/96, 2006/66 and the national transposition laws for the disposal of the product at the end of its life.

WARNINGS FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2012/19 / EU.

AEE REGISTRATION NUMBER IT21050000012993

At the end of its useful life, the product must not be disposed of with municipal waste. It can be delivered to the appropriate separate collection centers set up by municipal administrations, or from retailers that provide this service. Dispose of separately, the product avoids possible negative consequences for the environment and for health deriving from its inappropriate disposal. Allows also materials to be recovered in order to obtain an important saving of energy and resources. For point out the obligation to dispose of electromedical equipment separately on the product the mark of the crossed out mobile rubbish bin is shown.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date. DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.