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# DEMARTA VIRGINIO

## USER MANUAL FOLDABLE LIGHT TRANSIT WHEELCHAIR

I Class Medical Device according  
to directive UE 2017/745

## INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

## GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

## DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

## USE

The wheelchairs are an essential way of transport for disabled people with none or limited walking ability. The purpose is to facilitate movements and to safeguard the physical well-being of the

person, avoiding the appearance of complications related to the sitting position (bed sores, chronic postural pains ...).

#### **WARNINGS**

- Always pay attention to the presence of moving parts that could cause limb entrapment and injury.
- Pay particular attention to wet surfaces.
- It is recommended to pay attention to the entrapment of the fingers in some parts such as wheels or brakes when using the wheelchair.

Features	
Code	01.04.C02
Frame	Aluminum
Cross Bar	Double
Color	Powder coating
Front Wheels	15 cm (6 ") PVC
Rear Wheels	30 cm (12 ") PU
Armrest	Detachable , PU
Footrest	Foldable
Footplate	Tipper
Seat	Nylon
Brake	United brake
Backrest	Foldable
Dimensions	
Total width (frame + wheels)	61 cm
Width with closed frame	29 cm
Seat height from the ground	46 cm
Total height of the frame	92 cm
Overall frame length	94 cm
Backrest height	44 cm
Seat dimension	46 x 36.5 cm
Wheelchair weight	10.8 Kg
Box dimension	73 x 29 x 70 cm
Maximum capacity	110 Kg

## ASSEMBLY

Frame opening and closing:

THE BACKREST TILTS BACK, THIS MECHANISM ALLOWS THE CLOSING OF THE FRAME, THE OVERALL IN THIS WAY IS REDUCED AND EASY TO TRANSPORT

A

As shown in figure A, press the button outward to position the handle (as shown in figure B).

B

1. Hold the handrail firmly to push it in the direction of the arrow (as shown in figure A).
2. Press the handrail firmly to support when the handle is pulled up. When the handle is locked and secured, you will hear a click.

### Frame opening:


Grasp the armrests, pull them outwards, push the seat down.

### Frame closure:

Lift the seat and close the wheelchair bringing the two armrests closer together.

### FOLDING FOOTPEGS

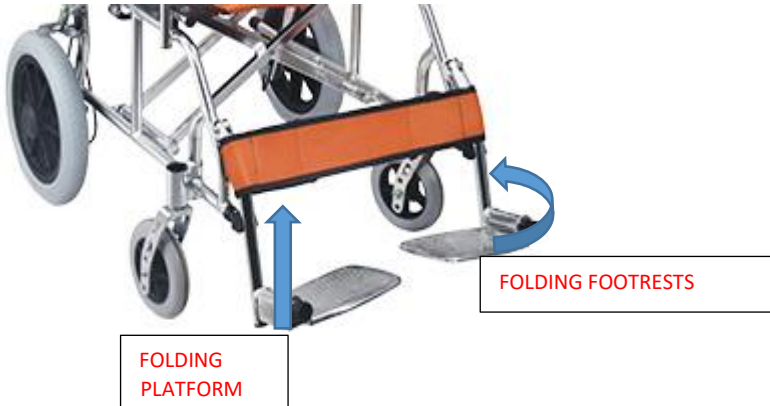### USING THE BRAKES

TO BE ABLE TO LOCK THE BRAKES, PUSH THE LEVER IN BASSO

USE THE GRIPS WHEN BRAKING NEEDED

## USE

### Get on the wheelchair

- Move the wheelchair close to the patient.
- Lock the brakes of the rear wheels; it is advisable to remove the armrest from the side. Lean on the frame, making force on the arms, then slowly go down towards the seat.

### Get off the wheelchair

- Approach as close to a new seat as possible.
- Lock the brakes of the rear wheels; remove the armrest from the side. Rest your arms on the frame of the seat on which you must move and then lift slowly towards it.

**IMPORTANT!** Assistance or supervision of qualified personnel is necessary to avoid slips and falls.

## Movement

The healthcare staff or the caregiver can push the wheelchair by the knobs on the backrest.

## CLEANING

Use a damp cloth with mild soap, the water temperature not above 30°C. Dry well before use. If necessary, disinfect the product using a common disinfectant detergent. Do not use water and/or steam jet washing machines. Must not use acid, alkaline or solvent products such as acetone.

## GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

## WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.