MANUALE D'USO

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USER MANUAL
2 WHEELS ONE BOTTON
FOLDING WALKER
01.06.210B5

INTRODUCTION

Thank you for choosing one of our products. The walkers have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

USE

The walkers are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance.

This aid is designed for a single user, in indoor environments.

SPECIFIC WARNINGS

- The walker should only be used by patients with a suitable motor capacity which ensures their use without risks.
- Advise with physician or therapist to determinate the correct adjustment and the correct use.
- Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly.
- The walker can be used only on a hard, flat surface.
- Do not use the walker to climb the stairs.
- Pay particular attention to wet surfaces.
- All the tips and wheels must be in contact with the floor at any time during use to ensure the proper balancing of the walker and maximum safety.

DESCRIPTION

- Foldable and height-adjustable aluminum structure.
- Two fixed front wheels (\varnothing 12.7 cm) and two rear anti-slip tips.
- Opening and closing of the two side sections of the frame are done by means of the central locking pins.
- Ergonomic and padded knobs.

FRAME

Code	Length	Width	Height	Weight Capacity	Net Weight	Carton Size
Measures in Cm						
01.06.210	52	58	80/97	120	2.4	58x15x80

ASSEMBLY


- On first assembly, insert the legs of the walker in the housings that you find on the frame.
IMPORTANT: the wheels must be placed in the front part, while the tips on the back.
- To open the frame, press the central locking pins until the side supports are completely open.

- Adjust the height of the walker before use. Press the button on the legs of the walker and pull up or down the lower legs to find the hole that corresponds to the most appropriate height.

Do the same operation on all 4 legs so they are at the same height.

- Check that the rubber tips and wheels are correctly fixed.
In the event that they are worn, they must be replaced immediately.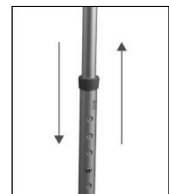

USED

1. Start pushing when you are in an upright position with both hands on the handle.
2. Make sure your hands are dry before using the walker to avoid the risk of slipping.
3. Walking maintaining an upright position with his eyes facing forward and not downward.
4. Do not push the walker too far away from the body, this may cause falls.
5. Do not use the walker to overcome steps.

CLEANING

Use a damp cloth with mild soap, the water temperature not above 30°C. Dry well before use. If necessary, disinfect the product using a common disinfectant detergent.

Do not use water and/or steam jet washing machines. Must not use acid, alkaline or solvent products such as acetone.

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.