MANUALE D'USO TRIPODI E QUADRIPODI

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USER MANUAL
THREE LEGS WALKING CANE
FOUR LEGS WALKING CANE

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

USE

The canes are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance. This aid is designed for the use of a single user, both in indoor or outdoor environments.


WARNINGS

1. The canes should only be used by patients with a suitable motor capacity which ensures their use without risks.
2. Advise with physician or therapist to determinate the correct adjustment and the correct use.
3. Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly.
4. The canes can be used only on a hard, flat surface.
5. Do not use the walker to climb the stairs.
6. Pay particular attention to wet surfaces.
7. Do not overcome the maximum capacity of 120 kg.

DESCRIPTION



The THREE LEGS WALKING CANE has a lightweight aluminum frame adjustable in height (9 levels), an ergonomic and soft handle for a more comfortable grip. 3 legged base with non-slip points. Maximum load: 120 Kg.

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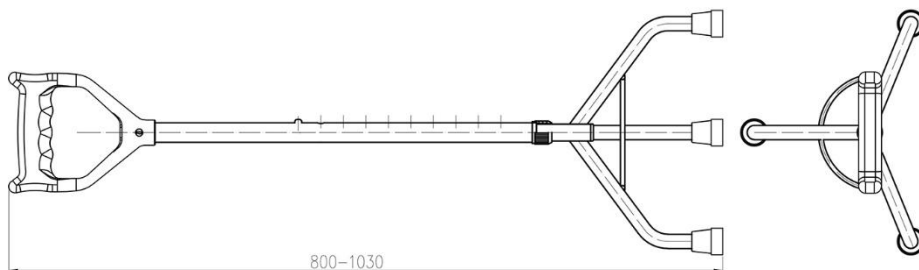The 4 LEGS WALKING CANE has a lightweight aluminum frame adjustable in height (10 levels), an ergonomic and soft handle for a more comfortable grip. 4 legged base with non-slip points. Maximum load: 120 Kg.

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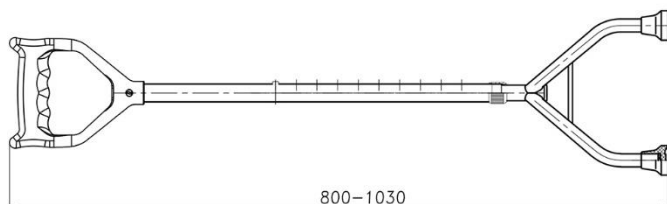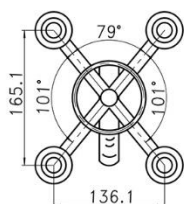Materials	
Frame	Aluminum
Tips	Anti-slip Plastic

CLEANING

Use a damp cloth with mild soap, the water temperature not above 30°C. Dry well before use. If necessary, disinfect the product using a common disinfectant detergent. Do not use water and/or steam jet washing machines. Must not use acid, alkaline or solvent products such as acetone.

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.