DEMARTA
VIRGINIO

MANUALE D'USO
ASCOT CON COPERCHIO
CODICE 02.02.675

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USER MANUAL
ASCOT WITH COVER
SKU 02.02.675

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

Bath safety are all those devices that can be used by a disabled or elderly person to prevent, compensate or eliminate the difficulty in performing all the functions in the bathroom.

SPECIFIC WARNINGS

- The life of the device is determined by the wear of parts that cannot be repaired and / or replaced.
- Use with only one patient is recommended.
- Sanitize the device before each use.

TECHNICAL DATA

Code	Height (cm)	Capacity (Kg)
02.02.675-2	5	130
02.02.675-4	10	130
02.02.675-6	15	130

DESCRIPTION

The raised toilet seat ASCOT consists of an ergonomic washable PVC seat. It has a front and a back opening to facilitate accessibility for personal hygiene. The fixing system to the toilet is made of 2 adjustable side holders.

INSTALLATION

- Take the toilet seat upright and assemble it by inserting the two brackets into the appropriate side guides.
 - Turn the screw knobs clockwise to secure the upward WC brackets.
 - Lift the lid and the toilet seat and place the riser on the ceramic, making sure to fit the edge of the toilet between the two edges of the rise.
 - Screw the knobs of the brackets clockwise until you find a slight resistance.
- Check that the riser is completely fixed to the toilet.

PRECAUTIONS

- Do not exceed the maximum capacity.
- Before use, always make sure the toilet seat is well fixed to the toilet and that there is no damage that could compromise the stability of the product.
- Caution in case of wet or slippery floor.

CLEANING

Washing can take place with warm water and neutral soap

Disinfection can take place with simple products of normal domestic use

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.