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USER MANUAL
SHOWER CHAIR WITH BACKREST
AND ARMRESTS

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

Bath safety are all those devices that can be used by a disabled or elderly person to prevent, compensate or eliminate the difficulty in performing all the functions in the bathroom.

TECHNICAL DATA

Code	Capacity (Kg)
02.02.799	120


DESCRIPTION

The SHOWER CHAIR WITH BACKREST AND ARMRESTS has a height adjustable aluminum frame. It has eight different height positions, a rectangular seat with holes for water drainage, backrest and side handles. The assembly does not require any type of tools.

INSTALLATION

- 1. LEGS X 4 PCS
- 2. SITTING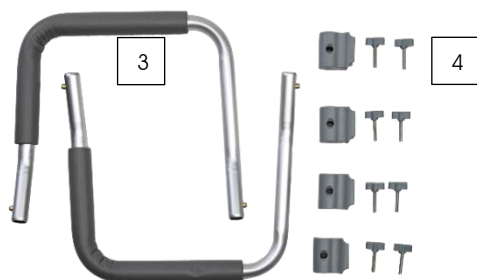

- 3. ARMRESTS X 2 PCS
- 4. LOCK WITH SCREWS X 4 PCS

- 5. BACKREST
- 6. BRACKETS FOR BACKREST X 2 PCS

INSTALLATION

A. Insert the 4 legs (1) into the slots on the seat (2).

B. Hook the block and screws (4) onto the seat.

C. Hook the backrest (5) to the seat with the brackets (6).

D. Attach the armrests (3) to the mounted supports, see step b. on the previous page.

PRECAUTIONS

- Do not stand on the Folding Bathroom/Shower Seat.
- Clean after each use and allow to dry before storing.
- Store in a cool, dry place.
- This is not a toy, keep away from children.
- Do NOT use if the support bar is not securely locked in place. Risk of collapse.

CLEANING

Washing can take place with warm water and neutral soap
Disinfection can take place with simple products of normal domestic use

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.