## MANUALE D'USO OBA HOSPISOFT

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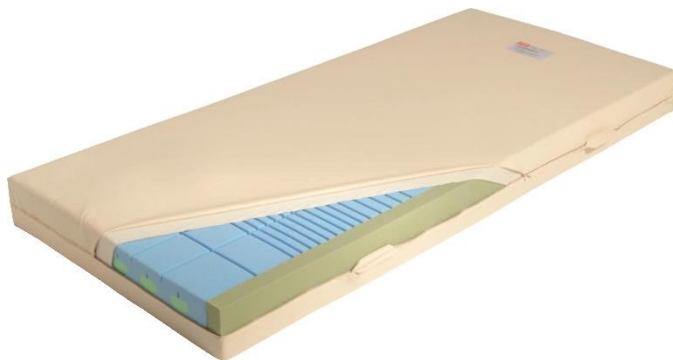
USER MANUAL  
OBA HOSPISOFT

## INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

## GENERAL WARNINGS

1. Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
2. Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
3. Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
4. DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
5. Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

## DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

## INTENDED USE

Device designed for the prevention of bedsores in subjects who have to maintain the lying position for a long time.

## IMPROPER USE

Improper use of the product means use in the following conditions:

- anything not specifically by this manual;
- non-compliance or use contrary to current safety standards;
- failure to comply with the manufacturer's instructions;
- patient handling and transport;
- in the absence of specific medical prescriptions;
- cleaning with unsuitable products;
- unsuitable storage.

## LIMITATIONS

The product has limitations due to the type of material in which is made and medical prescriptions.

## TECHNICAL FEATURES

Mattress	Dimensions (cm)	Cover	Capacity (kg)
05.05.OBAH	200 x 90 x 15	Tricot Sanilon	150
05.05.OBAHTX	200 x 90 x 15	Obatex	150

**TRICOT SANILON**, soft and durable with zipper closure on three sides cover.

**OBATEX**, breathable, waterproof, antibacterial and fireproof cover; made of polyurethane and polyester, it can be sterilized in an autoclave.

## DESCRIPTION

The HOSPISOFT OBA mattress is made of high quality special fire retardant foams; the central part, with a density of 40-45 Kg/m<sup>3</sup>, is characterized by horizontal and vertical ventilation channels, thus guaranteeing optimal ventilation. The side edges are reinforced so as to be able to stabilize the patient in a sitting position and to help him reach the upright position.

## CLEANING

Cover Cleaning	05.05.OBAMEM	Tricot Sanilon
		Water Temperature not Above 95°C or 200°F, Delicate
		Do not Bleach
		Maximum Temperature 150°C or 300°F
		Dryclean with any Solvent
		Tumble Dry Medium Heat

Cover Cleaning	05.05.OBAMEMTX	Obatex
		Water Temperature not Above 95°C or 200°F
		Do not Bleach
		Do not Iron
		Dryclean with any Solvent
		Tumble Dry Low Heat
Sterilization is possible in autoclave up to 134° C, rubber cycle		

Matress Cleaning
Possible cleaning with a damp cloth and neutral detergent
Before use dry the mattress carefully without exposing it to direct sunlight

## MAINTENANCE

The product does not require special maintenance actions, it is necessary to pay attention to a correct cleaning and to a correct conservation as indicated in this manual.

## STORAGE

In case of storage for prolonged periods of time, it is advisable to use the plastic packaging supplied. In order to protect it from dust and moisture.

## GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

## WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.