MANUALE D'USO  
POLTRONA ESSENTIEL

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## USER MANUAL ESSENTIEL LIFT CHAIRS

## INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

## GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

## DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

## INTENDED USE

Devicare lift chairs are indicated to facilitate daily difficulties in elderly and disabled people. The main help function is the lift-mode, which gives the possibility to reach the upright position in safety. The frame allows the backrest to be tilted and the footrests to be raised.

## WARNINGS

- For correct use of the device, refer carefully to the following manual.
- For correct use of the device always consult your doctor or therapist.
- Always pay close attention to the presence of moving parts that could cause limb entrapment and personal injury.
- Keep the packed product away from any source of heat as the packaging is made of cardboard.
- Always pay close attention to the presence of children.
- Immediately disconnect the electric cable from the mains if operating irregularities are found and check for any electrical leaks.
- Do not unplug or insert the socket with wet hands.
- The lift chair must be mounted on a horizontal surface. Avoid loose bumpy plans.

## DESCRIPTION

Features
Frame tilt / footboard elevation: <ul style="list-style-type: none"><li>• electric with 2-button remote control and battery kit (2 weeks autonomy)</li><li>• manual</li></ul>
Armchair with tilting frame (inclination up to 30 °)
4 wheels: Ø 125 mm with brakes
Padded backrest, seat and armrests
Removable outer cover: velvet, synthetic leather, polyester fabric
Table included
Height-adjustable footrests
Indoor use only
Maximum load: 130 Kg

Brown • Velvet

Cocoa • Leather

Gray • Fabric

## Electric Mode

Code	Color	Material
01.11.Z3-VME	Brown	Polyester Velvet
01.11.Z3-PCE	Cocoa	Leather
01.11.Z3-TGE	Grey	Polyester Fabric

## Manual Mode

Code	Color	Material
01.11.Z3-VMM	Brown	Polyester Velvet
01.11.Z3-PCM	Cocoa	Leather
01.11.Z3-TGM	Grey	Polyester Fabric

## TECHNICAL FEATURES

Dimensions	
Seat height from the ground	48 cm
Seat width	38 cm, 44 cm, 50 cm, 56 cm, 65 cm
Seat depth	47 cm, 48 cm, 49 cm
Total width (armrests + seat)	Seat + 26 cm
Weight	38 Kg

## USE

The attendant must always move the wheelchair by using the push bar and keeping it slightly reclined to avoid the footrest to touch the ground.

These wheelchairs are not designed to be used as a seat in a motorised vehicle.

The ergonomic shape of the shell-framed wheelchair and its various accessories allow a full-scale utilisation by people suffering from those following handicaps:

- paralysis
- weakness/malformation of articulations
- lesions of articulations
- diseases such as heart failure and circulatory deficiency, balance disorders or cachexia and clinical geriatric signs

Concerning the contraindications, the following factors must be considered:

- weight over 130 kg
- physical and psychological condition
- inadequate living environment

The shell-framed wheelchair is designed for people who lost their muscular strength, therefore it is necessary to be particularly vigilant during its utilization.

It is imperative to choose the model that is the most adapted to the patient's body size.

Before using the wheelchair, verify the proper functioning of the jacks and the brakes, and make sure the harness is fastened.

When a patient is being seated or lifted away from the shell, whatever method is being used, it is imperative to engage the brakes, fold the leg rest vertically, and adjust the boards on the ground to avoid any forward tilting.

Once the patient is seated in the shell, it is compulsory to buckle the body harness and adjust it close to the body in order to avoid the patient to slide under the harness. Once the harness is adjusted, the wheelchair must be tilted backwards to end the sliding effect.

Once the patient is installed, the wheelchair must be parked, brakes engaged, on a perfectly horizontal ground and after each move, the brakes must be engaged again.

#### A) ENVIRONMENT OF USE

Your wheelchair must only be used on surface areas where all four wheels touch the ground, with sufficient ground contact to allow the wheels to roll in a balanced way.

It is recommended to train oneself if the wheelchair is being used on uneven ground surfaces (cobblestones ...), slopes, curves and to cross obstacles (kerbstones, etc.).

The wheelchair is not designed to be a climbing device or to carry hot or heavy objects.

When used on doormats, carpets or loose floor coverings, the flooring can be damaged.

The manufacturer cannot be held responsible of damages due to absence of or inappropriate maintenance, or non-compliance to the instructions of this guide.

Complying to the utilisation and maintenance instructions is an essential condition of the guarantee.

This wheelchair is to be used under a temperature below 40° and in a dry environment to avoid any risks of corrosion. Should it be in contact with humidity, please wipe it dry after use.

#### B) ON TO AND OUT OF THE WHEELCHAIR

If you cannot proceed with a secured transfer, ask for help.

1. Place the wheelchair as close as possible to the chair, the couch or the bed you want to transfer to or from.

2. Make sure that the brakes of the shell-framed wheelchair are engaged.

3. Untighten the lever of the footrest so that the board touches the ground.

4. Proceed with the transfer to or from the shell-framed wheelchair.

#### C) CARRYING THE WHEELCHAIR

The best way to move the wheelchair is to roll it. If that is not the case, the frame must be firmly grabbed at its front and its back. Do not lift the wheelchair by holding the armrests or the footrests. Two people at least are needed to carry the wheelchair.

When transporting the shell-framed wheelchair, some instructions must be followed:

Carrying the patient seated in the shell-framed wheelchair is absolutely to be prohibited.

Place the seat base horizontally by unlocking the appropriate lever and disassemble the footrest.

The shell-framed wheelchair can now be carried by at least 2 people.

Never lift the shell-framed wheelchair by the wheels, rather grab fixed parts located at the front and the back of the seat base or on the side under the armrests.

Carrying the shell-framed wheelchair in staircases/on stair steps must always be undertaken by two people. Tilting the shell-framed wheelchair is prohibited during transportation.

Nobody and no objects should remain under the shell-framed wheelchair during transportation. It may cause injuries or damages.

#### D) STEERING THE WHEELCHAIR BY AN ATTENDANT

- Risk of burns, be careful when you are moving around in a hot or cold environment (light of the sun, extreme cold, saunas, etc.)

- When remaining a certain amount of time in such environments, the components may reflect its temperature.

- The attendant must set the wheelchair in motion with the push bar.

#### E) IN MOTION ON A SLOPE

The attendant must control its speed and move forward as slowly as possible on sloping grounds.

- Take into account the attendant's capacity – if he is not strong enough to control the wheelchair, pull the brakes.

- If the wheelchair is equipped with a safety belt, use it.

- Do not try to cross sharp sloped grounds.

## F) LIFESPAN

The expected lifespan of this product is five years when used on a daily basis and in compliance with safety instructions, periodic maintenance and appropriate use, as indicated in this user's guide. The actual lifespan depends on the frequency and the intensity of use.

## PRODUCT DESCRIPTION

### A) HEADREST

- The headrest is removable and adjustable in height.

### B) STRAPS

- The lap belt
- The 4-point harness

#### The lap belt

The wheelchair is equipped with a lap belt with adjustable and removable straps. Using the lap belt ensures a better positioning in case of muscular weaknesses leading to a bad posture and offers additional security against risks of fall.

Installing the lap belt on the wheelchair:

- Place and fasten the lap belt with the patient's lower back fully settled against the backrest of the seat. Tighten accordingly to prevent the user to slip under the belt and hurt himself.

The lap belt must be fastened. The adjustment by and the vigilance of a third party are necessary. The lap belt being removable, it is important to check at each use that it is correctly anchored to its fastenings.

#### The 4-point harness

Under no circumstances can the straps be considered as a restraint device and therefore strap loops must always remain accessible to the user for locking and unlocking purposes. Using the lap belt ensures a better positioning in case of muscular weaknesses leading to a bad posture and offers additional security against risks of fall.

The harness must be fastened. The adjustment by and the vigilance of a third party are necessary. The lap belt is removable, therefore it is important to check at each use that it is correctly installed.

## THE PUSH BAR

**Positioned behind the backrest, it allows you to guide the chair.**

## Remote control's using of the ELECTRIC NOSTRESS

NOTES: When operating the controls, you must be able to view the surrounding area in order to prevent people, animals or things from remaining under the moving parts.

NOTES: Do not lean or apply force downwards when the backrest is tilted. Risk of tipping over and injury.

NOTES: The lift chairs is not designed to move the user.

Move the lift chairs only when nobody is on it.

### FRONT AND REAR WHEELS

- Always check that your wheels are free of dirt before using the wheelchair.
- Check that the tyres are not worn before using the wheelchair. Replace them when necessary.

When crossing an obstacle, its height **MUST NOT** be higher than the wheel's radius. If the obstacle is higher than the wheel's radius, the wheelchair has to be lifted.

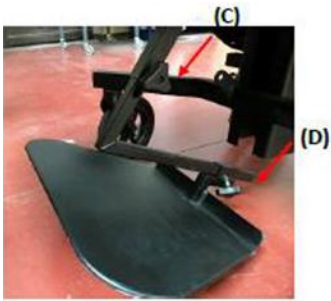
### Brakes


- Pull the brakes by pressing down with your foot (A)
- To release the brakes, lift up with your foot (A)
- These brakes must not be used to slow down the wheelchair.
- Check that the brakes are not in contact with the wheels when in motion.

## FOOTREST

1.

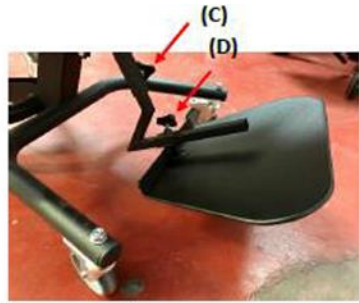2.

Adjusting the footrest

- Untighten the adjuster (C) to allow the adjustment in height of the footrest.
- Tighten the adjuster (D) to hide the footrest under the seat.

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- Always check that nothing lies under the footrest before putting the wheelchair in its upright position to avoid any crushing.
- Always check that the footrest boards are kept flat on the floor.
- DO NOT stand up or put weight on the footrest boards, if they are not kept flat on the floor.
- Always check that the boards are kept flat on the floor before getting seated in or moving out of the wheelchair.
- The footrest is a security component and must not be removed.

## TRAYS

Vertical storage tray (picture 7)-Model 1

- Insert the tray in the tube intended for that purpose on the frame.
- Tighten the adjuster.

Picture 7

Side tray (picture 8)-Model 2

- Insert the tray holder in the openings intended for that purpose in the column.

Picture 8

BODY SUPPORT (picture 9)-Optional

- Fasten the side bar to the push bar.
- Insert the body support(s) in the tube intended for that purpose.
- Tighten the adjuster(s)

## WARNINGS FOR USE

- Do not exceed the maximum permitted load (130 kg).
- Always assist the user during handling operations.
- Connect the electric cable directly to the socket. Do not use an extension cord.
- Before sitting on the lift chair, make sure that the remote control and its wire are not in the way.
- Move your hands and feet away from the attachment points such as the chair and footboard lifting mechanisms.
- When not in use, leave the lift chair in an upright position with the footboard closed.
- If the device does not work or malfunctions, immediately stop use, switch off the device, unplug it and contact the authorized dealer or the Service Center for repair.
- When the device is not used, make sure that the remote control is correctly placed in the appropriate pocket.
- When removing the plug from the socket, never pull on the cable, but take it directly to the plug near the socket.
- Do not use when wearing wet clothing.
- Do not use when the lift chair when is wet.
- Do not use with voltage other than that indicated on the label, also do not enter the plug into multiple sockets.
- Do not use in an environment with a high percentage of humidity such as in bathrooms or outdoors.
- Do not use in case of damage to the electric cable, plug and socket.
- Do not damage, break, modify, bend, pull, twist, wrap the electric cable during use.
- Do not place the power supply cable near heat sources or hot surfaces.

## MAINTENANCE

Checks periodically:

That all screws and bolts are tight.

The mechanical parts (hinges and pins) must be checked monthly and lubricated with a drop of oil in order to promote smoothness and wear resistance.

## CLEANING

The seat quilt is removable and can be machine washed at 30 °C.

We suggest you to put your quilt in a wash bag to protect its original appearance.

All modules can be machine-washed at 30° C

## WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, because these substances cause the abrasion of the components of the lift chair.

## GENERAL DISPOSAL CONDITIONS

In case of disposal of the device, never use the normal municipal solid waste disposal systems. It is recommended to dispose of the device through the common municipal ecological islands for the envisaged recycling operations of the materials used.

## TREATMENT OF EXHAUSTED BATTERIES (DIRECTIVE 2006/66 / EC)

### BATTERY REGISTRATION NUMBER IT21040P00006953

The device contains a non-removable battery and must not be disposed of as urban waste, as it is subject to separate collection to avoid damage to the environment. In compliance with the directives 2002/96, 2006/66 and the national transposition laws for the disposal of the product at the end of its life.

## WARNINGS FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2012/19 / EU.

### AEE REGISTRATION NUMBER IT21050000012993

At the end of its useful life, the product must not be disposed of with municipal waste. It can be delivered to the appropriate separate collection centers set up by municipal administrations, or from retailers that provide this service. Dispose of separately, the product avoids possible negative consequences for the environment and for health deriving from its inappropriate disposal. Allows also materials to be recovered in order to obtain an important saving of energy and resources. For point out the obligation to dispose of electromedical equipment separately on the product the mark of the crossed out mobile rubbish bin is shown.

## WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.