MANUALE D'USO
COMODINA CON INCAVO FRONTALE
COD 01.04.102.43

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USER MANUAL
COMMUNE WHEELCHAIR WITH FRONT
REF 01.09.TS02

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

Compared to a traditional manual wheelchair, the commode also acts as a toilet seat. It is therefore useful during daily hygiene functions of disabled people or elderly.

The commode wheelchair, thanks to the four wheels, can be easily moved by the healthcare staff or the caregiver at home.

SPECIFIC WARNINGS

- Keep the product packaging away from any heat source because it is made of cardboard, it is inflammable.
- Do not exceed the maximum capacity (120 Kg).
- Use with caution on wet and slippery surfaces.
- Always assist the patient during handling operations.
- Before transferring the patient from / to the comfort, make sure that the device is placed on a flat and stable surface.
- Avoid leaving the device outside.
- Always check the state of wear of the parts involved so as to ensure that the device is used in total safety for people and things.

Dimensions	
Total depth of the frame	57.5 cm
Seat size	44 x 44 cm
Seat height from the ground	50.5 cm
Length	111 cm
Backrest height	44.5 cm
Total height of the frame	92 cm
Maximum capacity	100 kg
Weight	13 kg
Features	
Folding frame	Steel
Sitting	Padded
Toilet Seat Closure Cap	Removable
Front quick-release toilet bowl	PVC
Armrests	Tipplers with PVC armrest
Leg rests platforms	Quick extraction

ASSEMBLAGGIO

Frame:

At the first use the frame need to be assembled: insert the two ends of the backrest tubes into the openings located on the rear of the chair. Fasten the backrest using the clips provided.

Footrests:

To install the footrests, align the holes in the supports with their pins on the wheelchair frame, and then rotate the footrest inward by correctly hooking the locking lever into the wheelchair frame.

To remove, release the footrest lever from the wheelchair frame, then turn them outwards and pull upwards.

Brakes:

Press the brake mechanism downwards.

WC Bowl:

The chair need to be fully braked before use.

Take off the removable seat part.

Remove the Wc bowl by grasping it by the handle.

Armrests:

Drop-down armrests, they move downward.

MAINTENANCE

Check list to be carried out periodically.

- Check the brakes: monthly.
- Cleaning of the parts (seat, backrest, armrests, wheels, etc.) as needed.
- Pin lubrication: monthly.

CLEANING AND DISINFECTION

Cleaning

Use a cloth warm water and mild soap and dry with a clean cloth.

Disinfection

If necessary, carry out the disinfection of the product using a delicate disinfectant detergent of normal domestic use.

- The toilet bucket must be regularly cleaned and disinfected using a normal bathroom disinfectant.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, because these substances cause the abrasion of the components of the shower chair.

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems. It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.