## MANUALE D'USO POLTRONA LIBERTY

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## USER MANUAL LIBERTY LIFT CHAIRS

## INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

## GENERAL WARNINGS

- a) Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- b) Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- c) Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- d) DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- e) Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

## DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

## INTENDED USE

Devicare lift chairs are indicated to facilitate daily difficulties in elderly and disabled people. The main help function is the lift-mode, which gives the possibility to reach the upright position in safety. The frame allows the backrest to be tilted and the footrests to be raised.

## WARNINGS

- For correct use of the device, refer carefully to the following manual.
- For correct use of the device always consult your doctor or therapist.
- Always pay close attention to the presence of moving parts that could cause limb entrapment and personal injury.
- Keep the packed product away from any source of heat as the packaging is made of cardboard.
- Always pay close attention to the presence of children.
- Immediately disconnect the electric cable from the mains if operating irregularities are found and check for any electrical leaks.
- Do not unplug or insert the socket with wet hands.
- The lift chair must be mounted on a horizontal surface. Avoid loose bumpy plans.

## DESCRIPTION

Features
Electric model with remote control: tilting of the frame / footboard elevation Manual model
Armchair with tilting frame (inclination up to 30 °)
4 wheels: Ø 125 mm with brakes
Padded backrest, seat and armrests
Removable outer cover: velvet, synthetic leather, polyester fabric
Foot support
Maximum Capacity: 130 Kg

Brown

Cocoa

Raspberry

Grey

## Electric Mode

Code	Color	Material
01.11.Z4-VME	Brown	Polyester Velvet
01.11.Z4-PCE	Cocoa	Leather
01.11.Z4-PLE	Raspberry	Leather
01.11.Z4-TGE	Grey	Polyester Fabric

## Manual Mode

Code	Color	Material
01.11.Z4-VMM	Brown	Polyester Velvet
01.11.Z4-PCM	Cocoa	Leather
01.11.Z4-PLM	Raspberry	Leather
01.11.Z4-TGM	Grey	Polyester Fabric

## TECHNICAL FEATURES

Dimensions	
Seat height from the ground	48 cm
Seat width	30 cm, 36 cm, 42 cm, 48 cm, 54 cm
Seat depth	48 cm
Total width (armrests + seat)	Seat + 25 cm
Weight	38 Kg

## DESCRIPTION

### THE ABDOMINAL BELT

The chair is equipped with an adjustable and removable lap belt.

Attach the restraint so that the patient is seated in the back of the seat with the lumbar region against the back of the chair. Adjust the tension to prevent the user from sinking and injuring himself.

Being removable, it is important every time you use it to check that it is properly anchored to its attachments.

**IMPORTANT!** Healthcare personnel or caregivers must constantly supervise the patient.

### HANDLE

Located behind the backrest, it allows a third person to push and guide the chair (photo 1).

## SEAT

Reclining seat with spring.

Adjustment of the inclination: use the handle located on the handle on the back of the seat.  
IMPORTANT!! this operation must be carried out exclusively by the accompanying person.

## REMOTE CONTROL

### TYPE 1

Photo 2

- The first two buttons allow you to use the electric AUTONOMY function (front / rear), allowing you to independently manage the inclination.

WHITE button: ascent to an upright position

GREEN button: down

- The BLUE button activates the RELAX system programmed to automatically change the inclination of your chair, thus relieving tension and the risk of prolonged compressions.

- The YELLOW button activates the MASSAGE function from the vibrations of the lumbar cushion.

### TYPE 2

Photo 3

The first two buttons allow you to benefit from the electric AUTONOMY function (front / rear), allowing you to manage the inclination independently.

WHITE button: up

GREEN button: descent

## REMOTE CONTROL LOCK

Press the two buttons (White and Green) for 15 seconds - The remote control is locked!

Press the two buttons (White and Green) again for 5 seconds - The remote control is unlocked!

When connecting the chair to the mains, DO NOT PRESS the remote control buttons for 60 seconds. If, however, a power failure or other places the remote control in "safety" mode (locked), please reset the remote control by performing a complete cycle of tilting the chair: from the front raised to " to fully recline the rear or unplug the wheelchair for 60 seconds. NOW the remote control is reset.

## COVERING

The covering of your chair is removable and can be machine washed at 30 °.

When washing, it is recommended to store the covering in a laundry net to keep its original appearance.

## WHEELS

- Always check that there is no debris on the wheels before use
- Check that the wheels are not worn before use. Replace them if necessary.

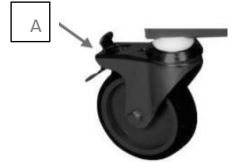
Photo 4

### TYPE 1 - PHOTO 4

#### Brake

- Activate the brake by pressing down with your foot (A)
  - To release the brake, lift upwards with your foot (A)
- Check that the brake does not touch the wheels when moving.

### TYPE 2 - PHOTO 5

#### Brake

- Activate the brake by pressing the paddle (A) with your feet
- To release the brake, press the paddle (B) with your foot. Check that the brake does not touch the wheels during movement.

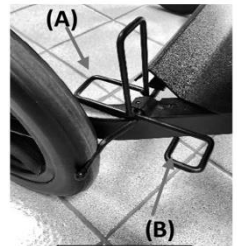
Photo 5

## LEG REST

The leg rest is adjustable in inclination.

Manual Model:

- The inclination of the adjustable leg rest is linked to the inclination of the seat and is done with the handle placed on the rear bar.

Electric Model:

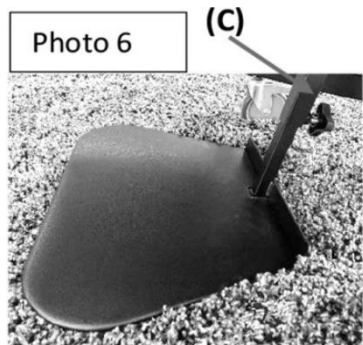
- The adjustment of the inclination of the leg rest is linked to the inclination of the seat and is carried out with the remote control.

**IMPORTANT!! THIS OPERATION MUST BE CARRIED OUT BY THE HEALTHCARE STAFF OR BY THE CAREGIVER.**

## FOOTREST (photo 6)

The footrest is removable and adjustable in height:

- Loosen the knob (C) to unlock the footrest height adjustment.
- Always check that there is nothing under the footrest before returning the chair to the upright position to avoid crushing
- The footrest must not be removed.

## TSMALL TABLE

### TYPE 1- PHOTO 7

- Insert the stage into the tube provided for this purpose on the frame.
- Screw the knob.

Photo 7

### TYPE 2 - PHOTO 8

- Insert the shelf in the openings provided for this purpose on the rear column.

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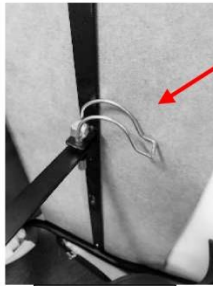
### WARNINGS FOR USE

- Do not exceed the maximum permitted load (130 kg).
- Always assist the user during handling operations.
- Connect the electric cable directly to the socket. Do not use an extension cord.
- Before sitting on the lift chair, make sure that the remote control and its wire are not in the way.
- Move your hands and feet away from the attachment points such as the chair and footboard lifting mechanisms.
- When not in use, leave the lift chair in an upright position with the footboard closed.
- If the device does not work or malfunctions, immediately stop use, switch off the device, unplug it and contact the authorized dealer or the Service Center for repair.
- When the device is not used, make sure that the remote control is correctly placed in the appropriate pocket.
- When removing the plug from the socket, never pull on the cable, but take it directly to the plug near the socket.
- Do not use when wearing wet clothing.
- Do not use when the lift chair when is wet.
- Do not use with voltage other than that indicated on the label, also do not enter the plug into multiple sockets.
- Do not use in an environment with a high percentage of humidity such as in bathrooms or outdoors.
- Do not use in case of damage to the electric cable, plug and socket.
- Do not damage, break, modify, bend, pull, twist, wrap the electric cable during use.
- Do not place the power supply cable near heat sources or hot surfaces.

### MAINTENANCE

Checks periodically:

That all screws and bolts are tight.

The mechanical parts (hinges and pins) must be checked monthly and lubricated with a drop of oil in order to promote smoothness and wear resistance.

## CLEANING

Backrest quilt set :

- Remove the velcro flap of the cover located on the back of the backrest.
- Pull down the right and left zip of the duvet

Seat / leg rest quilt :

Take off the velcro flap at the bottom, to the right and the left or the base

All modules can be machine-washed at 30° C

We advise you to put the modules in a washing net or in a pillowcase, to preserve their original aspect.

## WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, because these substances cause the abrasion of the components of the lift chair.

## GENERAL DISPOSAL CONDITIONS

In case of disposal of the device, never use the normal municipal solid waste disposal systems. It is recommended to dispose of the device through the common municipal ecological islands for the envisaged recycling operations of the materials used.

## TREATMENT OF EXHAUSTED BATTERIES (DIRECTIVE 2006/66 / EC)

### BATTERY REGISTRATION NUMBER IT21040P00006953

The device contains a non-removable battery and must not be disposed of as urban waste, as it is subject to separate collection to avoid damage to the environment. In compliance with the directives 2002/96, 2006/66 and the national transposition laws for the disposal of the product at the end of its life.

## WARNINGS FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2012/19 / EU.

### AEE REGISTRATION NUMBER IT21050000012993

At the end of its useful life, the product must not be disposed of with municipal waste. It can be delivered to the appropriate separate collection centers set up by municipal administrations, or from retailers that provide this service. Dispose of separately, the product avoids possible negative consequences for the environment and for health deriving from its inappropriate disposal. Allows also materials to be recovered in order to obtain an important saving of energy and resources. For point out the obligation to dispose of electromedical equipment separately on the product the mark of the crossed out mobile rubbish bin is shown.

## **WARRANTY**

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.