MANUALE D'USO POLTRONA NOSTRESS

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USER MANUAL
NOSTRESS LIFT CHAIRS

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

Devicare lift chairs are indicated to facilitate daily difficulties in elderly and disabled people. The main help function is the lift-mode, which gives the possibility to reach the upright position in safety. The frame allows the backrest to be tilted and the footrests to be raised.

WARNINGS

- For correct use of the device, refer carefully to the following manual.
- For correct use of the device always consult your doctor or therapist.
- Always pay close attention to the presence of moving parts that could cause limb entrapment and personal injury.
- Keep the packed product away from any source of heat as the packaging is made of cardboard.
- Always pay close attention to the presence of children.
- Immediately disconnect the electric cable from the mains if operating irregularities are found and check for any electrical leaks.
- Do not unplug or insert the socket with wet hands.
- The lift chair must be mounted on a horizontal surface. Avoid loose bumpy plans.

DESCRIPTION

Features
Electric/manual model with 2 button remote control
Relax model armchair
Backrest, seat padded with silicone hollow fiber, removable covers
Retractable armrests
Painted steel structure with 2 rear wheels
Possibility of having 3 positions: sitting, relaxation, rest
Maximum load: 130 Kg

Marrone
Brown

Antracite
Anthracite

Lampone
Raspberry

Grigio
Grey

Electric Mode

Code	Color	Material
01.11.Z1-VME	Brown	Polyester Velvet
01.11.Z1-PAE	Anthracite	Leather
01.11.Z1-PLE	Raspberry	Leather
01.11.Z1-TGE	Grey	Polyester Fabric

Manual Mode

Code	Color	Material
01.11.Z1-VMM	Brown	Polyester Velvet
01.11.Z1-PAM	Anthracite	Leather
01.11.Z1-PLM	Raspberry	Leather
01.11.Z1-TGM	Grey	Polyester Fabric

TECHNICAL FEATURES

Dimensions	
Seat height from the ground	48 cm
Seat width	53 cm
Seat depth	55 cm
Total width (armrests + seat)	72 cm
Total height	112 cm

USE

Sit down in your NOSTRESS

To recline it in TV position


- Put your hands in the middle of the two armrests (elbows bent) tighten the armrest with your hands.

- Push with your hands (tight on the armrests) forward and the same time push the backrest with your back.

To recline it in the rest position

- Raise the legs in order to release the bascule of the backrest.
- Push on the backrest with your back.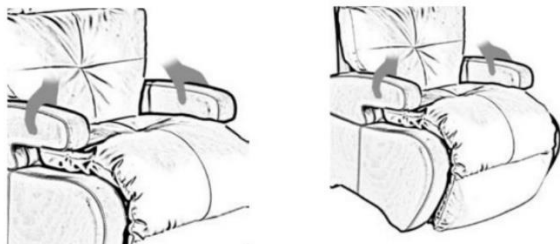


To go back in the sit down position


- You need to put the rest position first
- Then place your hands on the two armrest (arms outstretched) tighten the armrest with your hands.
- Pull with your hands (as far as possible) on the armrest and at the same time press the leg rest with your calves.

Lock the back :

In order to fix the back to the chair, please to turn the 2 little black ring to the ends (picture 1 and zoom picture 2). You will find the black ring turned on the left side in the basic position (picture 3). Please to turn back the black rings completely on the bottom (picture 4) in locked position, they will grasp to the mechanism's end stop and then fix the back.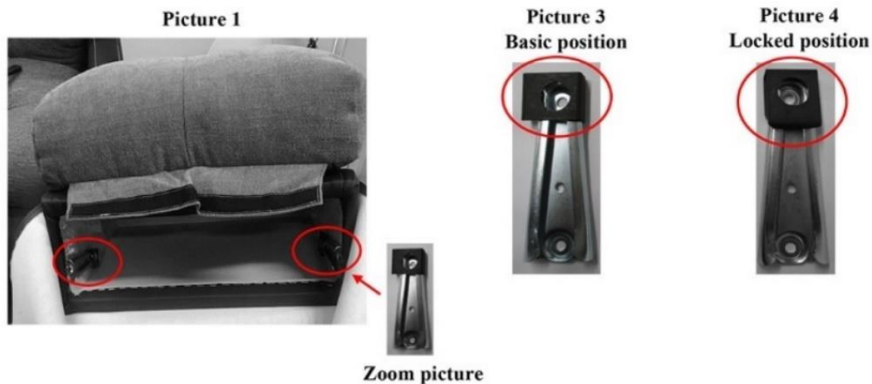

Remote control's using of the ELECTRIC NOSTRESS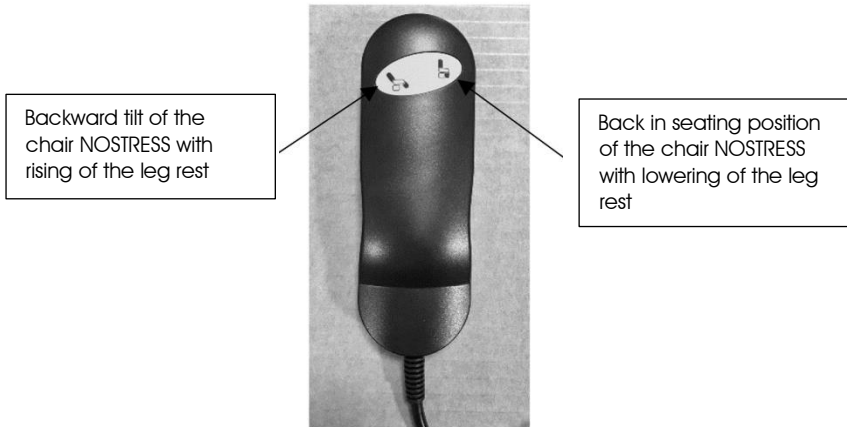


NOTES: When operating the controls, you must be able to view the surrounding area in order to prevent people, animals or things from remaining under the moving parts.

NOTES: Do not lean or apply force downwards when the backrest is tilted. Risk of tipping over and injury.

NOTES: The lift chair is not designed to move the user.
Move the lift chairs only when nobody is on it.

Equipment og tje ARMCHAIR NOSTRESS

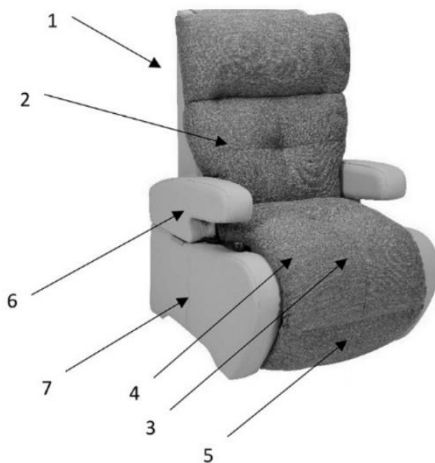- 1 Backrest
- 2 Backrest quilt removable / washable
- 3 Seat
- 4 Quilt base / leg rest removable / washable
- 5 Leg rest
- 6 Two armrest retractable / removable
- 7 Two armrest board

WARNINGS FOR USE

- Do not exceed the maximum permitted load (130 kg).
- Always assist the user during handling operations.
- Connect the electric cable directly to the socket. Do not use an extension cord.
- Before sitting on the lift chair, make sure that the remote control and its wire are not in the way.
- Move your hands and feet away from the attachment points such as the chair and footboard lifting mechanisms.
- When not in use, leave the lift chair in an upright position with the footboard closed.
- If the device does not work or malfunctions, immediately stop use, switch off the device, unplug it and contact the authorized dealer or the Service Center for repair.
- When the device is not used, make sure that the remote control is correctly placed in the appropriate pocket.
- When removing the plug from the socket, never pull on the cable, but take it directly to the plug near the socket.
- Do not use when wearing wet clothing.
- Do not use when the lift chair when is wet.
- Do not use with voltage other than that indicated on the label, also do not enter the plug into multiple sockets.
- Do not use in an environment with a high percentage of humidity such as in bathrooms or outdoors.
- Do not use in case of damage to the electric cable, plug and socket.
- Do not damage, break, modify, bend, pull, twist, wrap the electric cable during use.
- Do not place the power supply cable near heat sources or hot surfaces.

MAINTENANCE

Checks periodically:

That all screws and bolts are tight.

The mechanical parts (hinges and pins) must be checked monthly and lubricated with a drop of oil in order to promote smoothness and wear resistance.

CLEANING

Backrest quilt set :

- Remove the velcro flap of the cover located on the back of the backrest.
- Pull down the right and left zip of the duvet

Seat / leg rest quilt :

Take off the velcro flap at the bottom, to the right and the left or the base

All modules can be machine-washed at 30° C

We advise you to put the modules in a washing net or in a pillowcase, to preserve their original aspect.

WARNINGI

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, because these substances cause the abrasion of the components of the lift chair.

GENERAL DISPOSAL CONDITIONS

In case of disposal of the device, never use the normal municipal solid waste disposal systems. It is recommended to dispose of the device through the common municipal ecological islands for the envisaged recycling operations of the materials used.

TREATMENT OF EXHAUSTED BATTERIES (DIRECTIVE 2006/66 / EC)

BATTERY REGISTRATION NUMBER IT21040P00006953

The device contains a non-removable battery and must not be disposed of as urban waste, as it is subject to separate collection to avoid damage to the environment. In compliance with the directives 2002/96, 2006/66 and the national transposition laws for the disposal of the product at the end of its life.

WARNINGS FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EUROPEAN DIRECTIVE 2012/19 / EU.

AEE REGISTRATION NUMBER IT21050000012993

At the end of its useful life, the product must not be disposed of with municipal waste. It can be delivered to the appropriate separate collection centers set up by municipal administrations, or from retailers that provide this service. Dispose of separately, the product avoids possible negative consequences for the environment and for health deriving from its inappropriate disposal. Allows also materials to be recovered in order to obtain an important saving of energy and resources. For point out the obligation to dispose of electromedical equipment separately on the product the mark of the crossed out mobile rubbish bin is shown.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.