#674-675-1674-1675-676-677-679-800-810

PAVIS S.p.A.

Sede / Headquarter:

Via G. Rossini, 11/B - 21020 Buguggiate (VA) - Italy
Tel. (+39) 0332 45 54 63
Fax (+39) 0332 45 97 48

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(UK)
1 - INDICATIONS
• Abdominal content containment

2 - CONTRA-INDICATIONS AND SIDE-EFFECTS
The choice between different models must be done according to the pathology and the necessities stated by the doctor. To avoid weaken abdominal muscles, it is recommended not to use the binder more than time needed to solve the pathology. Do not use the device if it create pressure on wounds and swellings. It is preferable to wear it over a cotton t-shirt. Since this device is NON-INVASIVE and made with non-allergenic fabrics, no negative or side effects have ever been noted. Any serious incident occurred in relation to the device use, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

3 - MAINTENANCE AND CARE
• Before washing, close the hook and loop parts

This device is non-biodegradable. Even though it is made of natural fabrics, after use, avoid release the product to the environment. AT THE END OF ITS USABLE LIFE HAS TO BE DISCHARGED ALONG WITH SOLID ORDINARY WASTE.

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