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USER MANUAL X" TRUNK SUPPORT WITHOUT FRONT ZIP

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and manufactured to meet all your needs for practical, correct and safe use. This manual contains small tips for the correct use of the device of your choice and valuable advice for your safety. We recommend that you read this manual carefully before using the item you have purchased. If you have any doubts, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and the following manual without prior notice in order to improve its features.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO S.r.l.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the treating physician or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO S.r.l. declines any responsibility for damage to property or people resulting from improper use, not agreed with a professional figure, which does not correspond to what is indicated in the user manual.
- Any accidents, anomalies, breakages, must be reported to DEMARTA VIRGINIO S.r.l. in order to monitor the quality and criticalities of the product.
- Keep the packaged product away from any heat source as the packaging is made of cardboard.
- Always pay attention to the presence of children.
- The product has been designed for its intended use only. Do not abuse it in any way.

CE DECLARATION OF CONFORMITY

DEMARTA VIRGINIO S.r.l. declares under its sole responsibility that the product manufactured and placed on the market by DEMARTA VIRGINIO S.r.l. complies with the applicable provisions of EU Directive 2017/745 on medical devices.

To this end, DEMARTA VIRGINIO S.r.l. guarantees and declares under its sole responsibility the following:

- The devices meet the general safety and performance requirements as required by Annex I to Regulation 2017/745 as prescribed by Annex IV to that Regulation.
- The devices are NOT MEASURING INSTRUMENTS.
- The devices are NOT INTENDED FOR CLINICAL INVESTIGATION.
- The devices are marketed in NON-STERILE packaging.
- The devices are to be considered as belonging to class I in accordance with the provisions of Annex VIII of the aforementioned regulation.

INTENDED USE

The ADIGGY Front Zip-Free "X" Thoracic Fixation in Padded and Elastic Neoprene with plastic clip closure facilitates the correct positioning of the patient's spine and provides dynamic support in case of spastic movements. It is recommended to use it with the ADX20 or ADX21 pelvic/abdominal belt.

WARNINGS

- Retainers should be used by qualified healthcare personnel.
If used at home, you should seek advice from your doctor or therapist.
- Patients with restraints require frequent monitoring.

DESCRIPTION

- Padded, elastic and resistant neoprene fabric.
- Ergonomic design adapted to any type of morphology.
- Quick assembly with front opening with clips.
- Direct attachment to the wheelchair frame.


ASSEMBLY

WATCH ME!

CLOSURE SYSTEM WITH PLASTIC CLIPS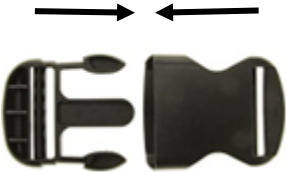

To open the plastic clip closure system, press inwards on each side to release the closure. To attach, insert the two parts together and apply light pressure.

TECHNICAL FEATURES

Code	Size	Dimensions (cm)
01.12.ADX24-S	S	23-33
01.12.ADX24-M	M	33-43
01.12.ADX24-L	L	43-53
01.12.ADX24-XL	XL	53-63

Materials	
Belt	Neoprene
Clips Closure	Metal and Plastic
Maintenance and Cleaning	
	Water temperature not above 30° C
	Do not iron
	Do not bleach
	Do not dry in dryer

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems. It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.
DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.