**KYARA®**

MA\_LBK100\_LBK110\_LBK120\_LBK130\_LBK140\_LBK200\_LBK210\_LBK220\_LBK225\_LBK230\_02\_A\_ITA\_05 2021

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---

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## INDICE

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

### 1. CODES

LBK100-M T-SHIRT - SIZE M  
 LBK100-L T-SHIRT - SIZE L  
 LBK100-XL T-SHIRT - SIZE XL  
 LBK110-M LEGGINGS - SIZE M  
 LBK110-L LEGGINGS - SIZE L  
 LBK110-XL LEGGINGS - SIZE XL  
 LBK120 KNEE PAD  
 LBK130 ELBOW PAD  
 LBK140 SCARF  
 LBK200 PILLOWCASE  
 LBK210-1 MATTRESS COVER - 80X190 CM  
 LBK210-2 MATTRESS COVER - 160X190 CM  
 LBK210-3 MATTRESS COVER - 170X190 CM  
 LBK220-42 WHEELCHAIR SEAT COVER - 42 CM  
 LBK220-46 WHEELCHAIR SEAT COVER - 46 CM  
 LBK220-50 WHEELCHAIR SEAT COVER - 50 CM  
 LBK220-58 WHEELCHAIR SEAT COVER - 58 CM  
 LBK225 UNIVERSAL SEAT COVER - 60X45 CM  
 LBK230-1 SINGLE MATTRESS  
 LBK230-2 DOUBLE MATTRESS

### 2. INTRODUCTION

Thank you for choosing a FITergy product in the Kyara range by Moretti. Before using the product, please read the user instructions carefully, particularly the safety precautions, and comply with them. Keep this manual for consultation for the entire lifetime of the product. If you give the product to someone else, make sure to give them all the documentation as well. This manual contains useful suggestions for using your device properly and safely. Should you have any queries, please contact your dealer for suitable advice and assistance.

**NB Check to ensure that no parts of the product have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.**

### 3. INTENDED USE

This product is intended for assistance in pain relief.

#### **WARNING!**

- Do not use the product for any purpose other than that specified in this manual.
- MORETTI S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual.
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features.

#### 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of FITERGY PRODUCTS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

#### 5. GENERAL WARNINGS

- After removing the product from its packaging, check that it is intact and has not been damaged during shipment. If you have any doubts, do not use the product and contact your local retailer. The packaging material is not a child's toy! Keep the plastic bag out of the reach of small children; suffocation risk
- Consult your doctor or therapist to find out if the product is suitable for you.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

**WARNING!**

The product does not cure bacterial infections, oncological diseases, fractures or metabolic or viral diseases.

**WARNING!**

No contra-indication emerged during the trial phase, but people with pacemakers and pregnant women should only use the product after seeking medical advice.

## 6. SYMBOLS

	Product code
	Month and year manufacture
	Unique device identification
	CE marking
	Instruction manual
	Medical Device
	Manufacturer

### 6.1 Washing symbols

	Hand wash
	Do not bleach
	Do not tumble dry
	Do not iron
	Do not dry clean
	Dry clean on gentle cycle

## 7. GENERAL DESCRIPTION

All products in the FITergy range are unique of their kind and the result of in-depth scientific research. Each FITergy product has a biophysical and biochemical pain relief function. It is sufficient to use it while resting to obtain rapid relief from the intense pain caused by:

- Sciatica
- Lower back pain
- Gonarthrosis
- Prolonged stress
- Neck pain
- Osteoarthritis
- Headaches
- Sleep disorders
- Menstrual pain
- Tendinitis

The indicative pain relief values of all FITergy articles are shown in the table below:

FITERGY PRODUCT	USE TIME	BENEFITS
T-SHIRT LEGGINGS KNEE PAD ELBOW PAD SCARF	5 DAYS	RELIEF FROM ALL TYPES OF PAIN
	15 DAYS	REDUCTION OF STRESS
PILLOWCASE MATTRESS COVER MATTRESS UNIVERSAL SEAT COVER WHEELCHAIR SEAT COVER	7 DAYS	PAIN RELIEF
	14 DAYS	RELIEF FROM INTENSE PAIN
	21 DAYS	70% REDUCTION OF THE MOST ACUTE PAIN REDUCTION OF STRESS PROCESSES IMPROVEMENT IN GENERAL CONDITIONS IMPROVED SLEEP/WAKE CYCLE

### WARNING!

The time periods shown for pain relief are indicative only. The benefits of the products vary from user to user, according to the amount of time it is used during the day and to the type and intensity of the pain.

## 8. HOW TO USE

### 8.1 LBK100-x, LBK110-x, LBK120, LBK130, LBK140

Wear the garment directly in contact with the skin while resting.

### 8.2 LBK200, LBK210-x, LBK220-xx, LBK225, LBK230-x

Use normally as for standard, everyday products like a pillowcase, mattress, mattress cover and seat cover.

**9. CLEANING AND MAINTENANCE**
**9.1 LBK100-x, LBK110-x, LBK120, LBK130, LBK140**

Hand wash inside out at a maximum 30°C

Do not bleach

Do not tumble dry

Do not iron

Do not dry clean

**9.2 LBK200, LBK210-x and cover LBK230-x**

Dry clean on gentle cycle

Do not bleach

Do not tumble dry

Do not iron

Dry flat

**9.3 LBK220-xx, LBK225**

Hand wash in cold water with detergent for delicates

Do not bleach

Do not iron

Do not tumble dry

Dry flat

**WARNING!**

LBK210-x, LBK220-xx and LBK225 products contain a very high amount of pure high quality merino virgin wool. Wool is a living fiber, therefore unstable, and may be subjected to changes in length and / or width of  $\pm 5\%$ : the frequency of the ordinary maintenance interventions will determine the conservation of the state of maximum hygiene and dimensions. It is recommended to restore the positioning of the mattress cover and seat cover every day to bring it back to its original size. It is advisable to expose them to the air in a moderately humid and shaded place. This way, the special wool is oxygenated and regenerated by regaining the original softness and its characteristic smell. This wool peculiarity is its self-cleaning capacity: every single fiber manages to free itself from both dust and foreign impurities, it is therefore recommended to shake it slightly before storing it to facilitate the natural process of cleaning and elimination of any unwanted hair.

## 10. CONDITIONS OF DISPOSAL

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

## 11. TECHNICAL SPECIFICATIONS

FITERGY PRODUCT	DIMENSIONS	MATERIALS
LBK100-X	SIZES M, L, XL	70% POLY BIOMOLECULES 22% POLYAMIDE 8% ELASTAM
LBK110-X	SIZES M, L, XL	
LBK120 LBK130 LBK140	ONE SIZE	
LBK200	77X47 CM	EXTERIOR: 100% POLYESTER PADDING: 50% POLY BIOMOLECULES, 50% POLYESTER
LBK210-1 LBK210-2 LBK210-3	80X190 CM 160X190 CM 170X190 CM	EXTERIOR: 100% POLYESTER PADDING: 50% POLY BIOMOLECULES, 50% PURE COTTON PLUS POLYESTER WADDING 100%
LBK220-42 LBK220-46 LBK220-50 LBK220-58 LBK225	42 CM 46 CM 50 CM 58 CM 60X45 CM	EXTERIOR: 100% POLYESTER INTERIOR: 50% POLY BIOMOLECULES, 50% POLYESTER PADDING: 100% ULTRA-FLEXIBLE POLYURETHANE FOAM
LBK230-1 LBK230-2	80X190X15 CM 160X190X15 CM	EXTERIOR: 100% POLYESTER PADDING: 50% POLY BIOMOLECULES, 50% PURE COTTON PLUS POLYESTER WADDING 100%

## **12. WARRANTY**

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject to the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual.

Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual.

Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual.

Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

### **12.1 Non-Liability clause**

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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