DEMARTA
VIRGINIO

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USER MANUAL
BATH STOOL
02.02.795

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

The BATH STOOL has an aluminum structure adjustable in height in eight positions, seat in resistant and easily washable material.

TECHNICAL DATA


Code	Capacity (Kg)
02.02.795	100

DESCRIPTION

This item is a height adjustable, lightweight shower seat with a rotating seat top, designed to provide extra comfort in the shower, for occasional use.

INSTALLATION


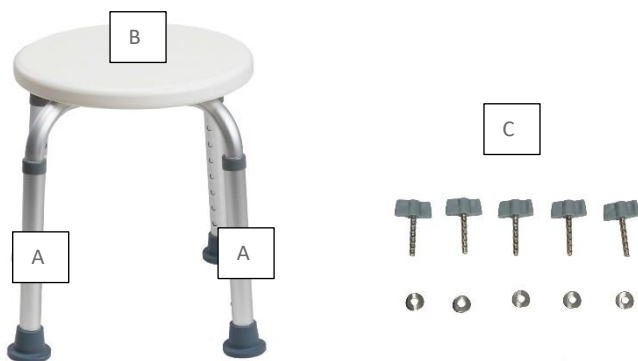
Take all parts out of packaging and check before assembly.


A-LEGS X 2 PCS

B-SEAT X 1 PCS

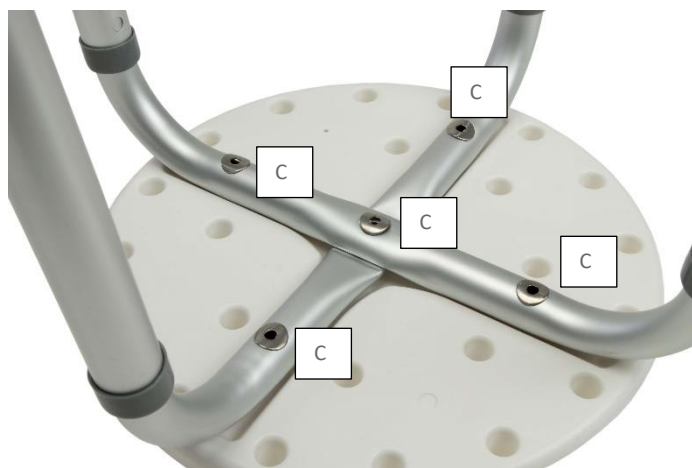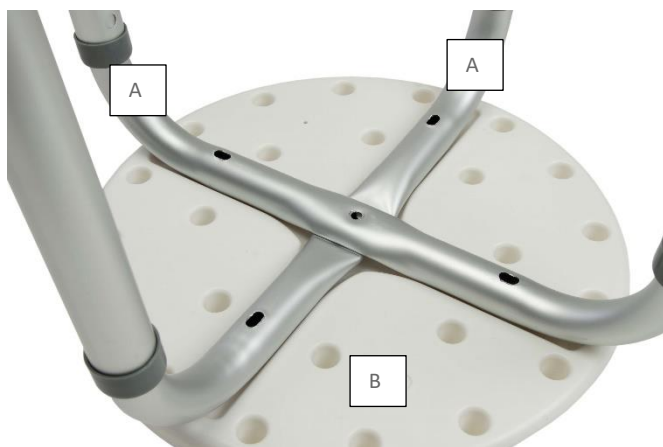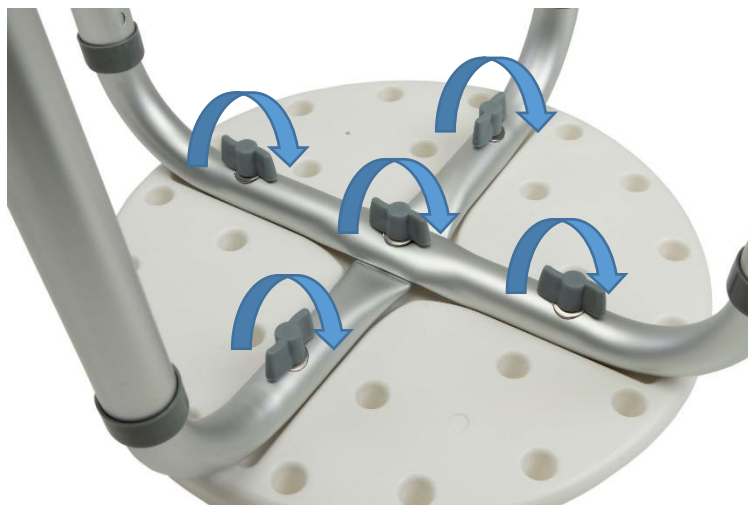C-TYPE 1 SCREWS AND NUTS

HEIGHT ADJUSTMENT

1. Grip leg shaft with one hand whilst pushing the metal locking pin down with the other hand so the locking pin retracts fully into the shaft.
2. Select required height by adjusting the leg in an upwards or downwards motion and release the metal locking pin in front of the required hole position (Fig. 6).
3. Prior to use always ensure that each locking pin fully protrudes from the selected hole position (Fig. 6).

PRECAUTIONS

- Always help the patient with difficulties to position on the seat.
- Do not exceed the maximum capacity.
- Caution in case of wet or slippery floor.
- Please keep the device in a dry place. Do not store the product outside.

CLEANING

Washing can take place with warm water and neutral soap
Disinfection can take place with simple products of normal domestic use

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.