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ENGLISH

ITALIANO

STANDARD	With Loops	With Clips																		
	 <p>TCS24.[S,M,L,X,C]</p> <ul style="list-style-type: none"> • Universal use • 4 or 2-point suspension with loop support straps • Head support 	 <p>TCS23.[S,M,L,X,C]</p> <ul style="list-style-type: none"> • Universal use • Use with balance spreader bar to facilitate lifting • Head support 																		
HYGIENE	 <p>TCH24.[S,M,L,X,C]</p> <ul style="list-style-type: none"> • Hygienic, ideal for toileting • 4 or 2-point suspension with loop support straps • Head support 	 <p>TCH23.[S,M,L,X,C]</p> <ul style="list-style-type: none"> • Hygienic, ideal for toileting • Use with balance spreader bar to facilitate lifting • Head support 																		
	CONTINUOUS	 <p>TCC04.[S,M,L,X,C]</p> <ul style="list-style-type: none"> • Ideal for continuous use to be worn all the time when seated • 4 or 2 point-suspension • Head support <p>CODES OF SIZES AVAILABLE</p> <table border="1" data-bbox="868 1585 1426 1769"> <thead> <tr> <th>Code</th> <th>Size</th> <th>Description</th> </tr> </thead> <tbody> <tr> <td>E</td> <td>E</td> <td>SMALL</td> </tr> <tr> <td>M</td> <td>M</td> <td>MEDIUM</td> </tr> <tr> <td>L</td> <td>L</td> <td>LARGE</td> </tr> <tr> <td>X</td> <td>XL</td> <td>EXTRA LARGE</td> </tr> <tr> <td>C</td> <td>XXL</td> <td>2EXTRA LARGE</td> </tr> </tbody> </table>		Code	Size	Description	E	E	SMALL	M	M	MEDIUM	L	L	LARGE	X	XL	EXTRA LARGE	C	XXL
Code	Size	Description																		
E	E	SMALL																		
M	M	MEDIUM																		
L	L	LARGE																		
X	XL	EXTRA LARGE																		
C	XXL	2EXTRA LARGE																		

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Summary

1.	GENERAL INFORMATION	5
1.1	MANUFACTURER	5
2.	PRODUCT DESCRIPTION AND INFORMATION	5
2.1	DESCRIPTION	5
2.2	INTENDED USE	5
2.3	USERS	5
2.4	ENVIRONMENT OF USE	5
2.5	STORING	5
2.6	MANUFACTURER'S DECLARATION	5
3.	GENERAL WARNINGS	5
3.1	SYMBOLS ON THE LABEL	6
3.2	SYMBOLS ON THE MANUAL	6
3.3	SYMBOLS ON THE PACKAGING	6
4.	PACKAGE CONTENT AND PRODUCT FEATURES	7
4.1	PACKAGE CONTENT:.....	7
4.2	PRODUCT DESCRIPTION	7
4.2.1	Standard loop model TCS24	7
4.2.2	Hygiene loop model TCH24	7
4.2.3	Standard clip model TCS23	7
4.2.4	Hygiene clip model TCH23	8
4.2.5	Continuous loop model TCC04	8
5.	READ LABEL	9
	USE	10
6.1	4.1 WARNINGS BEFORE USE.....	10
6.2	DEVICE USE	10
6.2.1	Standard loop harnesses	10
6.2.2	Standard clip harnesses	13
6.2.3	Hygiene loop or clip harnesses	13
6.2.4	Lifting from lying down variant.....	15
6.2.5	Harness removal.....	17
6.2.6	Continuous harnesses	18
7.	MAINTENANCE AND USEFUL LIFE	18
7.1	INSPECTION	18
7.1.1	Inspection after each use	18
7.1.2	Annual inspection	18
7.2	USEFUL LIFE	18
	CLEANING	18
8.1	WASHING.....	18
	DISPOSAL	18
10.	TECHNICAL SPECIFICATIONS	18
11.	WARRANTY	18
11.1	GENERAL CONDITIONS	18
12.	INSPECTION RECORDS	20

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
1. GENERAL INFORMATION

This manual contains information for practical, correct and safe use of the device and is intended for reading by specialised personnel and the user of the product. It is recommended that you read the entire manual carefully before using the product.

If you have any doubts or need clarification, please contact your dealer who will be able to advise you correctly.

The importance of reading and understanding the user manual is highlighted on the product by the following symbol: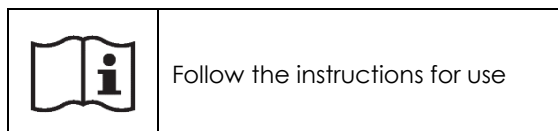

1.1 MANUFACTURER

CHINESPORT S.P.A, Via Croazia, 2-33100 Udine Italy

 +39 0432.621.621

 <https://www.chinesport.it>

 chinesport@chinesport.it

The company produces in accordance with the quality standard UNI EN ISO 13485:2016

2. PRODUCT DESCRIPTION AND INFORMATION

2.1 DESCRIPTION

Standard harnesses in mesh with loops or clips, hygiene harnesses in mesh with loops or clips, continuous harness with loops.

2.2 INTENDED USE

The harnessing systems, for lifters, can be used both at home and in specialised structures, and are suitable to lift and move people from and to: beds, wheelchairs, armchairs, baths, toilets, floor, swimming pools, treatment tables, etc. They are not suitable to lift loads or to transport materials in general.

Use of the device is forbidden for purposes other than the intended use in this manual. The manufacturer cannot be held in any way liable for damage to people or property due to misuse of the device or however use different to the intended use in this manual. The manufacturer reserves the right to make changes to the product and the manual without prior notification for the purpose of improving its features and performance.

2.3 USERS

Trained medical and technical staff, family members or carers responsible for assisting the person to lift and familiar with this manual.

It is up to the specialist to judge the physical fitness of the patient for whom the product is intended to be used.

Use under supervision of the operator or person providing assistance is always recommended.

Do not leave the person in the lifted position for long periods of time and unsupervised.

2.4 ENVIRONMENT OF USE

Clinical/hospital or home environment. Use the lifter on flat, dry, stable and obstacle-free surfaces. Refer to the manual of the lifter used for the environment and limitations of use.

2.5 STORING

Keep in a dry place, at environmental temperature. After washing, ensure the harness is perfectly dry before storage.

2.6 MANUFACTURER'S DECLARATION

The manufacturer declares:

- the device is not a measurement tool
- the device is not intended for clinical tests
- the device is not sterile and is not disposable
- to ensure proper use and safety for the user, the routine maintenance operations described in the relevant paragraph must be followed
- the device cannot be used differently to the intended use declared in this manual

3. GENERAL WARNINGS

Always refer to this manual for proper use of the device.

- Store the device in an environment that complies with the labels on the packaging and the specifications in this manual
- The **useful life** of the product is **5 years** provided routine operations are carried out correctly, as outlined in this manual. It is strictly forbidden to use the device beyond its declared useful life. At the end of its useful life, proceed according to the methods illustrated in the relevant paragraph
- The manufacturer cannot be held liable, to the fullest extent permitted by applicable law, for any direct or indirect, special, incidental or consequential damages caused by:
 - o Wrong use of the device
 - o Improper use of the device and outside of its intended use
 - o Using the device beyond its declared useful life
 - o Using the device in environments not covered by this manual
 - o Use with unsuitable patients
 - o Operator distraction or wrong application of the harness on the lifter
 - o Use without checking the status of the device as described in the relevant paragraph
 - o Incorrect maintenance or lack of maintenance
 - o Use with parts or accessories that are not compatible or not approved by the manufacturer
 - o Incorrect disposal or disposal is other than as described in this manual

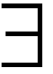
- o Faults or problems due to the lifter, in this case refer to the lifter manufacturer

The harness provides the maximum working load permitted on the label, i.e.:

250kg

Pay attention to the maximum working load declared for the lifter on which the harness is used as this could be less.

3.1 SYMBOLS ON THE LABEL

	Follow the instructions for use
	CE certified, the device complies with the Medical Devices Directive 93/2/EEC and as amended 2007/47/EC
	Maximum washing temperature of coloured garments. Do not exceed a temperature of 60°C / 140°F
	Do not bleach or use bleach-free washing products
	Do not dry in dryer
	Do not iron
	Do not dry clean. No professional dry cleaning. Do not use solvent-based stain removers
	Manufacturer
	Refer to the model and size indicated using the pinhole in the adjacent box
	Production batch composed by indicating the year and month of production using the pinhole in the adjacent box

3.2 SYMBOLS ON THE MANUAL

	Warning! Pay particular attention to safety instructions near this symbol to avoid hazardous situations.
---	--

3.3 SYMBOLS ON THE PACKAGING

	Fragile
	Recyclable
	Keep dry
	Keep this side up
	Do not use sharp blades to open

4. PACKAGE CONTENT AND PRODUCT FEATURES

The product is delivered in suitable packaging to ensure it is received intact. To open the package and remove its content, pay attention to warnings and symbols on the package.

Appropriately dispose of packaging and resulting material and follow the information on the package and in this manual.


4.1 PACKAGE CONTENT:

The package contains:

- Instruction manual
- Harness


4.2 PRODUCT DESCRIPTION

4.2.1 Standard loop model TCS24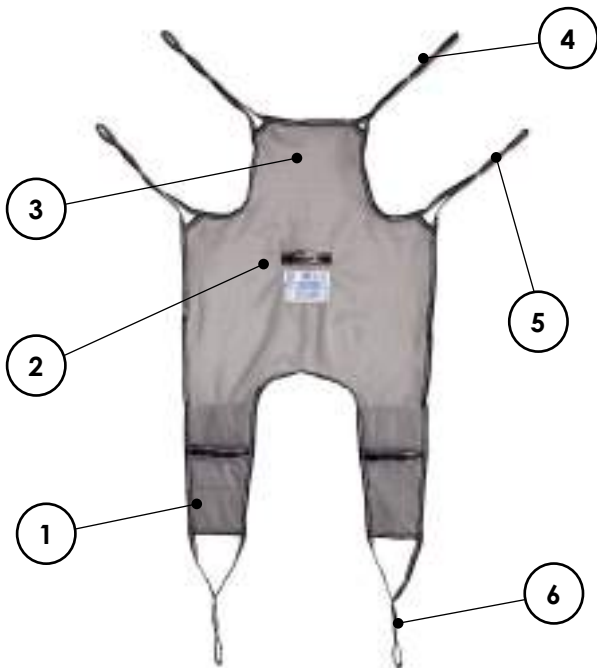

- | | |
|-----------------|-----------------------|
| 1 Thigh section | 4 Head support strap |
| 2 Chest section | 5 Chest support strap |
| 3 Head support | 6 Legs support strap |

4.2.2 Hygiene loop model TCH24

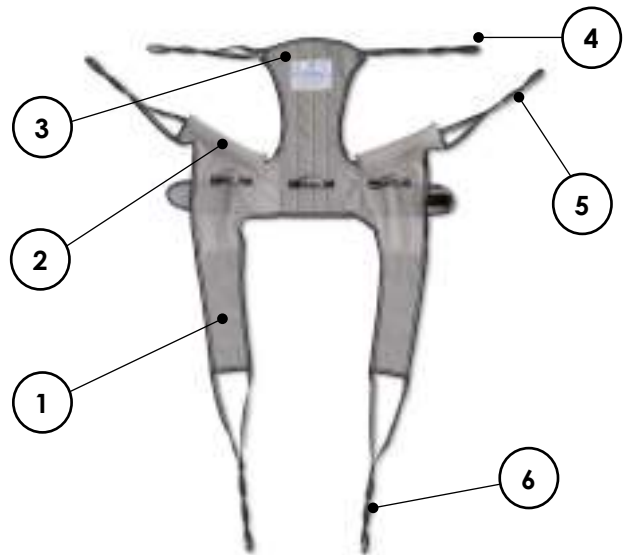
- | | |
|----------------------|-----------------------|
| 1 Thigh section | 4 Head support strap |
| 2 Under-arm supports | 5 Chest support strap |
| 3 Head support | 6 Legs support strap |

4.2.3 Standard clip model TCS23

- | | |
|-----------------|-------------------------------------|
| 1 Thigh section | 4 Snap buckle to block head support |
| 2 Chest section | 5 Chest support clips |
| 3 Head support | 6 Legs support clips |

4.2.4 Hygiene clip model TCH23

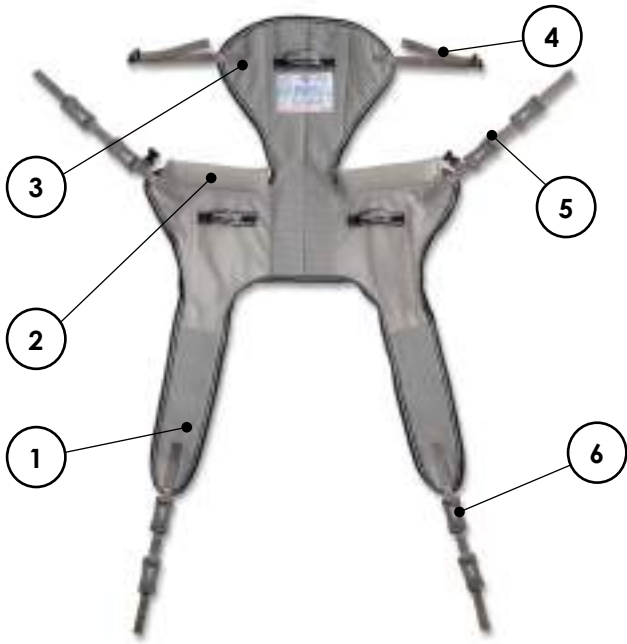- | | | | |
|---|--------------------|---|-----------------------------------|
| 1 | Thigh section | 4 | Snap buckle to block head support |
| 2 | Under-arm supports | 5 | Chest support clips |
| 3 | Head support | 6 | Legs support clips |


4.2.5 Continuous loop model TCC04

- | | | | |
|---|---------------|---|---------------------|
| 1 | Legs section | 4 | Head support strap |
| 2 | Chest section | 5 | Chest support strap |
| 3 | Head support | 6 | Legs support strap |

5. READ LABEL

Read the label carefully to use the product correctly.

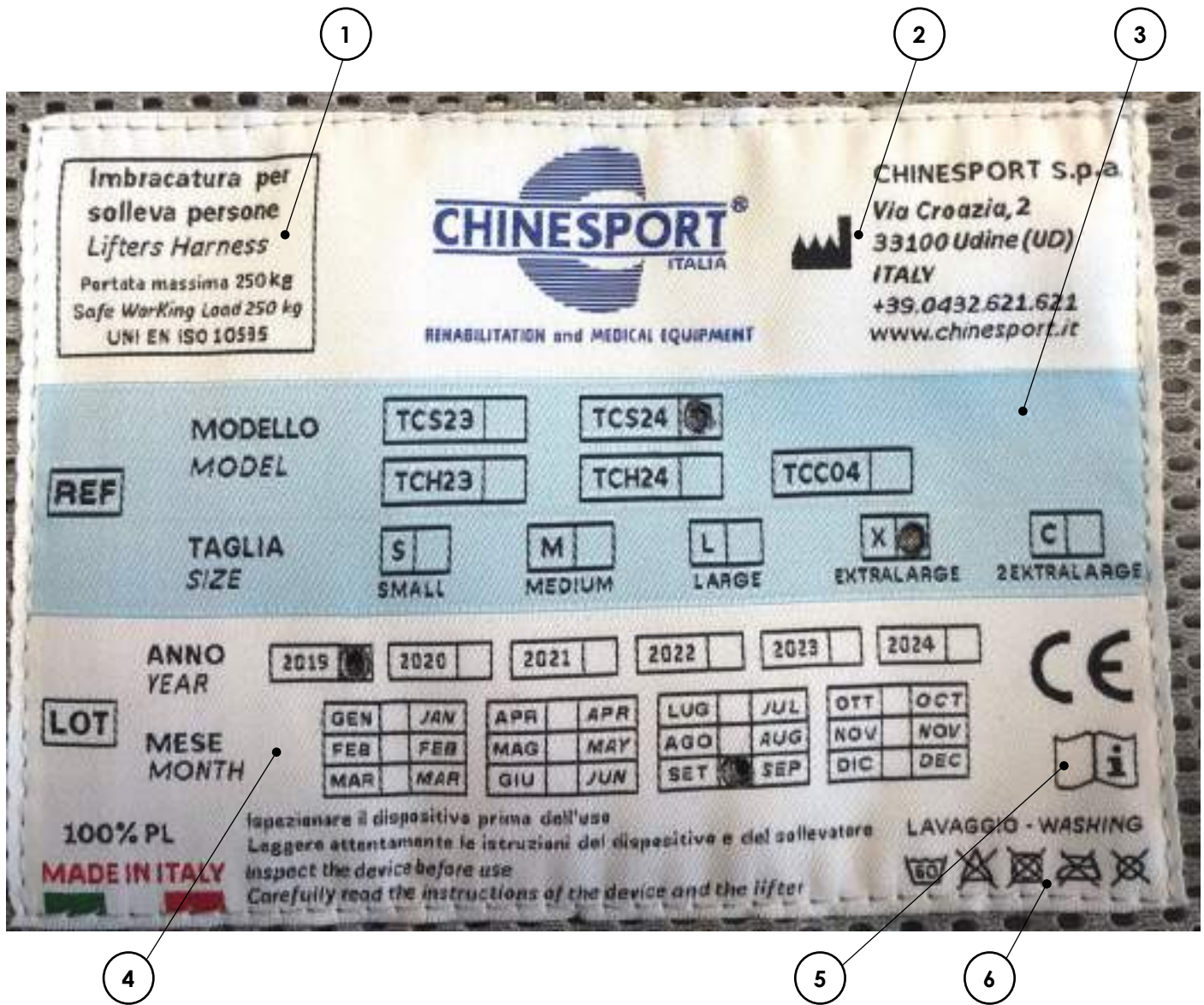1 Maximum permitted working load

4 Production batch obtained with pinhole in adjacent box. In the example, the production batch is September 2019

2 Manufacturer

5 Reading instruction of this manual

3 Refer to the model and size obtained with the pinhole in the adjacent box. In the example, the model is TCS24-X

6 Washing instructions which must be strictly complied with

CODES OF SIZES AVAILABLE

Code	Size	Description
E	E	SMALL
M	M	MEDIUM
L	L	LARGE
X	XL	EXTRA LARGE
C	XXL	2EXTRA LARGE

USE

6.1 4.1 WARNINGS BEFORE USE

Before use, ensure:

the harness is suitable for the person both in size and maximum load
Before each use, inspect the harness, check the fabric, the sewing, the clips or the loops, to check for possible wear, damage or fraying. If worn or damaged, immediately replace the harness and dispose of the old one.
If the red safety sewing on the shoulder loops is loose, do not use the harness: replace it immediately.

6.2 DEVICE USE

Each assistant has his or her own way of applying the harness based on experience and the needs of the person to lift. Guidelines are provided below which take into consideration the safety aspects and can be adapted based on needs, but you should take the principles of safe use into consideration .

Before starting lifting, ensure the person is positioned safely and that the clips or loops are fastened correctly.
Never lift the person higher than is necessary.
During assistance, never leave the person alone in the harness.
For maintenance of the harness, comply with the washing instructions on the label of the harness.

6.2.1 Standard loop harnesses

The loop harness must be used with all lifters which have a tilting lifting bar (spreader bar) with 2 or 4-point suspension that accept loop use:

Step 1

Stand to the rear of the seated person to lift ensuring the harness is held in the right direction, with the label towards the person providing assistance.

From now on, it is important for the seat to have the brake on to prevent sudden movements

Step 2

Slide the harness between the wheelchair seat and the person so the lower edge of the harness is level with the base of the spine. Important: harness insertion is easier if the person leans by themselves or is gently pushed forward.

Step 3

Fold the top part of the harness on the backrest, paying attention to leave the chest and head support straps at shoulder height. Gently pull the legs support so it is between the person and the seat at femur height.

Step 5

Repeat the operation also for the other leg so that both legs support straps are within hand's reach.

Step 4

Pass the legs support under the person's thigh, paying attention to spread the fabric well so it fits comfortably around the under-thigh.

Step 6

Bring the lifter closer or move the chair to the lifter if this is fixed and lower the arm to easily reach the suspension bar hooks (2 or 4-point) with the support straps.

Refer to the lifter use instructions to correctly work it. The photos are for indication purposes only and should be adapted to the type of lifter and the spreader bar in use.

Step 7

Couple the legs support straps and the chest support straps to the hooks. Whether the bar is 2 or 4-point or you decide to cross the legs support straps or not, the important thing is to balance the load. The straps have two sections each to differentiate tilting used to lift the person. These sections are highlighted by the sewing in red and blue.

Step 8

Gradually start lifting and ensure the fabric is properly unfolded and not stuck in the hooks or buttons on the seat or clothing.

Step 9

When the person just about starts to lift off the seat, couple the head support strap if head support is necessary.

Step 10

Continue to lift by holding the reinforced strap with one hand which is immediately over the label to avoid swinging the person caused by removal from the seat.


Once the person no longer touches the seat, its brake can be taken off and moved away. Direct the lifter to the desired point if mobile or manoeuvre it as desired.

Refer to the lifter use instructions to correctly work it.

6.2.2 Standard clip harnesses

The clip harnesses can be used with all the systems including coupling of the clips to speed up the lifting phases, i.e. with lifters equipped with a spreader bar balance type clip coupling that also allows easy tilting management of the lifted person.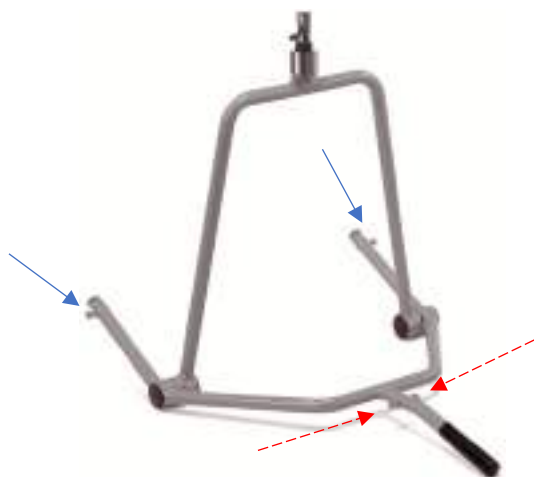

The arrows indicate the point where the clips should be positioned. The rear ones supporting the chest are indicated with a continuous blue line, while the front ones supporting the legs are indicated with a broken red line.

As for the loop support straps which have two sections to differentiate coupling tilt, the clips also have two per strap. The phases to wear the harness are exactly the same as those described for the loop harness, the only difference being coupling takes place by passing the clip through the coupling system and then sliding it until it locks.

Furthermore, the head support system is equipped with snap buckles to couple the support to the tie rods of the chest support:

The snap buckle allows adjustment of head support tilting.

6.2.3 Hygiene loop or clip harnesses

Loop or clip hygiene harnesses are ideal for toileting. They differ from one another in the coupling system with spreader bar, exactly like the standard harnesses. The difference for the wearing phase compared to the standard one is that the chest support tie rods pass under the arms, instead of over the shoulders. A suggested sequence follows.

Step 1

Stand to the rear of the seated person to lift ensuring the harness is held in the right direction, with the label towards the person providing assistance.

Step 2

Slide the harness between the wheelchair seat and the person so the lower edge of the harness is level with the lower back. Important: harness insertion is easier if the person leans by themselves or is gently pushed forward.

Step 3

Pass the padded supports under the armpits and close the snap buckle for better stability.

Step 4

Pass the legs support under the person's thigh, paying attention to spread the fabric well so it fits comfortably around the under-thigh.

Step 5

Repeat the operation also for the other leg so that both legs support and underarm support straps are within hand's reach.

Step 6

Bring the lifter closer and lower the arm to easily reach the suspension bar hooks (2 or 4-point) with the support straps or the coupling points with the clips.

Step 7

Gradually start lifting and ensure the fabric is properly unfolded and not stuck in the hooks or buttons on the seat or clothing. When the person just about starts to lift off the seat, couple the head support strap if head support is necessary. For clip harnesses, couple the snap buckle.

her side. Position the harness with the part of the label turned towards the person.

Step 8

Continue to lift by holding the reinforced strap with one hand which is immediately over the label to avoid swinging the person caused by removal from the seat.

Step 9

At this point, move the chair away and direct the lifter to the desired point.

6.2.4 Lifting from lying down variant

If the person is lying down instead of seated, the harness must be inserted under the person for coupling to the lifter. Follow the guidelines below.

Step 1

Fold the harness in two along the longitudinal axis so the label is outwards and folded exactly in two. Position the harness folded as such along the person's side, who will be positioned, alone or with the help of an assistant, on his or

Step 2

Place the harness as much as possible under the person, sliding the part more in contact with the body.

Step 3

Gently turn the person so he or she is turned on their opposite side and completely lying down with the side over the harness.

Step 6

At this point move the lifter close and put the brake on.

From now on, it is important for the lifter to have the brake on to prevent sudden movements. If it does not have a brake, position it so it cannot move by itself. Refer to the lifter instructions.

Start to couple the chest support straps. If looped, couple the spreader bar on the hooks, while if clipped, slide the hook passing it from the circular opening, as previously shown.

Step 4

Slide out the top part of the harness from under the person so the label is in contact with the mattress.

Step 5

Lay the patient on his or her back and completely lying on the harness. Ensure the fabric is well spread and no folds have formed.

Step 7

Pass the legs support towards the inner legs and couple the legs support tie rods to the spreader bar.

Step 8

Also couple the head support straps and gradually start to lift, paying attention the tie rods all tension together. Accompany the head with your hand.

Step 9

Continue to lift by holding the reinforced strap with one hand which is immediately over the label to avoid swinging the person caused by removal from the bed.

6.2.5 Harness removal

Once the destination has been reached, on the bed or seat, cautiously proceed with the descent phase. A brief guideline follows.

Descent

Approach the chair or bed where the person will be left or bring the seat to the lifter if this is fixed. Bring the lifter to a suitable height to continue descent.

Use with seat

Bring the chair as close as possible and put the brake on.

From now on, it is important for the seat to have the brake on to prevent sudden movements

With one hand, hold the reinforced strap which is immediately over the label to avoid swinging the person and accompany him or her during the descent phase.

Once the person is seated, lower the arm of the lifter until the tie rods loosen. When the tie rods are loose, the spreader bar can be released. If looped, slide out the hooks, while if clipped, release by sliding the hook towards the circular opening. Once the harness is released, you can move the lifter away or move the chair from the lifter to remove it. Proceed in the opposite manner to the wearing phase and then slide off the harness.

Use with bed

Bring the lifter as close as possible to the bed so the person is over the mattress and put the brake on.

From now on, it is important for the lifter to have the brake on to prevent sudden movements. If it does not have a brake, position it so it cannot move by itself. Refer to the lifter instructions

With one hand, hold the reinforced strap which is immediately over the label to avoid swinging the person and accompany him or her during the descent phase on the bed.

Once the person is lying down, lower the arm of the lifter until the tie rods loosen. When the tie rods are loose, the spreader bar can be released. If looped, slide out the hooks, while if clipped, release by sliding the hook towards the circular opening. Once the harness is released, the lifter can be moved away. Proceed in the opposite manner to the wearing phase and then slide off the harness.

6.2.6 Continuous harnesses

The continuous harnesses are suitable for long-time wear. The special fabric is breathable and allows good stability on the chair. The lifting operations are the same, the only difference is that once the person is positioned on the seat, the harness can be left in position, folding the head support on the backrest.

7. MAINTENANCE AND USEFUL LIFE

7.1 INSPECTION

For correct operation of the device, for correct use, for patient and operator safety and to guarantee the performance of the device, scheduled inspection interventions must be carried out.

7.1.1 Inspection after each use

Before each use, check the fabric, the sewing, the clips or the loops, to check for possible wear, damage or fraying. If worn or damaged, immediately replace the harness and dispose of the old one.

7.1.2 Annual inspection

Annual inspection is recommended of the harness to assess its wear by the manufacturer.

If the harness is heavily used and is washed frequently, the harness should be inspected more frequently.

Inspection is carried out by registered technicians authorised by the manufacturer. A report is issued by the technician documenting the inspection took place. If you want to independently carry out the annual inspection, ask the manufacturer for the **harness periodic inspection guide**. For independent inspection, the manufacturer cannot be held in any way liable for damage caused due to incorrect inspection.

7.2 USEFUL LIFE

The useful life of the device is defined at the start of this manual and is 5 years, providing the aforementioned inspection operations are carried out correctly. If the harness is used and washed very frequently, the useful life could drastically reduce.

At the end of its useful life, the device must be decommissioned according to the "disposal" paragraph. As outlined in the "General warnings" paragraph, the manufacturer cannot be held in any way liable for use of the device beyond the useful life established in this manual.

8. CLEANING

Clean the plastic parts of the harness with a damp cloth. If washing is necessary, follow the instructions below:

8.1 WASHING

Comply with the washing instructions on the label of the harness. If these instructions are not complied with, the

harness could be irreparably damaged and can no longer provide safe use, therefore it should be replaced.

Maximum washing temperature of coloured garments. Do not exceed a temperature of 60°C / 140°F

Do not bleach or use bleach-free washing products

Do not dry in dryer

Do not iron

Do not dry clean. No professional dry cleaning. Do not use solvent-based stain removers

9. DISPOSAL

The harnesses do not contain special materials and therefore must be disposed of as items of clothing. If the harness is dirty, caused by organic or infected material, proceed to dispose of it as medical waste.

10. TECHNICAL SPECIFICATIONS

Material	100% Polyester PL
Clips	Polyamide 6, 6AVGR
Patient maximum weight	250kg
Planned useful life	1 – 5 years

11. WARRANTY

11.1 GENERAL CONDITIONS

All Chinesport products are warranted against defects in materials or workmanship for a period of 24 months from the date of sale of the product, except for any exclusions, limitations or conditions defined at the time of delivery of the product.

The warranty is not valid in case of improper use, tampering with the device, abuse or modification of the product or for any use or operation not explicitly mentioned in this manual.

The warranty is not valid if the device has not been correctly maintained and documented in accordance with this manual, or if the instructions regarding storage, cleaning and sanitation are not followed.

The manufacturer is not responsible for any damage or injury or any situation caused by use that does not comply with the instructions outlined and planned in this manual for harnesses and the manuals of the devices used with it.

The manufacturer does not guarantee its products against defects or damages in the presence of extraordinary

conditions such as: natural disasters, unauthorised maintenance and repairs, delivery damage not directly managed by the manufacturer, lack of maintenance, obvious negligence on the part of the user or operator.

12. INSPECTION RECORDS

MODEL	PURCHASE DATE	BATCH OR IDENTIFICATION NUMBER
-------	---------------	--------------------------------

DATE OF INSPECTION	OPERATIONS CARRIED OUT	APPOINTED TECHNICIAN	SIGNATURE	NEXT INSPECTION

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