## HEINE Ophthalmic Examination Lamp

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### Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries).

We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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## Instructions for use

ENGLISH

### HEINE Ophthalmic Examination Lamp

Read these instructions carefully before using the HEINE ophthalmic examination lamp and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

#### Safety information

**Range of applications:** The HEINE ophthalmic examination lamp is used for examination of the eye, particularly the anterior segment.

**Safety in use:** Use only power sources which are designed for medical applications.

#### Use and function

By rotating the ribbed control (1) you can select one of the three circular apertures, a blue filter, or a slit aperture. Use the thumb and forefinger of the hand which holds the instrument to operate the control.

The images formed by the apertures are focused at a distance of 100 mm and have the following dimensions:

Large aperture and blue filter:	10,5 mm dia.
Medium aperture:	6,5 mm dia.
Small aperture:	4 mm dia.
Slit aperture:	10,5 x 0,7 mm

#### Changing the bulb

**Please note:** The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.

- Remove the lamp head from the handle.
- If the lamp is fitted with an automatic connector (2), unscrew this from the lamp head.
- Pull the used bulb out of the lamp head.
- Clean the glass of the new bulb before replacement with a soft cloth.
- If appropriate, screw the automatic connector (2) back onto the lamp head. Re-connect the lamp head to the handle.

#### Care and maintenance

Clean the housing with a mild cleaning agent and soft cloth. To clean the lens, use a cotton wool bud soaked in alcohol and wipe with a spiral movement starting from the centre of the lens.

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