## GARANZIA

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## WARRANTY

This product has **2-YEAR** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get your product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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I Class Medical Device  
according to ECC directive 93/42

### Codes RP737 (2 wheels - 2 tips - 1 button) - RP738 (4 wheels - 1 button)

Thank you for purchasing walkers **MOPEDIA** by Moretti manufactured to support people with movement difficulty. They are realised in lightweight anodized aluminium tubing with padded handles, non-slip rubber tips and casters. They are foldable and height-adjustable every 2,5 cm:

- **RP737** ..... from 82 to 99 cm
- **RP738** ..... from 85 to 103 cm

**To adjust the height:** push the button to adjust the height. Draw out the stick, paying attention to the corresponding holes up to the desired height which should reflect user's needs. Then leave the push button to fix the stick again. Repeat described procedure for each leg, at the end, check that all four legs have the same height. **Maximum loading weight 130 kg.**

**ATTENTION:** use extra caution one wet and slippery surfaces. Beware of electrical cords in way, wet surfaces, rugs, toys or debris, furniture, little steps and gaps in floors. Do not use the walker on stairs or escalators.

Avoid to leave the walker in contact with water for long periods and control its sturdiness periodically. Keep it away from children to avoid any possible risks. Replace rubber tips every time they are worn out.

With your arm extended down, handgrips should be at your wrist height. With your hand placed on the handgrip, your elbow should be flexed by 20° - 30°.

1. To transfer the patient from the wheelchair to the walker, be sure wheels of the wheelchair are locked.
2. To walk with the walker, lift it up and place it comfortably in front of you, making sure legs end up on the floor. Walk carefully forward, one step at a time.

**Use:** Aid for people with movement difficulty.

**Warning:** check and secure all parts were mounted well before the use of the walker. Help the patient, if necessary, during the movement with the walker.

**Self-locking system:** the self-locking function automatically set off when the patient press the handles of the walker, avoiding involuntary loss of balance. To unlock the walker it is necessary to reduce the pressure on the handles, in order to move the walker.

**Maintenance:** the walker does not need any particular maintenance, we suggest you to check periodically the rubber tips wear and the fitting of each part. Replace rubber tips every time they are worn out.

**Cleaning:** to clean the walker, use a damp cloth and a normal detergent or alcohol.

**Warning:** Moretti Spa decline all responsibility for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.

**Disposal:** the product must not be disposed of as urban waste. It must be taken a special local authority separate waste collection.